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
Revisions

BALTSO0191 Version 10.0 Template 4
Inserts

Rev from	Rev to	JOB #
01	02	8877-16

NOTES:

- BD Catalog Number: 260660, 260661, 260640
- Blank (Sheet) Size: Length: 8" Width: 13.38"
- Number of Pages: 24 Number of Sheets: 6
- Page Size: Length: 8" Width: 13.38" Final Folded Size: 8" x 6.69"
- Ink Colors: No. of Colors: 1 PMS#: Black
- Printed two sides: Yes ☒ No ☐
- Style (see illustrations below): # 5

- Vendor Printed ☒ Online/In House Printed ☐ Web ☐
- See specification control no. N/A for material information.
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Label Design	REVISED BY By Nancy Carlsen at 8:46 am, Apr 05, 2017	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION.	 Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA
Proofer	PROOFING APPROVED BY By Natalie Morio at 12:54 pm, Apr 05, 2017		
Checked By	THIRD EYE BY By Nichole Graham at 2:01 pm, Apr 21, 2017		
Part Number:	8801511JAA	Category and Description Package Insert, BD BBL™ Sensi-Disc™ 6-, 8-, and 12-Place Self Tamping Dispensers	Sheet: 1 of 25 Scale: N/A A

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INTENDED USE

12-Place

The 12-Place **BD BBL™ Sensi-Disc™** self-tamping dispenser accommodates up to 12 cartridges of **BD BBL Sensi-Disc** susceptibility test discs. It is used to deliver discs onto 150 mm-style Petri dishes when performing disc susceptibility testing, especially by the standardized method.¹⁻⁴

6- and 8-Place

These **BD BBL Sensi-Disc** self-tamping dispensers accommodate either 6 or 8 cartridges of **BD BBL Sensi-Disc** susceptibility test discs or **BD BBL™ Taxo™** anaerobe differentiation discs. They are used to deliver discs onto 100 mm-style Petri dishes when performing disc susceptibility testing or for the presumptive identification of gram-negative anaerobic bacilli.⁵⁻⁷

WARNINGS AND PROCEDURES

6-, 8-, and 12-Place

For *in vitro* Diagnostic Use. Observe aseptic techniques and established precautions against microbiological hazards throughout all procedures. After use, prepared tubes, specimen containers and other contaminated materials must be sterilized by autoclaving before discarding.

PRINCIPLES OF THE PROCEDURE

12-Place

Discs are dispensed when downward pressure is applied to the operating knob. This unit is designed to provide 33 mm distance between disc centers. No adjusting ring is required to control the height of the dispenser base above the agar surface. The discs are placed in a consistently uniform pattern as long as the depth of the agar is 3 to 5 mm thick and the height of the dish is less than 26 mm. The unit will automatically tamp the discs onto the agar surface and, therefore, eliminates the need for this manual step. Each dispenser is provided complete with interlocking base and cover to protect the unit from moisture when not in use.

6- and 8-Place

Discs are dispensed and tamped onto the agar surface by depressing the dispensing knob on the top of the dispenser. The step adjusting ring is used to control the height of the dispenser base above the agar surface. Each dispenser is provided complete with a storage case to protect the unit from moisture when not in use.

CARE OF THE DISPENSER AND STORAGE CASE

12-Place

1. If desired, the dispenser and protective base and cover may be wiped clean with water, isopropyl alcohol, or a 3% Lysol™ solution. Do not use alkaline cleaning agents, organic solvents, or abrasive cleaners. After cleaning, always air-dry the unit thoroughly before inserting cartridges.
2. Avoid the use of either an excessively cold or warm dispenser. Always allow it to reach room temperature before using. Return loaded dispenser to refrigerator as quickly as possible after use. Do not store in freezer.
3. It is recommended that a desiccant be placed in the dispenser base to help maintain a low-moisture environment. Each cartridge is packed with desiccant capsules that should be replaced each time the cartridges are replaced, or more often, as required. Each dispenser is packed with a desiccant canister that should be regenerated when the silica gel, as viewed through the monitor window on the top of the canister, has turned pink. To regenerate, heat the canister in a vented oven at 300 °F for approximately 3 hours or until the deep blue color returns. Upon cooling, the canister may be reused and will be capable of re-absorbing moisture to its full capacity. Replacement desiccant canisters can also be ordered.

6- and 8-Place

1. If desired, the dispenser and storage case may be wiped clean with water, isopropyl alcohol or a 3% Lysol solution. Do not use alkaline cleaning agents, organic solvents, or abrasive cleaners. After cleaning, always air-dry the unit thoroughly before inserting cartridges.
2. Avoid the use of either an excessively cold or warm dispenser. Always allow it to reach room temperature before using. Return the loaded dispenser to the refrigerator as quickly as possible after use. Do not store in the freezer.
3. It is recommended that the desiccant container, packed with each dispenser, be placed in the dispenser base to help maintain a low moisture environment. The desiccant should be regenerated when the silica gel, as viewed through the monitor window on the top of the container, has turned pink. To regenerate, heat the container in a vented oven at 300 °F for approximately 3 hours or until the deep blue color returns. Upon cooling, the container may be reused and will be capable of reabsorbing moisture to its full capacity.

OPERATING PROCEDURE

12-Place

This dispenser has been carefully manufactured and thoroughly tested. Trouble-free performance is assured by understanding and practicing the correct operating procedures. The following instructions describe the operation of the dispenser and precautions to be observed:

PREPARATION

1. Remove the dispenser from the refrigerator and allow it to reach room temperature.
2. Remove the dispenser from its storage case and place on work surface.
3. Slide the blue knob to the "Lock" position. The dispenser will not operate properly if in the "Unlock" position.
4. Lift the dispenser about an inch and set it down sharply on the work surface. Do this with the cartridges removed at the start of the day's usage or whenever the tampers have become displaced by turning the dispenser upside down. This action assures that the tampers are returned to their operating positions.

DRY CYCLING

1. Lift the dispenser and place it over an empty Petri dish.
2. Depress the knob firmly until the sound of the tampers is heard. (If the dispenser will not dry cycle, tap the side of the unit against the side of the work surface and repeat the procedure.)
3. Allow the knob to return to the upper position.
4. Repeat steps 1, 2, and 3 several times to get the feel of the dispenser.

INSERTING CARTRIDGES

1. Slide the blue knob to the "Unlock" position.
2. Insert the cartridges into the numbered holes and press down lightly.
3. Slide the blue knob to the "Lock" position. If resistance is encountered, the cartridges are not properly seated. To correct, steps 1 and 2 should be repeated.

6- and 8-Place

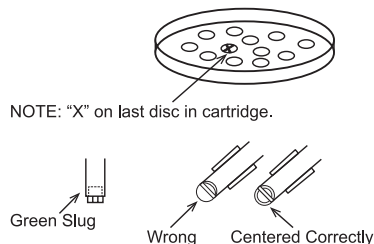
These dispensers are carefully manufactured and thoroughly tested. Trouble-free performance is assured by understanding and practicing the correct operating procedures.

1. Remove the dispenser from the refrigerator and allow it to reach room temperature.
2. Remove the dispenser from the storage case and place on the work surface.
3. Pull out the small black knob located on the side of the dispenser showing the BD name to the furthest point (about 1/2"), and slide the knob down so it locks in place.
4. Insert cartridges.
5. Release the black knob so that it returns to the original position to lock cartridges in the dispenser.

CARTRIDGES

6-, 8-, and 12-Place

The last disc in each cartridge is marked with an "X". A green plastic slug has been incorporated in each cartridge after the "X" marked disc. This prevents the dispenser from operating when an empty cartridge is in place. Watch for the appearance of discs marked "X" and immediately insert full cartridges. When inserting cartridges into the dispenser, make certain that the first disc is centered in the end of the cartridge.

INSTRUCTIONS FOR DISPENSING DISCS

6-, 8-, and 12-Place

1. Lift dispenser and place over the agar plate.
2. Depress knob firmly until the sound of the tampers is heard. Note: Partial depression of the knob may cause discs to jam in the dispenser. (See "CLEARING PROCEDURE.")
3. Allow knob to return to the upper position and lift dispenser from the agar plate.
4. Inspect the agar plate for the "X" on any of the dispensed discs.
5. Replace any empty cartridges.

SPECIAL INSTRUCTIONS

6-, 8-, and 12-Place

The dispenser must always complete a full cycle before it can be operated again. If a full cycle was not completed or if the dispenser has resistance greater than normally encountered in depressing the knob, STOP and immediately begin the "CLEARING PROCEDURE."

CLEARING PROCEDURE

12-Place

1. Reversing the cartridge, slide the blue knob to the "Unlock" position. Remove all the cartridges and reinsert into the dispenser upside down.
2. Place the blue knob in the "Lock" position.
3. Carefully inspect the end of each cartridge for the presence of a green plastic slug or deformed discs. The green slug indicates that the cartridge is empty and must be replaced. Deformed discs must be manually removed and discarded.
4. With the cartridges upside down in the dispenser, dry cycle the dispenser several times to assure that it is operating freely. This completes the "CLEARING PROCEDURE."

Occasionally, a disc will be delivered on edge and embedded in the agar; immediately remove the upright disc from the medium with sterile forceps and lay it flat on the plate centered over the point of initial contact. Horizontally embedded discs may occasionally occur. This could affect the accurate measurement of the zone area. Therefore, it is recommended that the test be repeated before accepting a minimally intermediate or minimally susceptible result.

If a disc is embedded in the medium, decontamination of the tamper is recommended. If no disc is delivered, or if a disc is delivered on edge, the tamper may be contaminated; to determine if tamper contamination has occurred, carefully examine the medium around the disc for tamper marks. If any marks are noted, it must be assumed that the tamper has become contaminated and the tamper should be decontaminated by following the "CLEANING AND DECONTAMINATION PROCEDURE."

6- and 8-Place

1. Pull out the small black knob located on the side of the dispenser to the furthest point and slide the knob down so that it locks in place.
2. Remove all cartridges.
3. Lift the dispenser and place over an empty Petri dish.
4. Cycle the dispenser several times as described in steps 6 through 8, under "Operating Procedures."
5. Replace cartridges.

CLEANING AND DECONTAMINATION PROCEDURE

Each time that cartridges are changed, or if a tamper becomes contaminated during use by touching the inoculated agar surface, clean and decontaminate the tamper(s) by the following procedure:

12-Place

1. Remove all **BD BBL Sensi-Disc** cartridges from the unit and replace the lock knob in the "Lock" position.
2. Place a **BD BBL Sensi-Disc** cartridge upside down in the dispenser in the position that is contaminated; or, for periodic cleaning, place cartridges upside down in all positions.
3. Fill one empty 150 mm style Petri dish with 65 to 70 mL of a 3% solution of Lysol.
4. Fill one empty 150 mm style Petri dish with 65 to 70 mL of an 85% solution of isopropyl alcohol.
5. Fill two empty 150 mm style Petri dishes with 65 to 70 mL of sterile distilled water.
6. Place the dispenser over the dish containing Lysol solution.
7. Push the operating knob down until the tamper(s) are released. Hold the knob in the depressed condition for 30 seconds.
8. Repeat steps 6 and 7 with the isopropyl alcohol solution.
9. Repeat steps 6 and 7 with each of the dishes of distilled water. This will help ensure that the cleaning agents have been completely removed.

6- and 8-Place

1. Remove all cartridges from the dispenser.
2. Fill one empty 100 mm-style Petri dish with 20–25 mL of a 3% solution of Lysol.
3. Fill one empty 100 mm-style Petri dish with 20–25 mL of an 85% solution of isopropyl alcohol.

4. Fill two empty 100 mm-style Petri dishes with 20–25 mL of sterile purified water.
5. Place the dispenser over the dish containing Lysol solution.
6. Push the dispensing knob down until the tampers are released. Hold the knob in the depressed condition for 30 sec.
7. Repeat steps 5 and 6 with the isopropyl alcohol solution.
8. Repeat steps 5 and 6 with each of the dishes of purified water. This will help ensure that the cleaning agents have been completely removed.

The tampers have now been cleaned and decontaminated. Any remaining alcohol and water left on the tampers will evaporate. The unit may be stored, or if testing has not been completed, dry the tampers by releasing them several times (depressing the knob fully) while the unit is placed over a sterile towel, gauze pad, swab, or other sterile absorbent material. The tampers can also be allowed to air dry, but this will require additional time, depending on the temperature and relative humidity in the area in which the dispenser is being stored.

TROUBLESHOOTING GUIDE

6-, 8-, and 12-Place

Read the insert before operating any **BD BBL Sensi-Disc** Dispenser.

If one or more discs fail to dispense:

1. Ensure the operating knob is UP.
2. Ensure the lock knob is on "Unlock."
3. Examine for empty or damaged cartridges.
4. Examine for loose discs.

To clear loose or jammed discs:

1. Remove cartridges.
2. Set lock knob on "Lock."
3. Push dispensing knob several times.
4. Examine bottom for trapped discs; if present, place inverted cartridges in appropriate positions and operate dispenser over a sterile surface.

SERVICE AND MAINTENANCE INFORMATION

6-, 8-, and 12-Place

Should any malfunction occur that cannot be corrected by the "CLEARING PROCEDURE," "CLEANING AND DECONTAMINATION PROCEDURE," or "TROUBLESHOOTING GUIDE," in the United States, telephone Technical Services for instructions at 1.800.638.8663. For International locations, please contact your local BD Representative.

REFERENCES

1. Bauer, A.W., W.M.M. Kirby, J.C. Sherris, and M. Turck. 1966. Antibiotic susceptibility testing by a standardized single disk method. *Am. J. Clin. Pathol.* 45: 493–496.
2. Ryan, K.J., F.D. Schoenkecht, and W.M.M. Kirby. 1970. Disc sensitivity testing. *Hospital Practice.* 5: 91–100.
3. Food and Drug Administration. Standardized disc susceptibility test. Section 460.1, Title 21 of the Code of Federal Regulations.
4. National Committee for Clinical Laboratory Standards. 1997. Approved Standard M2-A6, Performance standards for antimicrobial disk susceptibility tests, 6th ed. National Committee for Clinical Laboratory Standards, Wayne, Pa.
5. National Committee for Clinical Laboratory Standards. 2003. Approved Standard: M2-A8. Performance standards for antimicrobial disk susceptibility test, 8th ed. NCCLS, Wayne, Pa.
6. Jousimies-Somer, H.R., P.H. Summanen, and S.M. Finegold. 1999. *Bacteroides*, *Porphyromonas*, *Prevotella*, *Fusobacterium*, and other anaerobic gram-negative rods and cocci, p. 690–711. In P.R. Murray, E.J. Baron, M.A. Pfaller, F.C. Tenover, and R.H. Tenover (ed.), *Manual of clinical microbiology*, 7th ed. American Society for Microbiology, Washington, D.C.
7. Summanen, P., E.J. Baron, D.M. Citron, C.A. Strong, H.M. Wexler, and S.M. Finegold. 1993. *Wadsworth anaerobic bacteriology manual*, 5th ed. Star Publishing Company, Belmont, Calif.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

BD BBL Sensi-Disc 6-, 8-, and 12-Place Self-Tamping Dispensers

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For US: "For Investigational Use Only"

Lower limit of temperature / Долен лимит на температурата / Dolni hranice teploty / Nedre temperaturgränse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inférieure de température / Najnižja dovoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температурный төменгі рұқсат шеіі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurilimiet / Nedre temperaturgränse / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sıcaklık alt sınırı / Минимальна температура / 温度下限

CONTROL

Control / Контроль / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Control / Contrôle / Kontroll / Бақылау / 컨트롤 / Kontrolė / Kontrolle / Control / Control / Контроль / kontroll / Контроль / 对照

CONTROL +

Positive control / Положительный контроль / Pozitívny kontrola / Positiv kontrol / Positive Kontrolle / Οστικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitivna kontrola / Pozitiv kontrol / Controllo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívá kontrolė / Positive controle / Kontrola dodatnia / Controllo positivo / Control pozitiv / Положительный контроль / Pozitif kontrol / Позитивный контроль / 阳性对照试剂

CONTROL -

Negative control / Отрицательный контроль / Negatívny kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatieve kontroll / Contrôle négatif / Negativna kontrola / Negativ kontrol / Controllo negativo / Негативтік бақылау / 음성 컨트롤 / Neigiama kontrolė / Negatívá kontrolė / Negative controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативный контроль / 阴性对照试剂

STERILE EO

Method of sterilization: ethylene oxide / Метод на стерилизация: этиленов оксид / Zpusob sterilizace: Ethenoxid / Steriliseringmethode: ethylenoxid / Sterilisationsmethode: Ethenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseringismetode: etileenoksid / Méthode de stérilisation: oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszer: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Sterilizacijos būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksid / Gesteriliseerd met behulp van ethylenoxide / Steriliseringmetode: etylenoksid / Metoda sterylizacji: tlenek etylu / Método de esterilização: óxido de etileno / Metoda de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metoda sterilizacije: etylenoksid / Steriliseringmetode: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: этиленоксидом / 灭菌方法: 环氧乙烷

STERILE R

Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Zpusob sterilizace: záření / Steriliseringmethode: bestraling / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseringismetode: kiirgus / Méthode de stérilisation: irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszer: besugárzás / Metodo di sterilizzazione: irradiazione / Sterilizacijos būdas: spinduliuotė / Metoda de sterilizare: iradiere / Metoda sterylizacji: promienianie / Método de esterilização: irradiação / Metoda de sterilizare: iradiere / Метод стерилизации: облучение / Metoda sterilizacije: ožiarenie / Metoda sterilizacije: ozračavanje / Steriliseringmetode: stråling / Sterilizasyon yöntemi: ırradyasyon / Метод стерилизації: опромінення / 灭菌方法: 辐射

Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologisk risk / Biologisk far / Rischio biologico / Биологиялык төуеккерлер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologisk risiko / Biologisk risiko / Zagrozenia biologiczne / Perigo biológico / Riscuiri biologice / Биологическая опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyolojik Riskler / Биологічна небезпека / 生物学风险

Caution, consult accompanying documents / Внимание, направте справка в придружаващите документи / Pozor! Pristudujte si příloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precución, consultar la documentación adjunta / Ettevaatus! Lugea kaasevat dokumentatsiooni / Attention, consulter les documents joints / Uptozorente, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztató! / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dmesio, žiurkite pridėdamus dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Nalezaj zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atentie, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, prilike verlen belgelerle başyurun / Увага: див. супутню документацию / 小心, 请参阅附带文档。

Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgränse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dovoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температурный рұқсат етінен жоғары шеіі / 상한 온도 / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurilimiet / Øvre temperaturgränse / Gorna granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sıcaklık üst sınırı / Максимальна температура / 温度上限

Keep dry / Пазете сухо / Skladujte v suchém prostředí / Orbevares tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / 건조 상태 유지 / Laiykite sausi / Uzglabāt sausu / Droog houden / Hoides tørt / Przechowywać w stanie suchym / Uchovávať v suchu / Držite na suvom mestu / Förvaras tørt / Kuru bir şekilde muhafaza edin / Бергетти від вологи / 請保持干燥

Collection time / Време на събиране / Cas odbrú / Opsamlingstidspunkt / Entnahmehszeit / Όρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жынау уакыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de coleta / Ora colectării / Время сбора / Doba odberu / Vreme prikupljanja / Uppsamlingstid / Toplama zamanı / 4 час забору / 采集时间

Peel / Обелете / Otevřete zde / Äbn / Abziehen / Αποκολλήστε /
Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare /
Ћстинги қабатын алып таста / 벗기기 / Pléști ăla / Altimēt / Schillen / Trekk
av / Oderwać / Destacar / Se deszipeste / Otkleitje / Otdrhnite / Oljtušiti /
Dra isår / Айрма / Відкрити / 撕下

Perforation / Перфорация / Perforace / Perforering / Διτρήση /
Perforación / Perforasjon / Perforacija / Perforálás / Perforazione /
Тесик тесу / 찢히신 / Perforacija / Perforacija / Perforatie / Perforacja /
Perfuração / Perfore / Перфорация / Perforacia / Perforasyon /
Перфорация / 穿孔

Do not use if package damaged / He използвайте, ако опаковката е
повредена / Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis
emballagen er beskadiget / Inhal beschädigter Packungnicht verwenden
/ Μη χρησιμοποίητε εάν η συσκευασία έχει υποστεί ζημιά. / No usar si
el paquete está dañado / Mitte kasutada, kui pakend on kahjustatud /
Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako
je oštećeno pakiranje / Ne használja, ha a csomagolás sérült / Non
usare se la confezione è danneggiata / Егер пакет бұзылған болса,
пайдаланба / 패키지가 손상된 경우 사용 금지 / Jei pakuotė pažeista,
nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de
verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie
używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem
estiver danificada / A nu se folosi dacă pachetul este deteriorat / He
использовать при повреждении упаковки / Nepoužívejte, ak je obal
poškozený / Ne koristite ako je pakovanje oštećeno / Använd ej om
förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / He
використовувати за пошкодженної упаковки / 如果包装破损, 请勿使用

Keep away from heat / Пазете от топлина / Nevystavujte přilišnému
teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida
eemal valgestest / Protéger de la chaleur / Držati dalje od izvora topline /
Övja a melegtől / Tenere lontano dal calore / Сақын жерде сақта / 열을
피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargăt no karstuma /
Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z
dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / He
närpegav / Uchovávať mimo zdroja tepla / Držite dalje od toplote / Får ej
utsättas för värme / Isridan uzak tutun / Бергити від дії тепла / 请远离热源

Cut / Крежете / Odstrňňte / Klip / Schneiden / Κόψτε / Cortar / Lõigata /
Découper / Rezi / Vágja ki / Tagliare / Keciňz / 잘라내기 / Kiprti / Nogrietz /
Knippen / Kutt / Odciać / Cortar / Decupaj / Отрезать / Odstrňňte / Iseći /
Klipp / Kesme / Розрізати / 剪下

Collection date / Дата на събиране / Datum odběru / Opsamlingsdato
/ Entnahmdatum / Ημερομηνία συλλογής / Fecha de recogida /
Kogumiskupäev / Date de prélèvement / Dani prikupljanja / Mintavétel
dátuma / Data di raccolta / Жинаған тізбекүні / 수집 날짜 / Paemimo
data / Savākšanas datums / Verzameldatum / Data prøvetaking / Data
pobrania / Data de colheita / Data colectării / Дата сбора / Datum odboru
/ Datum prikupljanja / Uppsamplingsdatum / Toplama tarihi / Дата забору
/ 采集日期

µL/test / µL/rect / µL/Test / µL/εξέταση / µL/prueba / µL/teszt / µL/테스트 /
мкЛ/тест / µL/tyrimas / µL/pārbaude / µL/teste / мкЛ/анализ / µL/检测

Keep away from light / Пазете от светлина / Nevystavujte světlu / Må ikke
udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως /
Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de
la lumière / Držati dalje od svetla / Fény nem érheti / Tenere al riparo dalla
luce / Қазақстаннан жерде ұста / 빛을 피해야 함 / Laikyti atokiau nuo
šilumos šaltinių / Sargăt no gaismas / Niet blootstellen aan zonlicht / Må
ikke utsettes for lys / Przechowywać z dala od źródeł światła / Manter ao
abrigo da luz / Feriti de lumină / Хранить в темноте / Uchovávať mimo
dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Işıktan
uzak tutun / Бергити від дії світла / 请远离光线

Hydrogen gas generated / Образован е водород газ / Možnost úniku
plynného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt
/ Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno /
Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadrží hydrogen
vodík / Hidrogén gázt fejeleszt / Produzione di gas idrogeno / Газтектер
cyteri naída болды / 수소 가스 생성됨 / Išskiria vandenilio dujas /
Rodas ūdegradis / Waterstofgas gegenereerd / Hydrogengass generert
/ Powoduje powstawanie wodoru / Produção de gás de hidrógeno /
Generare gaz de hidrogen / Выделение водорода / Vyrobené použitím
vodíka / Oslobada se vodonik / Genererad vätgas / Açığa çıkan hidrojen
gazı / Реакция з виділенням водню / 会产生氢气

Patient ID number / ИД номер на пациента / ID pacienta / Patients
ID-nummer / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de
ID del paciente / Patsiendi ID / No d'identification du patient / Identifikációs
broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттин
идентификациялык нөмірі / 환자 ID 번호 / Paciento identifikavimo
numeris / Pacienta ID numurs / Identificationnummer van de patiënt /
Pasiensens ID-nummer / Numer ID pacienta / Número da ID do doente /
Număr ID pacient / Идентификационный номер пациента / Identifikačné
číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarası /
Идентификатор пациента / 患者标识号

Fragile, Handle with Care / Чупливо, Работете с необходимото
внимание. / Kéhekké. Pfi manipulaci posturujte opatně. / Forsigtig, kan gå
i stykker. / Zerbrechlich, vorsichtig handhaben. / Ευθραστο. Χειριστείτε
το με προσοχή. / Frágil. Manipular con cuidado. / Örn, käsitlege
ettevaatlikult. / Fragile. Manipuler avec précaution. / Lomjivo, rukujte
pazljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura.
/ Сыныш, абаканп пайданыңыз. / 조심 깨지기 쉬운 처리 / Trapu,
elkites atsargiai. / Trausis; rikoties uzmanīgi / Breekbaar, voorzichtig
behandelen. / Ömtålig, händter försiktigt. / Krucha zawartość, przenosić
ostrożnie. / Frágil, Manuseie com Cuidado. / Frágil, manipulat cu atenție.
/ Χρηστέο! Обращаться с осторожностью. / Krehké, vyžaduje sa opatrná
manipulácia. / Lomjivo - rukujte pažljivo. / Bräckligt. Hantera försiktigt. /
Kolay Kırılır, Dikkatli Taşın. / Тендітна, звертатися з обережністю / 易
碎, 小心轻放

WARRANTY INFORMATION

6-, 8-, and 12-Place

Becton Dickinson extends a two-year warranty for the **BD BBL Sensi-Disc** Dispenser from date of receipt. This warranty entitles the holder to repair or obtain a replacement at no charge if found to be defective as a result of normal wear. To obtain repair or replacement within the terms of this warranty, in the United States, please telephone Technical Services for instructions at 1.800.638.8663. For International locations, please contact your local BD Representative.

This warranty is null and void if the **BD BBL Sensi-Disc** Dispenser has been disassembled or subjected to misuse or abuse. Becton Dickinson makes no warranty expressed or implied except as stated above.

To activate this warranty, please complete and mail the Business Reply Card within 10 days of receipt of your **BD BBL Sensi-Disc** Dispenser. If the Business Reply Card is not returned, the warranty will expire two years after the date of manufacture.

Complete this section for your records. Serial Number: _____ Purchase Date: _____

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Lysol is a trademark of Lehn and Fink Products, Division of Sterling Drug, Inc.
© 2016 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.
Benex Limited