NAMJENA

BD BBL Coagulase Plasma, Rabbit (plazma s koagulazom, zečja) i BD BBL Coagulase Plasma, Rabbit with EDTA (plazma s koagulazom, zečja s EDTA-om) koriste se za kvalitativno određivanje patogenosti stafilocoka pomoću izravne metode u epruveti.

SAŽETAK I OBJAŠNJENJE

Identifikacija stafilocoka temelji se na mikroskopskom ispitivanju, morfologiji kolonija, karakteristikama kulture i biokemijskim karakteristikama. Stafilocoki vezani uz akutne upale (*Staphylococcus aureus* kod ljudi i *S. intermedius* i *S. hyicus* kod životinja) mogu zgrušati plazmu. Najpoznatiji i opće prihvaćen kriterij identifikacije tih patogenih organizama temelji se na prisutnosti enzima koagulaze.¹ Loeb² je 1903. prvi zabilježio sposobnost bakterije *Staphylococcus* za stvaranje koagulaze.

Koagulaza vezuje fibrinogen u plazmi, što uzrokuje aglutinaciju organizama ili zgrušavanje plazme. Postoje dvije vrste koagulaze, slobodna i vezana. Slobodna koagulaza izvanstanični je enzim koji se stvara kada se organizam uzgaja u bujonu. Vezana koagulaza, poznata kao faktor grušanja, ostaje na staničnoj stijenci organizma. Ispitivanjem u epruveti može se ustanoviti prisutnost vezane i slobodne koagulaze. Kod izolata koji ne stvaraju faktor zgrušavanja mora se ispitati sposobnost stvaranja izvanstanične koagulaze (slobodne koagulaze).

Za izravno ispitivanje u epruveti preporučuje se plazma s koagulazom BD BBL, zečja i plazma s koagulazom BD BBL, zečja s EDTA-om. Inokulum za ispitivanje mora biti čist jer kontaminirajuće tvari mogu dati lažne rezultate nakon duže inkubacije. U ispitivanju koagulaze, plazma s koagulazom BD BBL, zečja s EDTA-om ima prednost pred plazmom s citratima jer kod organizama koji koriste citrate, kao što je vrsta *Pseudomonas*, *Serratia marcescens*, *Enterococcus faecalis* i sojevi bakterije *Streptococcus*, nakon 18 h dolazi do zgrušavanja plazme s citratima.³

NAČELA POSTUPKA

S. aureus stvara dvije vrste koagulaze, slobodnu i vezanu. Slobodna koagulaza izvanstanični je enzim koji se stvara kada se organizam uzgaja u bujonu. Vezana koagulaza, poznata kao faktor grušanja, ostaje na staničnoj stijenci organizma.

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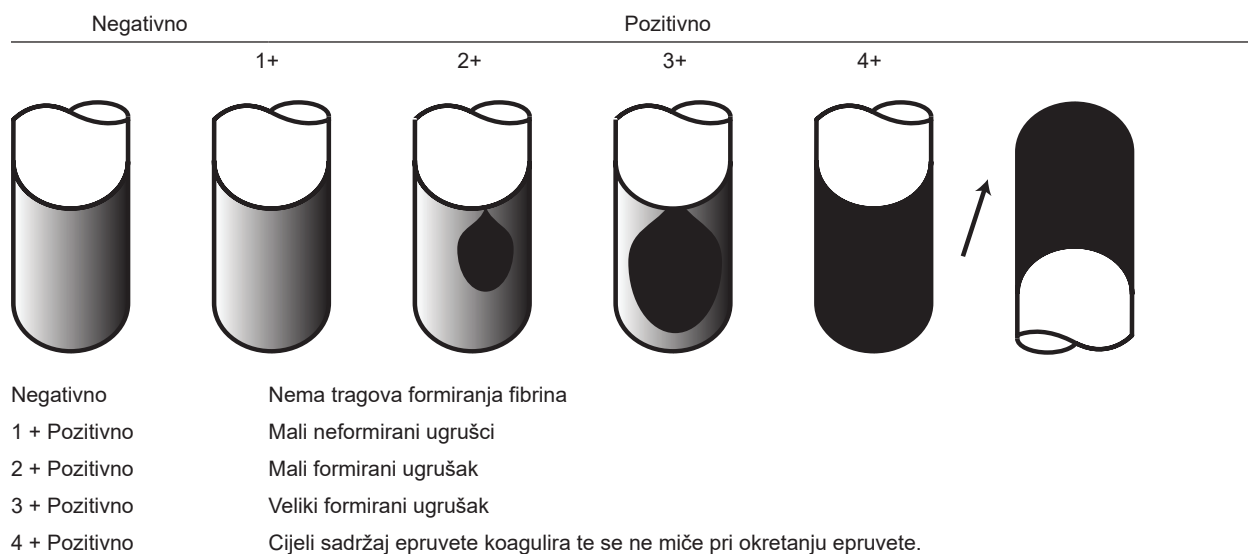
DOSTUPNOST

Kat. br. Opis

- 240658 BD BBL Coagulase Plasma, Rabbit, 10 X 3,0 ml
240661 BD BBL Coagulase Plasma, Rabbit, 10 X 15,0 ml
240827 BD BBL Coagulase Plasma, Rabbit with EDTA, 10 X 3,0 ml
240826 BD BBL Coagulase Plasma, Rabbit with EDTA, 10 X 15,0 ml

REFERENCE

1. Kloos, W. E., and T. L. Bannerman. 1999. *Staphylococcus* and *Micrococcus*, p. 264-282. In P.R. Murray, P.R., E.J. Baron, M.A. Pfaller, F.C. Tenover and R.H. Tenover, Manual of clinical microbiology, 7th ed. American Society for Microbiology, Washington, D.C.
2. Loeb, L. 1903. The influence of certain bacteria on the coagulation of the blood. J. Med. Res. 10:407-419.
3. Bayliss, B.G. and E.R. Hall. 1965. Plasma coagulation by organisms other than *Staphylococcus aureus*. J. Bacteriol. 89:101-104.
4. Pezzlo, M. (ed.). 1994. Aerobic bacteriology, p. 1.0.0.-1.20.47. In H. D. Isenberg (ed.), Clinical microbiology procedures handbook, vol. 1. American Society for Microbiology, Washington, D.C.
5. Baron, E.J., L.R. Peterson and S.M. Finegold. 1994. Bailey & Scott's diagnostic microbiology, 9th ed. Mosby-Year Book, Inc., St. Louis, MO.
6. Association of Official Analytical Chemists. 2000. Official methods of analysis of AOAC International, 17th ed. AOAC International, Arlington, VA.
7. Association of Official Analytical Chemists. 2001. FDA Bacteriological analytical manual online. <<http://www.cfsan.fda.gov/~ebam/bam-mm.html>>.
8. Downes, F.P. and K. Ito (ed.). 2001. Compendium of methods for the microbiological examination of foods, 4th ed. American Public Health Association, Washington, D.C.
9. Flowers, R.S., W. Andrews, C.W. Donnelly and E. Koenig. 1993. Pathogens in milk and milk products, p. 103-212. In R.T. Marshall (ed.), Standard methods for the examination of dairy products, 16th ed. American Public Health Association, Washington, D.C.
10. Luijendijk, A., A. van Belkum, H. Verbrugh and J. Kluytmans. 1996. Comparison of five tests for identification of *Staphylococcus aureus* from clinical samples. J. Clin. Microbiol. 34:2267-2269.
11. McDonald, C.L. and K. Chapin. 1995. Rapid Identification of *Staphylococcus aureus* from blood culture bottles by a classic 2-hour tube coagulase test. J. Clin. Microbiol. 33:50-52.

Tehnički servis i podrška: obratite se lokalnom predstavniku tvrtke BD ili posjetite bd.com.

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Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:

Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

ATCC® is a trademark of the American Type Culture Collection.

BD, the BD Logo, BACTEC, Bacto, BBL, and Trypticase are trademarks of Becton, Dickinson and Company or its affiliates.

© 2019 BD. All rights reserved.