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BD BBL PYR Reagent Droppers anvendes i den hurtige tentative identifikation af gruppe A β -hæmolytiske streptococci og gruppe D enterococci.^{1,2}

RESUMÉ OG FORKLARING

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Kat. nr. **Beskrivelse**
261196 **BD BBL** PYR Reagent Droppers, 50.

LITTERATUR

1. Gordon, D.B., P.C. DeGirolami, S. Bolivar, G. Karafotias, and K. Eichelberger. 1988. A comparison of the identification of group A streptococci and enterococci by two rapid pyrrolidonyl aminopeptidase methods. *Am. J. Clin. Pathol.* 90:210-212.
2. Facklam, R.R., L.G. Thacker, B. Fox, and L. Eriquez. 1982. Presumptive identification of streptococci with a new test system. *J. Clin. Microbiol.* 15:987-990.
3. Ellner, P.D., D.A. Williams, M.E. Hosmer, and M.A. Cohenford. 1985. Preliminary evaluation of a rapid colorimetric method for the presumptive identification of group A streptococci and enterococci. *J. Clin. Microbiol.* 22:880-881.
4. Baron, E.J., and S.M. Finegold. 1990. *Bailey and Scott's diagnostic microbiology*, 8th ed. The C.V. Mosby Co., St Louis.
5. Balows, A., W.J. Hausler, Jr., K.L. Herrmann, H.D. Isenberg, and H.J. Shadomy (ed.) 1991. *Manual of clinical microbiology*, 5th ed. American Society for Microbiology, Washington, D.C.
6. Facklam, R.R. 1987. Specificity study of kits for detection of group A streptococci directly from throat swabs. *J. Clin. Microbiol.* 25:504-508.
7. Lawrence, J., D.M. Yajko, and W.K. Hadley. 1984. Comparison of Strep-A-Check (a 10 min chromogenic substrate test) with bacitracin for presumptive identification of *Streptococcus pyogenes* (group A strep). Program Abstr. 24th Intersci. Conf. Antimicrob. Agents Chemoth., abstr 19.
8. Godsey, J., R. Schulman, and L. Eriquez. 1981. The hydrolysis of L-pyrrolidonyl-β-naphthylamide as an aid in the rapid identification of *Streptococcus pyogenes*, *S. aviium*, and group D-enterococci. Abstr. C-84, p. 276. Abstr. 81st Annu. Meet. Am. Soc. Microbiol. 1981.
9. Bosley, G.S., R.R. Facklam, and D. Grossman. 1983. Rapid identification of enterococci. *J. Clin. Microbiol.* 18:1275-1277.
10. Oberhofer, T.R. 1986. Value of the pyrrolidonyl-β-naphthylamide hydrolysis test for identification of select gram-positive cocci. *Diagn. Microbiol. Infect. Dis.* 4:43-47.
11. Data on file, BD Diagnostic Systems.

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For US: "For Investigational Use Only"

Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuriipiir / Limite inférieure de température / Najnižja dovoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrænse / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sicaklık alt sınırı / Мінімальна температура

Control / Контролно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Controllo / Бақылау / Kontrolé / Kontrole / Controle / Controllo / Контроль / kontroll / Контроль

Positive control / Положителен контрол / Pozitivní kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitivna kontrola / Positiv kontroll / Controllo positivo / Оң бақылау / Teigiamą kontrolę / Pozitívá kontrolle / Positive controle / Kontrola dodatnia / Controllo positivo / Control pozitiv / Положительный контроль / Pozitif kontrol / Позитивный контроль

Negative control / Отрицателен контрол / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negativeve kontroll / Contrôle négatif / Negativna kontrola / Negativ kontroll / Controllo negativo / Негативтік бақылау / Neigiamą kontrolę / Negatívá kontrolle / Negativeve controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативный контроль

Method of sterilization: ethylene oxide / Метод на стерилизация: этиленов оксид / Způsob sterilizace: etylenoxid / Steriliseringmethode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseerimismetode: etüleenoksiid / Méthode de stérilisation: oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Стерилизация әдісі – этилен тотығы / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksīds / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringmethode: etylenoksid / Metoda sterylizacji: tlenek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metóda sterilizácie: etylénoxid / Metoda sterilizacije: etilen oksid / Steriliseringsmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: этиленоксидом

Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Způsob sterilizace: záření / Steriliseringmethode: bestråling / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseerimismetode: kiirgus / Méthode de stérilisation: irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metodo di sterilizzazione: irradiazione / Стерилизация әдісі – сәуле тисі / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringmethode: bestråling / Metoda sterylizacji: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: radiere / Метод стерилизации: облучение / Metóda sterilizácie: ožiarovanie / Metoda sterilizacije: ozračevanje / Steriliseringsmetod: strålning / Sterilizasyon yöntemi: irradyasyon / Метод стерилизації: опроміненням

Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Bioloogilised riskid / Risques biologiques / Biološki rizik / Biológiai veszélyes / Rischio biologico / Биологиялық тәуекелдер / Biologinis pavojus / Biologiskie riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyolojik Riskler / Биологічна небезпека

Caution, consult accompanying documents / Внимание, направьте справка в придружащите документи / Pozor! Pristudujte si příloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatust! Lugeka kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Urozorenje, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / Dėmesio, žiūrėkite pridėdamus dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацію

Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuriipiir / Limite supérieure de température / Gornja dovoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrænse / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Övre temperaturgräns / Sicaklık üst sınırı / Максимальна температура

Keep dry / Пазете сухо / Skladujte v suchém prostředí / Opbevares tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / Laikykite sausiai / Uzglabāt sausu / Droog houden / Holdes tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávať vo suchu / Držite na suvom mestu / Förvaras tørt / Kuru bir şekilde muhafaza edin / Берегти від вологи

Collection time / Време на събиране / Čas odběru / Opsamlingstidspunkt / Entnahmehurzeit / Ώρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жинау уақыты / Paémimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de colheita / Ora colectării / Время сбора / Doba odboru / Vreme prikupljanja / Uppsamlingstid / Toplama zamanı / Час забору

Peel / Обелете / Otevířte zde / Abn / Abziehen / Αποκολλήστε / Desprendre / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Ўстїгнї қабатын алып таста / Plești ăia / Atlīmēt / Schillen / Trek av / Oderwać / Destacar / Se dezlipeste / Отклеить / Odkrhnite / Oljuštiti / Dra isār / Ayırma / Відклеїти

Perforation / Перфорация / Perforace / Perforering / Διάτρηση / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Тесик тесу / Perforacija / Perforăcija / Perforatie / Perforacja / Perfuração / Perforare / Перфорация / Perforácia / Perforasyon / Перфорация

Do not use if package damaged / Не използвайте, ако опаковката е повредена / Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhal beschädigtet Packungnicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιό. / No usar si el paquete está dañado / Mitte kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használja, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Erep naket бүзылган болса, пайдаланба / Jei pakuotė pažeista, nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Не использовать при повреждении упаковки / Nepoužívejte, ak je obal poškozený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки

Keep away from heat / Пазете от топлина / Nevystavujte přilísnému teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgusest / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Salқын жерде сақта / Laikyti atokiau nuo šilumos šaltinių / Sargāt no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / Не нагревать / Uchovávať mimo zdroja tepla / Držite dalje od toplote / Får ej utsättas för värme / Isidan uzak tutun / Берегти від дії тепла

Cut / Срежете / Odstřihněte / Klip / Schneiden / Κόψτε / Cortar / Lõigata / Découper / Reži / Vágja ki / Tagliare / Kesiñiz / Kirpti / Nogriez / Knippen / Kutt / Odciąć / Cortar / Decupați / Отрежьте / Odstrihnite / Iseći / Klipp / Kesme / Подрізати

Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmedatum / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuupäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жинаған тізбекүні / Paémimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colheita / Data colectării / Дата сбора / Dátum odboru / Datum prikupljanja / Uppsamlingsdatum / Toplama tarihi / Дата забору

µL/test / µL/тест / µL/Test / µL/εξέταση / µL/prueba / µL/teszt / мкл/тест / µL/tyrimas / µL/pārbaude / µL/teste / мкл/анализ

Keep away from light / Пазете от светлина / Nevystavujte světlu / Må ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svetla / Fény nem érheti / Tenere al riparo dalla luce / Қараңғыланған жерде ұста / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Må ikke utsettes for lys / Przechowywać z dala od źródła światła / Manter ao abrigo da luz / Feriti de lumină / Хранить в темноте / Uchovávať mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Işıktan uzak tutun / Берегти від дії світла

Hydrogen gas generated / Образуван е водород газ / Možnost úniku plyného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadržai hydrogen vodik / Hidrogén gáz fejleszt / Produzione di gas idrogeno / Газтектеcs сутери пайда болды / Išiskiria vandenilio dujas / Rodas ūdeņradis / Waterstofgas gegenereerd / Hydrogengass generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Выделение водорода / Vyrobené použitím vodíka / Oslobađa se vodonik / Genererad vätgas / Açığa çıkan hidrojen gazı / Реакция з виділенням водню

Patient ID number / ИД номер на пациента / ID pacienta / Patientens ID-nummer / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de ID del paciente / Patsienti ID / No d'identification du patient / Identifikacijski broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттің идентификациялық нөмірі / Paciento identifikavimo numeris / Pacienta ID numurs / Identificatienummer van de patiënt / Patientens ID-nummer / Numer ID pacjenta / Número da ID do doente / Număr ID pacient / Идентификационный номер пациента / Identifikačné číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarası / Идентифікатор пацієнта

Fragile, Handle with Care / Чупливо, Роботете с необходимото внимание. / Křehké. Při manipulaci postupujte opatrně. / Forsigtig, kan gå i stykker. / Zerbrechlich, vorsichtig handhaben. / Ευθραusto. Χειριστείτε το με προσοχή. / Frágil. Manipular con cuidado. / Őrn, kásítsege ettevaatlilikult. / Fragile. Manipuler avec précaution. / Lomljivo, rukujte pažljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura. / Сынгыш, абайлап пайдаланыңыз. / 조심 깨지기 쉬운 처리 / Trapu, elkites atsargiai. / Trausls; rikoties uzmanīgi / Breekbaar, voorzichtig behandelen. / Ømtålig, håndter forsigtig. / Kruha zawartość, przenosić ostrożnie. / Frágil, Manuseie com Cuidado. / Fragil, manipulați cu atenție. / Хрупкое! Обращаться с осторожностью. / Krehké, vyžaduje sa opatrná manipulácia. / Lomljivo - rukujte pažljivo. / Bräckligt. Hantera försiktigt. / Kolay Kırılır, Dikkatli Taşın. / Тендітна, звертатися з обережністю / 易碎, 小心轻放

Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Australian Sponsor:
Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland