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261182 **BD BBL** Acridine Orange Reagent Droppers, 50

REFERENSER

1. Kronvall, G., and E. Myhre. 1977. Differential staining of bacteria in clinical specimens using acridine orange buffered at low pH. *Acta. Pathol. Microbiol. Scand. Sect. B* 85:249-254.
2. Lauer, B.A., L.B. Reller, and S. Mirrett. 1981. Comparison of acridine orange and Gram stains for detection of microorganisms in cerebrospinal fluid and other clinical specimens. *J. Clin. Microbiol.* 14:201-205.
3. McCarthy, L.R., and J.E. Senne. 1980. Evaluation of acridine orange stain for detection of microorganisms in blood cultures. *J. Clin. Microbiol.* 11:281-285.
4. Meseguer, M., L. de Rafael, M. Baquero, M. Martínez Ferrer, and M. Martínez López-Brea. 1984. Acridine orange stain in the early detection of bacteria in blood cultures. *Eur. J. Clin. Microbiol.* 3:113-115.

BD Diagnostics teknisk service: Kontakta närmaste BD-representant eller besök www.bd.com/ds.

Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbicante / Atқарушы / Gamintojas / Ražotājs / Tilvirker / Producent / Producător / Производител / Výrobca / Proizvođač / Tillverkare / Üretici / Виробник

Use by / Използвайте до / Spotføjbytte do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейін пайдалануға / Naudokite iki / Izljetot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Исползовать до / Použít do / Upotrebiti do / Använd före / Son kulanma tarihi / Використати до / YYYY-MM-DD / YYYY-MM (MM = end of month) / ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца) / RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) / AAAA-MM-DD / AAAA-MM (MM = slutning af måned) / JJJJ-MM-TT / JJJJ-MM (MM = Monatsende) / EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα) / AAAA-MM-DD / AAAA-MM (MM = fin del mes) / AAAA-KK-PP / AAAA-KK (KK = kuu lõpp) / AAAA-MM-JJ / AAAA-MM (MM = fin du mois) / GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca) / ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja) / AAAA-MM-GG / AAAA-MM (MM = fine mese) / ЖӨЖӨК-АА-КК / ЖӨЖӨК-АА / (АА = айдың соңы) / MMMM-MM-DD / MMMM-MM (MM = mensesio pabaiga) / GGGG-MM-DD / GGGG-MM (MM = mēneša beigas) / JJJJ-MM-DD / JJJJ-MM (MM = einde maand) / AAAA-MM-DD / AAAA-MM (MM = slutten av månaden) / RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) / AAAA-MM-DD / AAAA-MM (MM = fim do mês) / AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii) / ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца) / RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) / GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca) / AAAA-MM-DD / AAAA-MM (MM = slutet av månaden) / YYYY-AA-GG / YYYY-AA (AA = ayın sonu) / PPPP-MM-DD / PPPP-MM (MM = кінець місяця)

Catalog number / Каталоген номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalógusszám / Numero di catalogo / Каталог номер / Katalogo numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalogové číslo / Kataloški broj / Katalog numarası / Номер за каталогом

Authorized Representative in the European Community / Оторизирани представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret repræsentant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Voilittatut esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizirani predstavnik u Evropskoj uniji / Meghatalmazott képviselő az Európai Közösségekben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті өкіл / Įgaliotasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autoriseret representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupca v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Автура Топлудуğu Yetkili Temsilcisi / Уповноважений представник у країнах ЄС

In vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiniaparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In vitro Dijagnostiku / In vitro diagnosztikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жүргізілетін медициналық диагностика аспабы / In vitro diagnostikos prietaisai / Medicinas ierices, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medicinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispositiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики ин витро / Medicinska pomôcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Dijagnostik Tibbi Cihaz / Медицинский пристрій для діагностики in vitro

Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrænsning / Temperaturbegrenzung / Περιορισμό θερμοκρασίας / Limitación de temperatura / Temperatuuri piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны шектеу / Laikymo temperatūra / Temperatūras ierobežojumi / Temperaturlimit / Temperaturbegrensning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ohraničenie teploty / Ograničenje temperature / Temperaturgräns / Sıcaklık sınırlaması / Обмеження температури

Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Тонтама коды / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot number / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (lot) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партії

Contains sufficient for <n> tests / Съдържанието е достатъчно за <n> теста / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Küllaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenuto sufficiente per <n> test / <n> тесттегі үшін жеткілікті / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor "n" testen / Inholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conținut suficient pentru <n> teste / Достаточо для <n> тестов(a) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli miktarda içerir / Вистачить для аналізів: <n>

Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i bruksanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se bruksanvisningen / Kullanım Talimatları'na başvurun / Див. інструкції за використання

Keep away from light / Пазерте од светлина / Nevystavujte světu / Må ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svetla / Fény nem érheti / Tenere al riparo dalla luce / Қазақыланған жерде ұста / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Må ikke utsettes for lys / Przechowywać z dala od źródła światła / Manter ao abrigo da luz / Feriti de lumină / Хранить в темноте / Uchovávať mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Işıktan uzak tutun / Берегти від дії світла

Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:

Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

ATCC is a trademark of the American Type Culture Collection.

BD, BD Logo and BBL are trademarks of Becton, Dickinson and Company. © 2015 BD.