

BD Macro-Vue™ RPR Card Test Control Cards

R_x Only



0210745JAA(05)

2020-02

English

INTENDED USE

The BD Macro-Vue™ RPR Card Test Control Cards are dehydrated control specimens of predetermined reactivity for quality-control testing of antigen before use in performing the BD Macro-Vue RPR Card Tests for the serologic detection of syphilis.

SUMMARY AND EXPLANATION

BD Macro-Vue RPR Card Test Control Cards were designed for routine, daily use as check-test specimens to determine the acceptable reactivity of RPR Card antigen suspension. RPR Card antigen is a cardiolipin type antigen, which detects “reagin”, an antibody-like substance present in serums from syphilitic persons, and occasionally in serums of persons with other acute and chronic conditions.¹⁻⁴

PRINCIPLES OF THE PROCEDURE

The RPR 18 mm Circle Control Cards have labelled test areas containing dehydrated specimens with designated patterns of graded reactivity. Upon rehydration and proper execution of the Control Card procedure, the RPR Card antigen suspension should reproduce the established reactivity pattern (see “Procedure”) to confirm optimal reactivity of the antigen.

REAGENTS

The RPR 18 mm Circle Card Test Control Card consists of three circles designated as: Reactive, Reactive minimal-to-moderate and Nonreactive. The labelled areas on this card contain graded reactivity specimens. The specimen material is human plasma purchased from licensed blood banks as Reactive or Nonreactive for syphilis.

WARNINGS AND PRECAUTIONS:

For *in vitro* Diagnostic Use.

Before proceeding, review “Procedure” and “Warnings and Precautions” in the package insert supplied in all BD Macro-Vue RPR Card Test Kits.

The plasma used on the control cards is derived from human blood tested by an FDA (U.S. Food and Drug Administration)-approved method for the presence of the antibody to HIV (human immunodeficiency virus) and HBsAg (hepatitis B surface antigen) and found to be nonreactive.

WARNING: Because no test method can offer complete assurance that HIV, hepatitis B virus, or other infectious agents are absent, SPECIMENS AND THESE REAGENTS SHOULD BE HANDLED AS THOUGH CAPABLE OF TRANSMITTING AN INFECTIOUS DISEASE. “Standard Precautions”⁵⁻⁸ and institutional guidelines should be followed in handling all items contaminated with blood and other body fluids.

Controls and RPR Card antigen suspension should be at room temperature (23 to 29 °C) when used.

If the established reactivity pattern is not obtained, delay routine testing until the optimal reactivity is obtained by rechecking antigen suspension, test procedures, room temperature and equipment (including use of a rotator with humidifying cover). Non-acceptable results invalidate tests on individual specimens.

Once the dehydrated specimens on the control card are reconstituted, the specimen solutions should be tested immediately using the RPR Card Test procedure. Delay in testing may allow evaporation to affect the reactivity pattern.

Prolonged storage at cool room temperatures (21 to 25 °C) and at elevated temperatures (above 27 °C) causes deterioration of the specimens, showing a decrease in reactivity and difficulty in reconstituting the card. Once the sealed envelope is opened, the card should be used immediately.

Reconstitution of Control Card: Each test area of the control card should be reconstituted with 0.05 mL of *deionized/distilled water*.

Storage Instructions: In the unopened foil-sealed envelopes, the control cards retain an acceptable reactivity pattern for the assigned expiration date when stored under refrigeration at 2 to 8 °C, which is the recommended storage temperature.

PROCEDURE AND RESULTS

Materials Provided: BD Macro-Vue RPR Card Test Control Cards

Materials Required But Not Provided:

1. One of the following BD Macro-Vue RPR Card Test Kits*:
18 mm Circle Test—Kit No. 115, 110, 104, 510 or 532 (qualitative) or Kit No. 112 (quantitative)
All kits contain sufficient overage of materials for performance of daily control card tests.
2. A micropipettor which dispenses 0.05 mL (50 µL) and 0.03 mL (30 µL).
3. A rotator, 100 ± 2 rpm, 2 cm diameter, automatic timer, friction drive, with a humidifying cover containing a moistened blotter or sponge.

*For availability of kits, refer to the package insert supplied with all BD Macro-Vue RPR Card Test Kits.

TEST PROCEDURE:

1. Place 0.05 mL (50 µL) deionized/distilled water onto the three circles, using an automatic micropipettor (or 0.05 mL capillary with rubber bulb attached). *Do not use BD Dispensstirs™ device to reconstitute a Control Card.*
2. Using the stirring end of an 0.05 mL BD Dispensstirs device (or broad end of a stirrer) for each circle, mix until dehydrated control specimen is dissolved. Spread specimen to the inside of the circle. *A new BD Dispensstirs device or stirrer must be used for each circle!*
3. Shake the antigen bottle before use. Holding in a vertical position, dispense one or two drops in the cap to make sure needle passage is clear. Place *one* "free-falling" drop of antigen [1/60 mL antigen – (20 G yellow hub needle)] onto each test area. *Do not restir*; mixing of antigen and specimen is accomplished during rotation. Pick up pre-dropped antigen from the bottle cap.
4. Rotate for eight min, on a mechanical rotator at 100 ± 2 rpm, using a moistened humidifying cover. Following rotation, to help differentiate Nonreactive from Reactive minimal-to-moderate results, a brief rotating and tilting of the card by hand (three or four to-and-fro motions) must be made.
5. *Immediately* read the card macroscopically in the "wet" state under a high intensity incandescent lamp or strong daylight.
The Reactive control should show characteristic strong clumping; the Nonreactive control should show the smooth, grayish appearance of unclumped particles. The Reactive minimal-to-moderate control should show minimal-to-moderate clumping. See the Reading Guide in the package insert provided in all BD Macro-Vue RPR Card Test Kits.

User Quality Control:

Quality control requirements must be performed in accordance with applicable local, state and/or federal regulations or accreditation requirements and your laboratory's standard Quality Control procedures. It is recommended that the user refer to pertinent CLSI guidance and CLIA regulations for appropriate Quality Control practices.

LIMITATIONS OF THE PROCEDURE

If the specimens do not dissolve satisfactorily when reconstituted with deionized/distilled water, the card should not be used. Delay in performance of the test procedure following reconstitution, or performing the test procedure in an environment of low humidity (particularly without a humidifying cover on the rotator) may cause an undesirable reactivity pattern (a granular appearance in the Nonreactive specimen). Since the dehydrated specimens are quantitatively controlled for reactivity, the proper execution of the Control Card and RPR Card Test procedures are essential for the determination of the expected reactivity pattern.

The intensity of clumping observed should not be used as a reading standard or guide in interpreting reactions observed with unknown specimens. The Reading Guide contained in the package insert provided with all BD Macro-Vue RPR Card Test Kits is intended for this purpose.

EXPECTED VALUES AND PERFORMANCE CHARACTERISTICS

Each lot of 18 mm Circle Control Cards is tested and compared for the designated pattern of reactivity against reference antigen suspensions and meets the U.S. Centers for Disease Control and Prevention (CDC) product specifications.

The expected patterns of reactivity on the control cards following the performance of RPR Card Test procedures are shown in the Reading Guides contained in the package insert provided in all BD Macro-Vue RPR Card Test Kits.

Reactive Circle – Similar to undiluted Reactive titer spot shown under the RPR Card quantitative test.

Reactive minimal-to-moderate Circle – Similar to Reactive minimal spot of RPR Card qualitative test and to Reactive (1:4) spot of RPR Card quantitative test.

Nonreactive Circle – Similar to RPR Card Nonreactive spot.

AVAILABILITY

Cat. No. DESCRIPTION

276709 BD Macro-Vue™ RPR Card Test Control Cards

REFERENCES

1. Creighton, E.T. 1990. Rapid plasma reagin (RPR) 18-mm circle card test, p. 193–207. *In* S.A. Larsen, E.F. Hunter, and S.J. Kraus (ed.), *A manual of tests for syphilis*, 9th ed. American Public Health Association, Washington, D.C.
2. Portnoy, J., J.H. Brewer, and A. Harris. 1962. Rapid plasma reagin card test for syphilis and other treponematoses. *Public Health Rep.* 77:645–652.
3. Portnoy, J. 1963. Modifications of the rapid plasma reagin (RPR) card test for syphilis, for use in large scale testing. *Am. J. Clin. Pathol.* 40:473–479.
4. Falcone, V.H., G.W. Stout, and M.B. Moore, Jr. 1964. Evaluation of rapid plasma reagin (circle) card test. *Public Health Rep.* 79:491–495.
5. National Committee for Clinical Laboratory Standards. 2001. Approved Guideline M29-A2. Protection of laboratory workers from occupationally acquired infections, 2nd ed. NCCLS, Wayne, Pa.
6. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. *Infect. Control Hospital Epidemiol.* 17:53–80.
7. U.S. Department of Health and Human Services. 1999. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 4th ed. U.S. Government Printing Office, Washington, D.C.
8. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). *Official Journal L262*, 17/10/2000, p. 0021–0045.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or bd.com.

Change History

Revision	Date	Change Summary
05	2020-02	Converted printed instructions for use to electronic format and added access information to obtain the document from bd.com/e-labeling . Updated Reference No. 1. Updated Australian Sponsor address and added New Zealand Sponsor address.

US Customers only: For symbol glossary, refer to bd.com/symbols-glossary



Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbicante / Аткарушы / 제조업체 / Gamintojas / Ražotājs / Tilvirker / Producent / Producător / Производител / Výrobca / Proizvođač / Tillverkare / Üretici / Виробник / 生产厂商



Use by / Използвайте до / Spotfobujete do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / 사용 기한 / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейин пайдалануға / Naudokite iki / Izljetot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Използovať до / Použite do / Upotrebiti do / Använd före / Son kulanma tarihi / Використати до / 使用截止日期

YYYY-MM-DD / YYYY-MM (MM = end of month)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = slutning af måned)
JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)
EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)
AAAA-MM-DD / AAAA-MM (MM = fin del mes)
AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)
AAAA-MM-JJ / AAAA-MM (MM = fin du mois)
GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)
EEEE-HH-NN / EEEE-HH (HH = hónap utolsó napja)
AAAA-MM-GG / AAAA-MM (MM = fine mese)
ЖОЖОЖ-АА-КК / ЖОЖОЖ-АА / (АА = айдың соңы)
YYYY-MM-DD/YYYY-MM (MM = 월말)
MMMM-MM-DD / MMMM-MM (MM = mėnesio pabaiga)
GGGG-MM-DD/GGGG-MM (MM = mėneša beigas)
JJJJ-MM-DD / JJJJ-MM (MM = einde maand)
AAAA-MM-DD / AAAA-MM (MM = slutten av månaden)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = fim do mês)
AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
GGGG-MM-DD / GGGG-MM (MM = kraj meseca)
AAAA-MM-DD / AAAA-MM (MM = slutet av månaden)
YYYY-AA-GG / YYYY-AA (AA = ayın sonu)
PPPP-MM-DD / PPPP-MM (MM = кінець місяця)
YYYY-MM-DD / YYYY-MM (MM = 月末)



Catalog number / Каталоген номер / Katalógové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalógusszám / Numero di catalogo / Каталог номер / 카탈로그 번호 / Katalogo / numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalógové číslo / Kataloški broj / Katalog numarası / Номер за каталогом / 目录号



Authorized Representative in the European Community / Оторизирани представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret representant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatud esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizuirani predstavnik u Evropskoj uniji / Meghatalmazott képviselő az Európai Közösségekben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті өкіл / 유럽 공동체의 위임 대표 / Įgaliotasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autoriseret representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupce v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Avrupa Topluluğu Yetkili Temsilcisi / Уповноважений представник у країнах ЄС / 欧洲共同体授权代表



In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiiniparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnosztikai orvosi eszköz / Dispositivo medical per diagnostica in vitro / Жасанды жағдайда жүргізетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietais / Medicinas ierīces, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispozitiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики in vitro / Medicinska pomůcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diagnostik Tibbi Cihaz / Медицинский прибор для диагностики in vitro / 体外诊断医疗设备



Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrænsning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperaturi piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны шектеу / 온도 제한 / Laikymo temperatūra / Temperatūras ierobežojumi / Temperaturlimit / Temperaturbegrensning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ohraničenje teploty / Ograničenje temperature / Temperaturgräns / Sıcaklık sınırlaması / Обмеження температури / 温度限制



Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Топтама коды / 배치 코드(코트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot number / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партії / 批号 (亚批)



Contains sufficient for <n> tests / Съдържанието е достатъчно за <n> теста / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Kállaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenido suficiente per <n> test / <n> тесттері үшін жеткінікті / <n> 테스트가 충분히 포함됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor "n" testen / Innholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Continut suficient pentru <n> teste / Достаточное для <n> тестов(a) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli malmaze içerir / Вистачить для аналізів: <n> / 足够进行 <n> 次检测



Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / 사용 지침 참조 / Skaityti naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i brugsanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se brugsanvisningen / Kullanım Talimatları'na başvurun / Див. інструкції з використання / 请参阅使用说明



Do not reuse / Не използвайте отново / Nepoužívejte opakovaně / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / No reutilizar / Mitte kasutada korduvalt / Ne pas réutiliser / Ne koristiti ponovo / Egyyszer használatos / Non riutilizzare / Пайдаланбаңыз / 재사용 금지 / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolositi / Не использовать повторно / Nepoužívejte opakovane / Ne upotrebljavajte ponovo / Får ej återanvändas / Tekrar kulanmayin / Не використовувати повторно / 请勿重复使用



Serial number / Серийен номер / Sériové číslo / Seriennummer / Seriennummer / Σειριακός αριθμός / N° de serie / Seerianumber / Numéro de série / Serijski broj / Sorozatszám / Numero di serie / Топтамалық нөмірі / 일련 번호 / Serijos numeris / Sėrijas numurs / Serie nummer / Numer seryjny / Número de série / Număr de serie / Серийный номер / Seri numarası / Номер серії / 序列号

	For IVD Performance evaluation only / Само за оценка качество на работа на IVD / Pouze pro vyhodnocení výkonu IVD / Kun til evaluering af IVD ydelse / Nur für IVD-Leistungsbewertungszwecke / Μόνο για αξιολόγηση απόδοσης IVD / Sólo para la evaluación del rendimiento en diagnóstico in vitro / Ainult IVD seadme hindamiseks / Réserve à l'évaluation des performances IVD / Samo u znanstvene svrhe za In Vitro Dijagnostiku / Kizárolag in vitro diagnosztikához / Solo per valutazione delle prestazioni IVD / Жасанды жағдайда «пробирка ішінде» диагностикада тек жұмысты бағалау үшін / IVD 성능 평가에 대해서만 사용 / Tik IVD prietaisų veikimo charakteristikoms tikrinti / Vienīgi IVD darbības novērtēšanai / Uitsluitend voor doeltreffendheidsonderzoek / Kun for evaluering av IVD-ytelse / Tylko do oceny wydajności IVD / Uso exclusivo para avaliação de IVD / Numai pentru evaluarea performanței IVD / Только для оценки качества диагностики in vitro / Určenie iba na diagnostiku in vitro / Samo za procenu učinka u in vitro dijagnostici / Endast för utvärdering av diagnostisk användning in vitro / Yalnızca IVD Performans değerlendirmesi için / Тільки для оцінювання якості діагностики in vitro / 仅限 IVD 性能评估
	For US: "For Investigational Use Only"
	Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiir / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūros žemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrense / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sıcaklık alt sınırı / Мінімальна температура / 温度下限
CONTROL	Control / Контролно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Controllo / Бақылау / 컨트롤 / Kontrolé / Kontrolle / Controle / Controllo / Контроль / kontroll / 对照
CONTROL +	Positive control / Положителен контрол / Pozitivní kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitivna kontrola / Pozitív kontroll / Controllo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívá kontrolé / Positieve controle / Kontrola dodatnia / Controllo positivo / Control pozitiv / Положительный контроль / Pozitif kontrol / Позитивний контроль / 阳性对照试剂
CONTROL -	Negative control / Отрицателен контрол / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negativna kontrola / Negativ kontroll / Controllo negativo / Негативті бақылау / 음성 컨트롤 / Neigiama kontrolė / Negatívá kontrolé / Negatieve controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативний контроль / 阴性对照试剂
STERILE EO	Method of sterilization: ethylene oxide / Метод на стерилизация: етиленов оксид / Způsob sterilizace: etylenoxid / Steriliseringsmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseerimismetod: etüleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etilén-oxid / Metoda di sterilizzazione: ossido di etilene / Sterilizacija edici – етилен тогызы / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksisds / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringsmetode: etylenoksid / Metoda sterylizacji: tienek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metodá sterilizácie: etylenoxid / Metoda sterilizacije: etilen oksid / Steriliseringsmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: этиленоксидом / 灭菌方法: 环氧乙烷
STERILE R	Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Způsob sterilizace: záření / Steriliseringsmetode: bestrålning / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseerimismetod: kiirgus / Méthode de stérilisation : irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metoda di sterilizzazione: irradiazione / Sterilizacija edici – сәуле тисы / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringsmetode: bestrålning / Metoda sterylizacji: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: iradiere / Метод стерилизации: облучение / Metodá sterilizácie: ožiarenie / Metoda sterilizacije: ozračevanje / Steriliseringsmetod: strålning / Sterilizasyon yöntemi: ırradyasyon / Метод стерилизації: опроміннення / 灭菌方法: 辐射
	Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologilised riskid / Risques biologiques / Biološki rizik / Biológiai veszélyes / Rischio biologico / Биологический тауекелдер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Biologische опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyolojik Riskler / Біологічна небезпека / 生物学风险
	Caution, consult accompanying documents / Внимание, направете справка в придружаващите документи / Pozor! Pristudujte si příloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatust! Lugeka kaasnevat dokumentatsiooni! / Attention, consulter les documents joints / Urozorenje, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dmesio, žiūrėkite priedamus dokumentus / Piesardziba, skaiti pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Vystraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацію / 小心, 请参阅附带文档。
	Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / 상한 온도 / Aukščiausia laikymo temperatūra / Ausgējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrense / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sıcaklık üst sınırı / Міксимальна температура / 温度上限
	Keep dry / Пазете сухо / Skladujte v suchém prostředi / Opbevarer tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / 건조 상태 유지 / Laikykite sausai / Uzglabāt sausu / Droog houden / Holdes tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávať v suchu / Držite na suvom mestu / Förvaras tørt / Kuv bir çekilde muhafaza edin / Бергетти від вологості / 请保持干燥
	Collection time / Време на събиране / Čas odběru / Opsamlingsstidspunkt / Entnahmeuhrzeit / Ώρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жынау уақыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid förövtagning / Godzina pobrania / Hora de colheita / Ora colectării / Время сбора / Doba odboru / Vreme prikupljanja / Uppsamlingsstid / Toplama zamanı / Час забору / 采集时间
	Peel / Обелете / Otevfete zde / Abn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skinu / Húzza le / Staccare / Ўстиғи қабатын алып таста / 벗기 / Plęsti čia / Atfimt / Schillen / Trek av / Oderwać / Destacar / Se dezlipęte / Отклеить / Odrhните / Oljuštiti / Dra isär / Ayırma / Відкрити / 撕下
	Perforation / Перфорация / Perforace / Perforering / Διάτρηση / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Тесик тесу / 찢취선 / Perforacija / Perforácia / Perforatie / Perforacja / Perfuração / Perforare / Перфорация / Perforácia / Perforasyon / Перфорация / 穿孔
	Do not use if package damaged / Не използвайте, ако опаковката е повредена / Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packungnicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά / No usar si el paquete está dañado / Mit kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használnia, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Ереп пакет бұзылған болса, пайдаланба / 패키지가 손상된 경우 사용 금지 / Jei pakuotė pažeista, nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Не использовать при повреждении упаковки / Nepoužívať, ak je obal poškodený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки / 如果包装破损, 请勿使用
	Keep away from heat / Пазете от топлина / Nevystavujte přílišnému teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgusest / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Сапқын жерде сақта / 열을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / Не награвать / Uchovávať mimo zdroja tepla / Držite dalje od toplote / Fär ej utsättas för värme / Isidan uzak tutun / Бергетти від дії тепла / 请远离热源
	Cut / Срежете / Odstřihněte / Klip / Schneiden / Κόψτε / Cortar / Lõigata / Découper / Reži / Vágja ki / Tagliare / Keciңiz / 잘라내기 / Kirpti / Nogriez / Knippen / Kutt / Odciąć / Cortar / Decupaji / Отрезать / Odstřihnite / Iseći / Klipp / Kesme / Pozpizati / 剪下



Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmedatum / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuurpäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жинаған тізбекүні / 수집 날짜 / Paemimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colheita / Data colectării / Дата сбора / Datum odberu / Datum prikupljanja / Uppsamlingsdatum / Toplama tarihi / Дата забору / 采集日期



µL/test / µL/тест / µL/Test / µL/εξέταση / µL/prueba / µL/teszt / µL/테스트 / мкл/тест / µL/tyrimas / µL/pårbaude / µL/teste / мкл/анализ / µL/检测



Keep away from light / Пазете от светлина / Nevystavujte světlu / Må ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svetla / Fény nem érheti / Tenere al riparo dalla luce / Қараңғыланған жерде ұста / 빛을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Må ikke utsettes for lys / Przechowywać z dala od źródła światła / Manter ao abrigo da luz / Feriți de lumină / Хранить в темноте / Uchovávajte mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Işıktan uzak tutun / Беретти від дії світла / 请远离光线



Hydrogen gas generated / Образуван е водород газ / Možnost úniku plynného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadrží hydrogen vodik / Hidrogén gázt fejleszt / Produzione di gas idrogeno / Газтөктес сутері пайда болды / 수소 가스 생성됨 / Išskiria vandenilio dujas / Rodas ūdenradis / Waterstofgas gegenereerd / Hydrogengass generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Выделение водорода / Vyrobené použitím vodíka / Osloбаda se vodonik / Genererad vätgas / Açığa çıkan hidrojen gazı / Реакция з виділенням водню / 会产生氢气



Patient ID number / ИД номер на пациента / ID pacienta / Patientens ID-nummer / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de ID del paciente / Patsiendi ID / No d'identification du patient / Identifikacijski broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттің идентификациялық нөмірі / 환자 ID 번호 / Paciento identifikavimo numeris / Pacienta ID numurs / Identificationnummer van de patiënt / Pasientens ID-nummer / Numer ID pacienta / Número da ID do doente / Număr ID pacient / Идентификационный номер пациента / Identifikačné číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarası / Идентификатор пациента / 患者标识号



Fragile, Handle with Care / Чупливо, Работете с необходимото внимание. / Křehké. Při manipulaci postupujte opatrně. / Forsiktig, kan gå i stykker. / Zerbrechlich, vorsichtig handhaben. / Εύθραστο. Χειριστείτε το με προσοχή. / Frágil. Manipular con cuidado. / Öm, käsitsege ettevaatlikult. / Fragile. Manipuler avec précaution. / Lomljivo, rukujte pažljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura. / Сыныш, абайлап пайдаланыңыз. / 조심 깨지기 쉬운 처리 / Trapu, elkites atsargiai. / Trausls; rikoties uzmanīgi / Breekbaar, voorzichtig behandelen. / Ømtål, håndter forsigtig. / Krucha zawartość, przenosić ostrożnie. / Frágil, Manuseie com Cuidado. / Frágil, manipulați cu atenție. / Хрупкое! Обращаться с осторожностью. / Křehké, vyžaduje sa opatrná manipulácia. / Lomljivo - rukujte pažljivo. / Bräckligt. Hantera försiktigt. / Kolay Kırılır, Dikkatli Taşın. / Тендітна, звертатися з обережністю / 易碎, 小心轻放

R_x Only

This only applies to US: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / S'applique uniquement aux États-Unis: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Vale solo per gli Stati Uniti: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Gilt nur für die USA: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Sólo se aplica a los EE.UU.: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner."



bd.com/e-labeling

Europe, CH, GB, NO:		+800 135 79 135
International:		+31 20 794 7071
AR	+800 135 79 135	LT 8800 30728
AU	+800 135 79 135	MT +31 20 796 5693
BR	0800 591 1055	NZ +800 135 79 135
CA	+1 855 805 8539	RO 0800 895 084
CO	+800 135 79 135	RU +800 135 79 135
EE	0800 0100567	SG 800 101 3366
GR	00800 161 22015 7799	SK 0800 606 287
HR	0800 804 804	TR 00800 142 064 866
IL	+800 135 79 135	US +1 855 236 0910
IS	800 8996	UY +800 135 79 135
LI	+31 20 796 5692	VN 122 80297



Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA



Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:

Becton Dickinson Pty Ltd.
66 Waterloo Road
Macquarie Park NSW 2113
Australia

New Zealand Sponsor:

Becton Dickinson Limited
14B George Bourke Drive
Mt. Wellington Auckland 1060
New Zealand

BD, the BD Logo, Dispenstirs, and Macro-Vue are trademarks of Becton, Dickinson and Company or its affiliates.
© 2020 BD. All rights reserved.