Part Number:	8860321JAA	BALTSO0191 Version 13.0 Template 4 Inserts	
Category and Description:	Package Insert Sensi-Disc Single Dispenser	Rev from: 02	Rev to: 03
		Job Number:	448-18

Catalog Number: 260457, 260459

Blank (Sheet) Size: Length: 5.5" Width: 10.5"

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
Number of Pages: 6 Number of Sheets: 1

Page Size: Length: 5.5" Width: 3.5" Final Folded Size: 5.5" X 3.5"

Ink Colors: Number of Colors: 1 PMS #: BLACK

Printed Two Sides: Yes: ☒ No: ☐

Style (see illustrations below): # 3

Vendor Printed: ☒ Online / In House Printed: ☐ Web Printed: ☐

See Specification control no. N/A for material information.

BD Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Company confidential. This document is the property of Becton, Dickinson and Company and is not to be used outside the company without written permission. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Revised By: **REVISED BY**
By Tori Pagani at 2:08 pm, Aug 24, 2018

Proofing
Approved By: **PROOFING APPROVED BY**
By Nichole Graham at 6:33 pm, Aug 28, 2018

Third Eye By: **THIRD EYE BY**
By Sonia F. Thompson at 2:01 pm, Sep 04, 2018

BD BBL™ Sensi-Disc™

Single Disc Dispenser

REF 260457

260459

8860321JAA(03)

2018-09

Rx Only

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokyny vám poskytne místní zástupce společnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteyks lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőjétől. / Нұсқаулар үшін жергілікті BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielem BD. / Contacte o representante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указаний обратитесь к местному представителю компании BD. / Instrukcie získate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD.

INTENDED USE

The BD BBL Sensi-Disc™ Single Disc Dispenser is recommended for the individual application of cartridge BBL™ brand antimicrobial susceptibility test and biochemical differentiation discs onto the surface of inoculated plated media.

The Single Disc Dispenser replaces the manual placement of discs using sterile forceps.

Precautions: For *in vitro* Diagnostic Use.

Observe aseptic techniques and established precautions against microbiological hazards throughout all procedures.

PRODUCT DESCRIPTION

Single Disc Dispensers are made of durable 24 gauge stainless steel. Store the loaded dispenser in the refrigerator at 2–8 °C when not in use.

INSTRUCTIONS

Load a dispenser by placing the bottom of a cartridge into the bottom of the dispenser, then press the top of the cartridge into the top of the dispenser where it will be retained by a friction fit.

Operate the loaded unit by pressing the spring steel arm which will eject one disc.

Technical Information: In the United States, contact BD Technical Service and Support at 800.638.8663 or www.bd.com.

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BD BBL Sensi-Disc **Single Disc Dispenser**

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 BD BBL Sensi-Disc
Single Disc Dispenser

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R_x Only

This only applies to US: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / S'applique uniquement aux États-Unis: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Vale solo per gli Stati Uniti: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Gilt nur für die USA: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Sólo se aplica a los EE.UU.: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner."

Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

© 2018 BD. BD and the BD Logo are trademarks of Becton, Dickinson and Company.