


Revisions

SO 0191-5


Rev from	Rev to	ECO #
01	2011/01	5677-11

Notes:

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
- BD Cat. Number 448939
- Blank (Sheet) Size : Length: 14.0" Width: 8.50"
 Number of Pages: 2 Number of Sheets: 1
 Page Size: Length 14.0" Width 8.50" Final Folded Size: No Fold.
- Style (see illustrations below): # 1




#1

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


#2

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


#3

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


#4

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

#5

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

#6

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

#7

- See Specification Control Number N/A for Material Information
- Ink Colors: Printed two sides ☒ Yes ☐ No
 No. of Colors: 1 PMS# Standard Black
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level

PC-Generated Insert

Label Design	Date	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION	 Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA	
Proofer	Date			
Checked By	Date			
Part Number: L009459		Category and Description Phoenix Insert, NMIC/ID-57	Sheet: 1 of 3	A
			Scale: N/A	

Table 1

Antimicrobial / Antibiotiques / Antibiotikum / Antibiotico / Agente antimicrobiano / Anti-microbiano		(µg/mL)	<i>E. coli</i> ATCC 25922	<i>P. aeruginosa</i> ATCC 27853	<i>E. coli</i> ATCC 35218	<i>K. pneumoniae</i> ATCC 700603	
Amikacin	AN	4 – 32	≤0.5 – 4	1 – 4			
Amoxicillin/Clavulanate	AMC	2/1 – 16/8	2/1 – 8/4		4/2 – 16/8		
Ampicillin	AM	2 – 32	2 – 8				
Aztreonam	ATM	2 – 16	≤0.5	2 – 16			
Cefazolin	CZ	4 – 16	1 – 4				
Cefepime	FEP	1 – 16	≤0.5	1 – 8			
Cefotaxime	CTX	1 – 32	<0.5	16 – 64			
Ceftazidime	CAZ	1 – 16	≤0.5	1 – 4			
Cefuroxime	CXM	4 – 16	2 – 8				
Ciprofloxacin	CIP	0.5 – 4	≤0.125	0.25 – 1			
Colistin	CL	1 – 4		≤0.5 – 1			
Gentamicin	GM	1 – 8	<0.5 – 1	<0.5 – 2			
Meropenem	MEM	1 – 8	≤0.25	≤0.25 – 1			
Netilmicin	NET	1 – 16	≤0.5 – 1	≤0.5 – 8			
Nitrofurantoin	FM	16 – 64	≤8 – 16				
Norfloxacin	NOR	2 – 8	≤0.25	1 – 4			
Piperacillin/Tazobactam	TZP	4/4 – 64/4	1/4 – 4/4	1/4 – 8/4	≤0.5/4 – 2/4		
Trimethoprim	TMP	0.5 – 4	<0.5 – 2	>16			
Trimethoprim/Sulfamethoxazole	SXT	0.5/9.5 – 2/38	≤0.5/9.5	8/152 – >16/304			
ESBL	ESBL	–	NEG			POS	
Cefotaxime/Clavulanate (ESBL)	CCX	<9	N/A			N/A	
Ceftazidime/Clavulanate (ESBL)	CCZ	<9	N/A			N/A	
Cefpodoxime – proxetil (ESBL)	CPD	<9	N/A			N/A	
Ceftazidime (ESBL)	CAZ	<9	N/A			N/A	
Ceftriaxone/Clavulanate (ESBL)	CCR	<9	N/A			N/A	

Table 2

Organism / Organisme / Organismus / Organismo / Organismo / Microrganismo	Not Reported / Nonrapporté / Nicht berichtet / Non riportato / No registrado / Nao Participado	Alternate Method / Utiliser autre méthode / Alternativmethode anwenden / Usare metodo alternativo / Método alternativo recomendado / Metodo Alternativo
<i>Proteus mirabilis</i>		Aztreonam, Cefazolin

Table 3

Organism / Organisme/ Organismus / Organismo / Organismo / Microrganismo	Not Reported / Nonrapporté / Nicht berichtet / Non riportato / No registrado / Nao Participado	Alternate Method / Utiliser autre méthode / Alternativmethode anwenden / Usare metodo alternativo / Método alternativo recomendado / Metodo Alternativo
<i>Stenotrophomonas maltophilia</i>	Colistin	
<i>Serratia marcescens</i>		Colistin