

# Revisions

SO 0191-5

| Rev from | Rev to | ECO #   |
|----------|--------|---------|
| 2011/01  | 01     | 6048-12 |

## Notes:


- BD Cat. Number: 448087
- Blank (Sheet) Size: Length: 8.50" Width: 14"  
Number of Pages: 2 Number of Sheets: 1  
Page Size: Length: 8.5" Width: 14" Final Folded Size: No Fold
- Style (see illustrations below): # 1




#1



#2

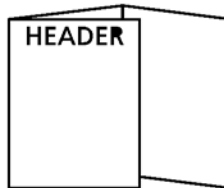


#3

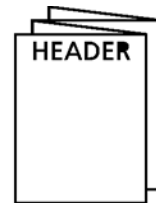


#4



#5

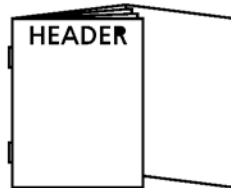

#6

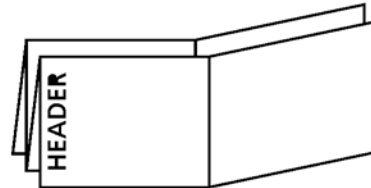

#7

- See Specification Control Number N/A for Material Information.
- Ink Colors: Printed two sides ☒ Yes ☐ No  
No. of Colors: 1 PMS # Standard Black
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

PC-Generated Insert

|                                    |      |                                                                                                                                                            |                                                                                                                                                                          |   |
|------------------------------------|------|------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---|
| Label Design                       | Date | COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION. |  <b>BD</b> Becton, Dickinson and Company<br>7 Loveton Circle<br>Sparks, MD 21152 USA |   |
| Proofer                            | Date |                                                                                                                                                            |                                                                                                                                                                          |   |
| Checked By                         | Date |                                                                                                                                                            |                                                                                                                                                                          |   |
| Part Number:<br><div>L009928</div> |      | Category and Description<br><div>Package Insert,<br/>Phoenix NMIC/ID-75</div>                                                                              | Sheet: 1 of 3                                                                                                                                                            | A |
|                                    |      |                                                                                                                                                            | Scale: N/A                                                                                                                                                               |   |



Table 1

| Antimicrobici / Антимикробен агент / Zášita od mikroba / Antimikrobiální činidlo / Antimikroob / Antimicrobien / Antibiotikum / Антибиотики / Antibiotico / Микрообка карсы зат / Antimikrobinis vaistas / Antimikrobiel agents / Антибиоти / Antimicrobiano / Antibiotic / Антибиотик / Antimikrobiel / Antimikrobiálna látka / Antimikrobiikum | ( µg/mL )      | E. coli ATCC™ 25922 | P. aeruginosa ATCC 27853 | E. coli ATCC 35218 | K. pneumoniae ATCC 700603 |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------|---------------------|--------------------------|--------------------|---------------------------|
| Amikacin                                                                                                                                                                                                                                                                                                                                         | 8 - 32         | ≤ 0.5 - 4           | 1 - 4                    |                    |                           |
| Amoxicillin/Clavulanate                                                                                                                                                                                                                                                                                                                          | 4/2 - 16/8     | 2/1 - 8/4           |                          | 4/2 - 16/8         |                           |
| Ampicillin                                                                                                                                                                                                                                                                                                                                       | 4 - 16         | 2 - 8               |                          |                    |                           |
| Aztreonam                                                                                                                                                                                                                                                                                                                                        | 2 - 16         | ≤ 0.5               | 2 - 16                   |                    |                           |
| Cefazolin                                                                                                                                                                                                                                                                                                                                        | 4 - 16         | 1 - 4               |                          |                    |                           |
| Cefoxitin                                                                                                                                                                                                                                                                                                                                        | 4 - 16         | 2 - 8               |                          |                    |                           |
| Ceftazidime                                                                                                                                                                                                                                                                                                                                      | 1 - 16         | ≤ 0.5               | 1 - 4                    |                    |                           |
| Ceftriaxone                                                                                                                                                                                                                                                                                                                                      | 1 - 32         | ≤ 0.5               | 8 - 64                   |                    |                           |
| Cefuroxime                                                                                                                                                                                                                                                                                                                                       | 4 - 16         | 2 - 8               |                          |                    |                           |
| Ciprofloxacin                                                                                                                                                                                                                                                                                                                                    | 0.125 - 2      | ≤ 0.125             | 0.25 - 1                 |                    |                           |
| Colistin                                                                                                                                                                                                                                                                                                                                         | 0.5 - 2        |                     | ≤ 0.5 - 1                |                    |                           |
| Gentamicin                                                                                                                                                                                                                                                                                                                                       | 2 - 8          | ≤ 0.5 - 1           | ≤ 0.5 - 2                |                    |                           |
| Levofloxacin                                                                                                                                                                                                                                                                                                                                     | 1 - 4          | ≤ 0.25              | 0.5 - 4                  |                    |                           |
| Meropenem                                                                                                                                                                                                                                                                                                                                        | 1 - 8          | ≤ 0.25              | ≤ 0.25 - 1               |                    |                           |
| Nitrofurantoin                                                                                                                                                                                                                                                                                                                                   | 16 - 64        | ≤ 8 - 16            |                          |                    |                           |
| Norfloxacin                                                                                                                                                                                                                                                                                                                                      | 2 - 8          | ≤ 0.25              | 1 - 4                    |                    |                           |
| Piperacillin                                                                                                                                                                                                                                                                                                                                     | 4 - 64         | 1 - 4               |                          |                    |                           |
| Piperacillin/Tazobactam                                                                                                                                                                                                                                                                                                                          | 4/4 - 64/4     | 1/4 - 4/4           | 1/4 - 8/4                | ≤ 0.5/4 - 2/4      |                           |
| Tobramycin                                                                                                                                                                                                                                                                                                                                       | 2 - 8          | 0.25 - 1            | 0.25 - 1                 |                    |                           |
| Trimethoprim                                                                                                                                                                                                                                                                                                                                     | 1 - 8          | ≤ 0.5 - 2           | >16                      |                    |                           |
| Trimethoprim/Sulfamethoxazole                                                                                                                                                                                                                                                                                                                    | 0.5/9.5 - 2/38 | ≤ 0.5/9.5           | 8/152 - >16/304          |                    | POS                       |
| ESBL                                                                                                                                                                                                                                                                                                                                             | -              | NEG                 |                          |                    | N/A                       |
| Cefotaxime/Clavulanate (ESBL)                                                                                                                                                                                                                                                                                                                    | <9             | N/A                 |                          |                    | N/A                       |
| Ceftazidime/Clavulanate (ESBL)                                                                                                                                                                                                                                                                                                                   | <9             | N/A                 |                          |                    | N/A                       |
| Cefpodoxime-proxetil (ESBL)                                                                                                                                                                                                                                                                                                                      | <9             | N/A                 |                          |                    | N/A                       |
| Ceftazidime (ESBL)                                                                                                                                                                                                                                                                                                                               | <9             | N/A                 |                          |                    | N/A                       |
| Ceftriaxone/Clavulanate (ESBL)                                                                                                                                                                                                                                                                                                                   | <9             | N/A                 |                          |                    | N/A                       |

Table 2

| Organism / Микроорганизм / Organizam / Organismus / Organisme / Οργανισμός / Mikroorganizmus / Microorganismo / Организм / Mikroorganizmas / Drobnostrój / Microorganism / Микроорганизм / Organismus / Microorganismo / Mikroorganism / Organisma | Not Reported / Неотечени / Nije zabilježeno / Nenahlášeno / Ikke rapportert / Estamata / Non rendu / Nicht berichtet / Δεν αναφέρεται / Nincs eredmény / Non riportato / Берингерен / Nepranešta / Ikke rapportert / Nie zgłoszono / Não reportado / Nu sunt raportate / He указано /Nisu prijavljeni / Neuvádza sa / No informado / Ej rapporterad / Raportlamnadi | Alternate Method / Alternative metode / Evoluční metódy / Alternatív módszer / Método alternativo / Banama edic / Alternatyvus metodos / Metoda zamienna / Método alternativo / Metodă alternativă / Альтернативный метод / Druga metoda / Alternativna metoda / Alternativ metod / Alternativ Yöntem |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <i>Pseudomonas aeruginosa</i>                                                                                                                                                                                                                      |                                                                                                                                                                                                                                                                                                                                                                     | Piperacillin                                                                                                                                                                                                                                                                                          |
| <i>Morganella morganii</i>                                                                                                                                                                                                                         | Piperacillin                                                                                                                                                                                                                                                                                                                                                        |                                                                                                                                                                                                                                                                                                       |
| <i>Proteus mirabilis</i>                                                                                                                                                                                                                           |                                                                                                                                                                                                                                                                                                                                                                     | Cefazolin, Aztreonam                                                                                                                                                                                                                                                                                  |

Table 3

| Organism / Микроорганизм / Organizam / Organismus / Organisme / Οργανισμός / Mikroorganizmus / Microorganismo / Организм / Mikroorganizmas / Drobnostrój / Microorganism / Микроорганизм / Organismus / Microorganismo / Mikroorganism / Organisma | Not Reported / Неотечени / Nije zabilježeno / Nenahlášeno / Ikke rapportert / Estamata / Non rendu / Nicht berichtet / Δεν αναφέρεται / Nincs eredmény / Non riportato / Берингерен / Nepranešta / Ikke rapportert / Nie zgłoszono / Não reportado / Nu sunt raportate / He указано /Nisu prijavljeni / Neuvádza sa / No informado / Ej rapporterad / Raportlamnadi | Alternate Method / Alternative metode / Evoluční metódy / Alternatív módszer / Método alternativo / Banama edic / Alternatyvus metodos / Metoda zamienna / Método alternativo / Metodă alternativă / Альтернативный метод / Druga metoda / Alternativna metoda / Alternativ metod / Alternativ Yöntem |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <i>Serratia marcescens</i>                                                                                                                                                                                                                         |                                                                                                                                                                                                                                                                                                                                                                     | Colistin                                                                                                                                                                                                                                                                                              |
| <i>Stenotrophomonas maltophilia</i>                                                                                                                                                                                                                | Colistin                                                                                                                                                                                                                                                                                                                                                            |                                                                                                                                                                                                                                                                                                       |