

# BD Onclarity™ Human Papillomavirus (HPV) Liquid-Based Cytology (LBC) Diluent Tubes



8089883(08)  
2021-01  
English

REF 442840 BD Onclarity™ HPV LBC Diluent Tubes

REF 444046 BD Onclarity™ HPV LBC Diluent Tubes

## INTENDED USE

BD Onclarity™ Human Papillomavirus (HPV) Liquid-Based Cytology (LBC) Diluent Tubes are designed to allow for detection of Human Papillomavirus in gynecological specimens that are collected in BD SurePath™ Vial or PreservCyt® Solution and tested with the BD Onclarity HPV Assay on the BD Viper™ LT or BD COR™ Systems using an aliquot that is removed prior to or after processing for gynecological testing.

## REAGENTS

**Materials Provided:** Each BD Onclarity HPV LBC Diluent Tube contains approximately 1.7 mL of a Tris/Sodium Chloride solution with surfactant and preservative.

**Materials Required For Manual Transfer but Not Provided:** Displacement pipettes and polypropylene aerosol-resistant tips capable of delivering  $0.5 \pm 0.05$  mL, vortex mixer, and gloves.

**Materials Required for Automated LBC aliquot transfer using the BD Totalys™ MultiProcessor but Not Provided:** BD Totalys MultiProcessor, BD Syringing Pipettes, vortex mixer, gloves.

**Storage and Handling Requirements:** Reagents may be stored at 2–25 °C. Do not use past the expiration date.

## WARNINGS AND PRECAUTIONS:

1. For *in vitro* Diagnostic Use.
2. For liquid-based cytology specimens, use only the BD Onclarity HPV LBC Diluent Tubes.
3. Do not store specimens in BD Onclarity HPV LBC Diluent Tubes beyond the product expiration date.
4. **Warning**



**H317** May cause an allergic skin reaction.

**H411** Toxic to aquatic life with long lasting effects.

**P261** Avoid breathing dust/fume/gas/mist/vapors/spray. **P272** Contaminated work clothing should not be allowed out of the workplace. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P273** Avoid release to the environment. **P302+P352** IF ON SKIN: Wash with plenty of water. **P333+P313** If skin irritation or rash occurs: Get medical advice/attention. **P321** Specific treatment (see on this label). **P363** Wash contaminated clothing before reuse. **P391** Collect spillage. **P501** Dispose of contents/container to an appropriate treatment and disposal facility in accordance with applicable laws and regulations, and product characteristics at time of disposal.

5. Specimen transfer on BD Totalys MultiProcessor is for BD SurePath™ Vial specimens only.

## SPECIMEN COLLECTION, TRANSFER, STORAGE AND TRANSPORT

Liquid-Based Cytology (LBC) specimens must be collected using either an endocervical broom or a brush/spatula combination as described in the respective manufacturer's product insert. LBC specimens in their original vials can be stored and/or transported for up to 30 days at 2–30 °C, 180 days at 2–8 °C, or 180 days at -20 °C from the date of collection.

A 0.5 mL aliquot of the LBC specimen must be transferred from the original LBC vial to the BD Onclarity HPV LBC Diluent Tube.

Wear gloves when handling the BD Onclarity HPV LBC Diluent Tube and the LBC specimen vial. If gloves come in contact with the specimen, immediately change them to prevent contamination of other specimens.

## LBC Specimen Transfer to BD Onclarity HPV LBC Diluent Tubes - Transfer Processing for the BD SurePath or ThinPrep® PAP Test

### Manual BD SurePath Specimen Transfer Prior to or After Processing for the BD SurePath test

**NOTE:** Refer to the BD PrepStain™ Slide Processor Product Insert or the BD Totalys SlidePrep Product Insert for instructions on removing an aliquot from the BD SurePath Vial prior to performing the BD SurePath Liquid-based Pap Test.

**NOTE:** Handle one specimen at a time for processing.

1. Label a BD Onclarity HPV LBC Diluent Tube with patient identification information.
2. Remove the cap from the BD Onclarity HPV LBC Diluent Tube.
3. In order to ensure a homogenous mixture, vortex the BD SurePath Vial for 10–20 s.
4. Quickly transfer 0.5 mL from the specimen vial using an aerosol-resistant tip to the BD Onclarity HPV LBC Diluent Tube within 1 minute of vortexing.
5. Discard pipette tip.

**NOTE:** A separate pipette tip must be used for each specimen.

6. Tighten the cap on the BD Onclarity HPV LBC Diluent Tube securely.
7. Invert the BD Onclarity HPV LBC Diluent Tube 3 to 4 times to ensure that the specimen and diluent are well mixed.

**Automated BD SurePath Specimen Transfer using BD Totalys MultiProcessor Prior to or After Processing for the BD SurePath test**

1. Shaking BD Onclarity HPV LBC Diluent Tubes may create bubbles above the liquid surface which could interfere with instrument level sensing. To avoid level detection errors, BD Onclarity HPV LBC Diluent Tubes should not be agitated for at least 24 hours prior to loading on the BD Totalys MultiProcessor.
2. In order to ensure a homogenous mixture, vortex the BD SurePath Vial for 8–12 s prior to loading on the BD Totalys MultiProcessor Input Tray.
3. On the BD Totalys MultiProcessor Batch Login screen, select the Aliquot option and “(0.5 mL) HPV” in the dropdown menu. Consult the BD Totalys MultiProcessor User’s Manual for detailed instructions.
4. After removing the BD Onclarity HPV LBC Diluent Tube from the BD Totalys MultiProcessor, invert 3 to 4 times to ensure that the specimen and diluent are well mixed.

**Manual Transfer Only: PreservCyt Specimen Transfer Prior to or After Processing for the ThinPrep Pap test**

**NOTE:** Refer to the ThinPrep 2000/3000 System Operator’s Manual Addendum for instructions on removing an aliquot from the PreservCyt specimen vial prior to performing the ThinPrep test.

**NOTE:** Handle one specimen at a time for processing.

1. Label a BD Onclarity HPV LBC Diluent Tube with patient identification information.
2. Remove the cap from the BD Onclarity HPV LBC Diluent Tube.
3. In order to ensure a homogenous mixture, vortex the PreservCyt specimen vial at high speed for 8–12 s.
4. Immediately transfer 0.5 mL from the specimen vial using an aerosol-resistant tip to the BD Onclarity HPV LBC Diluent Tube.
5. Discard pipette tip.

**NOTE:** A separate pipette tip must be used for each specimen.

6. Tighten the cap on the BD Onclarity HPV LBC Diluent Tube securely.
7. Invert the BD Onclarity HPV LBC Diluent Tube 3 to 4 times to ensure that the specimen and diluent are well mixed.

**LBC SPECIMEN STORAGE AND TRANSPORT IN THE HPV LBC DILUENT TUBE**

Store the LBC specimens that have been transferred to the BD Onclarity HPV LBC Diluent Tube at 2–30 °C for up to 15 days or at -20 °C for up to 90 days. Specimen storage should not exceed the expiration date of the BD Onclarity HPV LBC Diluent Tube.

For domestic and international shipments, specimens should be packaged and labeled in compliance with applicable state, federal, and international regulations covering the transport of clinical specimens and etiologic agents/infectious substances. Time and temperature conditions for storage must be maintained during transport.

Technical Service and Support: Contact your local BD representative or [bd.com](http://bd.com).

## Change History

| Revision | Date    | Change Summary                                                        |
|----------|---------|-----------------------------------------------------------------------|
| (06)     | 2018-04 | Converted printed package insert to electronic insert.                |
| (07)     | 2019-03 | Added BD COR™ System to the package insert.                           |
| (08)     | 2021-01 | Updated Warnings and Precautions section. Made typographical updates. |

Some symbols listed below may not apply to this product.

US Customers only: For symbol glossary, refer to [bd.com/symbols-glossary](http://bd.com/symbols-glossary)



Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbicante / Аткарушы / 제조업체 / Gamintojas / Ražotājs / Tilvirker / Producent / Producător / Производитель / Výrobca / Proizvođač / Tillverkare / Üretici / Виробник / 生产厂商



Use by / Исполняйте до / Spółfeujebiti do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / 사용 기한 / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейін пайдалануға / Naudokite iki / Izlietot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Исползовать до / Použite do / Upotrebiti do / Använd före / Son kulanma tarihi / Використати до / 使用截止日期

YYYY-MM-DD / YYYY-MM (MM = end of month)  
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)  
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)  
AAAA-MM-DD / AAAA-MM (MM = slutning af måned)  
JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)  
EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)  
AAAA-MM-DD / AAAA-MM (MM = fin del mes)  
AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)  
AAAA-MM-JJ / AAAA-MM (MM = fin du mois)  
GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)  
ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja)  
AAAA-MM-GG / AAAA-MM (MM = fine mese)  
ЖЖЖЖ-АА-КК / ЖЖЖЖ-АА / (АА = айдың соңы)  
YYYY-MM-DD/YYYY-MM (MM = 월말)  
MMMM-MM-DD / MMMM-MM (MM = menses pabaiga)  
GGGG-MM-DD/GGGG-MM (MM = menses beigas)  
JJJJ-MM-DD / JJJJ-MM (MM = einde maand)  
AAAA-MM-DD / AAAA-MM (MM = sluten av måneden)  
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)  
AAAA-MM-DD / AAAA-MM (MM = fim do mês)  
AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)  
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)  
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)  
GGGG-MM-DD / GGGG-MM (MM = kraj meseca)  
AAAA-MM-DD / AAAA-MM (MM = slutet av månaden)  
YYYY-AA-GG / YYYY-AA (AA = ayın sonu)  
PPPP-MM-DD / PPPP-MM (MM = кінець місяця)  
YYYY-MM-DD / YYYY-MM (MM = 月末)



Catalog number / Каталоген номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalógusszám / Numero di catalogo / Каталог номер / 카탈로그 번호 / Katalogo / numeris / Kataloga numurs / Catalogus number / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalogové číslo / Kataloški broj / Katalog numarası / Номер за каталогом / 目錄号



Authorized Representative in the European Community / Оторизирани представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret repræsentant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatud esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizuirani predstavnik u Europskoj uniji / Meghatalmazott képviselő az Európai Közösségben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті екіл / 유럽 공동체의 위임 대표 / Įgaliotasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autoriseret representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupce v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Avrupa Topluluğu Yetkili Temsilcisi / Уповноважений представник у країнах ЄС / 歐洲共同体授權代表



In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařizení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiiniparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnosztikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жүргізілетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietaisas / Medicinas ierices, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispozitiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики in vitro / Medicinska pomôcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diyagnostik Tibbi Cihaz / Медицинский пристрій для діагностики in vitro / 体外诊断医疗设备



Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrænsning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperaturi piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны шектеу / 온도 제한 / Laikymo temperatūra / Temperaturas ierobežojumi / Temperaturlimit / Temperaturbegrænsning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ochránenie teploty / Ograničenje temperature / Temperaturgräns / Sicaklık sinirlaması / Обмеження температури / 温度限制



Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Топтама коды / 배치 코드(로트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot nummer / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партии / 批号 (亚批)



Contains sufficient for <n> tests / Съдържа достатъчно за <n> теста / Dostatečné množství pro <n> testů / Indeholder tilstrækkelig til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Küllaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenuto sufficiente per <n> test / <n> тесттері үшін жеткілікті / <n> 테스트가 충분히 포함됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor "n" testen / Innholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Conținut suficient pentru <n> teste / Достаточно для <n> тестов(a) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli mizeme içerir / Вистачить для аналізів: <n> / 足够进行 <n> 次检测



Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / 사용 지침 참조 / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i brugsanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se brugsanvisningen / Kullanım Talimatları'na başvurun / Див. інструкції за використання / 请参阅使用说明



Do not reuse / Не използвайте отново / Nepoužívejte opakovaně / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / No reutilizar / Mitte kasutada korduvalt / Ne pas réutiliser / Ne koristiti ponovo / Egyszer használatos / Non riutilizzare / Пайдаланбаңыз / 재사용 금지 / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolositi / Не использовать повторно / Nepoužívejte opakovaně / Ne upotrebjavajte ponovo / Får ej återanvändas / Tekrar kulanmayin / Не використовувати повторно / 请勿重复使用



Serial number / Серийн номер / Sériové číslo / Seriennummer / Σειριακός αριθμός / N° de serie / Seerianummer / Numéro de série / Serijski broj / Sorozatszám / Numero di serie / Топтамалық нөмірі / 일련 번호 / Serijos numeris / Sérijas numurs / Serie nummer / Numer seryjny / Numéro de série / Număr de serie / Серийный номер / Seri numarası / Номер серії / 序列号

|                                                                                     |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                         |
|-------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|    | For IVD Performance evaluation only / Само за оценка качеството на работа на IVD / Pouze pro vyhodnocení výkonu IVD / Kun til evaluering af IVD ydelse / Nur für IVD-Leistungsbewertungszwecke / Μόνο για αξιολόγηση απόδοσης IVD / Sólo para la evaluación del rendimiento en diagnóstico in vitro / Ainult IVD seadme hindamiseks / Réserve à l'évaluation des performances IVD / Samo u znanstvene svrhe za In Vitro Dijagnostiku / Kizárolag in vitro diagnosztikához / Solo per valutazione delle prestazioni IVD / Жасанды жағдайда «пробирка ішінде» диагностикада тек жұмысты бағалау үшін / IVD 성능 평가에 대해서만 사용 / Tik IVD prietaisų veikimo charakteristikoms tikrinti / Vienīgi IVD darības novērtēšanai / Uitsluitend voor doeltreffendheidsonderzoek / Kun for evaluering av IVD-ytelse / Tiklo do oceny wydajności IVD / Uso exclusivo para avaliação de IVD / Numai pentru evaluarea performanței IVD / Только для оценки качества диагностики in vitro / Určené iba na diagnostiku in vitro / Samo za procenu učinka u in vitro dijagnostici / Endast för utvärdering av diagnostisk användning in vitro / Yalnızca IVD Performans değerlendirmesi için / Тільки для оцінювання якості діагностики in vitro / 仅限 IVD 性能评估 |
|    | For US: "For Investigational Use Only"                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                  |
|    | Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiir / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūros žemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrense / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sıcaklık alt sınırı / Мінімальна температура / 温度下限                                                                                                                                                                                                                                                                                                                                                                                                                                                     |
| <b>CONTROL</b>                                                                      | Control / Контролно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Controllo / Бақылау / 컨트롤 / Kontrolé / Kontrolle / Controle / Controllo / Контроль / kontroll / 对照                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                              |
| <b>CONTROL +</b>                                                                    | Positive control / Положителен контрол / Pozitivní kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitivna kontrola / Pozitív kontroll / Controllo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívá kontrolé / Positieve controle / Kontrola dodatnia / Controllo positivo / Control pozitiv / Положительный контрол / Pozitif kontrol / Позитивний контроль / 阳性对照试剂                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                   |
| <b>CONTROL -</b>                                                                    | Negative control / Отрицателен контрол / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negativna kontrola / Negatív kontroll / Controllo negativo / Негативті бақылау / 음성 컨트롤 / Neigiama kontrolė / Negatívá kontrolé / Negatieve controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контрол / Negatif kontrol / Негативний контроль / 阴性对照试剂                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                            |
| <b>STERILE EO</b>                                                                   | Method of sterilization: ethylene oxide / Метод на стерилизация: етиленов оксид / Způsob sterilizace: etylenoxid / Steriliseringsmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseerimismetod: etüleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etilén-oxid / Metoda di sterilizzazione: ossido di etilene / Sterilizacija edici – етилен тотығы / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksisds / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringsmetode: etylenoksid / Metoda sterylizacji: tlenek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metodá sterilizácie: etylenoxid / Metoda sterilizacije: etilen oksid / Steriliseringsmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: этиленоксидом / 灭菌方法: 环氧乙烷                                                                                                                                                              |
| <b>STERILE R</b>                                                                    | Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Způsob sterilizace: záření / Steriliseringsmetode: bestrålning / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseerimismetod: kiirgus / Méthode de stérilisation : irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metoda di sterilizzazione: irradiazione / Sterilizacija edici – сәуле типі / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringsmetode: bestrålning / Metoda sterylizacji: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: iradiere / Метод стерилизации: облучение / Metodá sterilizácie: ožiarenie / Metoda sterilizacije: ozračevanje / Steriliseringsmetod: strålning / Sterilizasyon yöntemi: ırradyasyon / Метод стерилизації: опроміненням / 灭菌方法: 辐射                                                                                                                                                                                                                                            |
|    | Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologilised riskid / Risques biologiques / Biološki rizik / Biológiai veszélyes / Rischio biologico / Биологическi тәуекелдер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Biologische опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyolojik Riskler / Біологічна небезпека / 生物学风险                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                      |
|   | Caution, consult accompanying documents / Внимание, направете справка в придружаващите документи / Pozor! Pristudujte si příloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatust! Lugeka kaasnevat dokumentatsiooni! / Attention, consulter les documents joints / Urozorenje, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dmesio, žiūrėkite priedamus dokumentus / Piesardziba, skaiti pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Vystraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацію / 小心, 请参阅附带文档。                     |
|  | Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / 상한 온도 / Aukščiausia laikymo temperatūra / Ausgējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrense / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sıcaklık üst sınırı / Міксимальна температура / 温度上限                                                                                                                                                                                                                                                                                                                                                                                                                                             |
|  | Keep dry / Пазете сухо / Skladujte v suchém prostředi / Opbevarer tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / 건조 상태 유지 / Laikykite sausai / Uzglabāt sausu / Droog houden / Holdes tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávaťe v suchu / Držite na suvom mestu / Förvaras tørt / Kuru bir şekilde muhafaza edin / Бергетти від вологості / 请保持干燥                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                        |
|  | Collection time / Време на събиране / Čas odběru / Orpsamlingsstidspunkt / Entnahmeuhrzeit / Ώρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жынау уакыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de colheita / Ora colectării / Време сбора / Doba odboru / Vreme prikupljanja / Uppsamlingsstid / Toplama zamanı / Час забору / 采集时间                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                |
|  | Peel / Обелете / Otevfete zde / Abn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Ўстиғи қабатын алып таста / 벗기 / Pięsti čia / Atfimt / Schillen / Trek av / Oderwać / Destacer / Se dezlipşte / Отклеить / Odrhните / Oljušiti / Dra isär / Ayırma / Відкелити / 撕下                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                             |
|  | Perforation / Перфорация / Perforace / Perforering / Διάτρηση / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Тесик тесу / 찔러친 / Perforacija / Perforácia / Perforatie / Perforacja / Perfuração / Perforare / Перфорация / Perforácia / Perforasyon / Перфорация / 穿孔                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                        |
|  | Do not use if package damaged / Не използвайте, ако опаковката е повредена / Ne používejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packungnicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά / No usar si el paquete está dañado / Mit kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használnia, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Егер пакет бүзылган болса, пайдаланба / 패키지가 손상된 경우 사용 금지 / Jei pakuotė pažeista, nenaudoti / Nelietoti, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Не использовать при повреждении упаковки / Ne použivati, ak je obal poškozený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки / 如果包装破损, 请勿使用                                                                  |
|  | Keep away from heat / Пазете от топлина / Nevystavujte přílišnému teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgusest / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Сапқын жерде сақта / 열을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / Не нагпевать / Uchovávaťe mimo zdroja tepla / Držite dalje od toplote / Fär ej utsättas för värme / Isidan uzak tutun / Бергетти від дії тепла / 请远离热源                                                                                                                                                                                                                                                                                                                                                                                                                                                                                            |
|  | Cut / Срежете / Odstřihněte / Klip / Schneiden / Κόψτε / Cortar / Lőigata / Découper / Reži / Vágja ki / Tagliare / Keciңiz / 잘라내기 / Kirpti / Nogriez / Knippen / Kutt / Odciąć / Cortar / Decupați / Отрезать / Odstřihnite / Iseći / Klipp / Kesme / Pozpizati / 剪下                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                   |



Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmedatum / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuuupäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жинаған тізбекүні / 수집 날짜 / Paëmimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colheita / Data colectării / Дата сбора / Dátum odberu / Datum prikupljanja / Uppsamlingsdatum / Toploma tarihi / Дата забору / 采集日期



µL/test / µL/тест / µL/Test / µL/εξέταση / µL/prueba / µL/teszt / µL/테스트 / мкл/тест / µL/tyrimas / µL/pārbaude / µL/teste / мкл/анализ / µL/检测



Keep away from light / Пазете от светлина / Nevystavujte světlu / Må ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svetla / Fény nem érheti / Tenere al riparo dalla luce / Қараңғыланған жерде ұста / 빛을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Må ikke utsettes for lys / Przechowywać z dala od źródła światła / Manter ao abrigo da luz / Feriți de lumină / Хранить в темноте / Uchovávajte mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Işıktan uzak tutun / Беретти від дії світла / 请远离光线



Hydrogen gas generated / Образуван е водород газ / Možnost úniku plynného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadrží hydrogen vodík / Hidrogén gázt fejleszt / Produzione di gas idrogeno / Газтекес сутері пайда болды / 수소 가스 생성됨 / Išskiria vandenilio dujas / Rodas ūdenradis / Waterstofgas gegenereerd / Hydrogengass generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Выделение водорода / Vyrobené použitím vodíka / Oslobođa se vodonik / Genererad vätgas / Açığa çıkan hidrojen gazı / Реакція з виділенням водню / 会产生氢气



Patient ID number / ИД номер на пациента / ID pacienta / Patientens ID-nummer / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de ID del paciente / Patsiendi ID / No d'identification du patient / Identifikacijski broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттің идентификациялық нөмірі / 환자 ID 번호 / Paciento identifikavimo numeris / Pacienta ID numurs / Identificatienummer van de patiënt / Pasientens ID-nummer / Numer ID pacjenta / Número da ID do doente / Număr ID pacient / Идентификационный номер пациента / Identifikačné číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarası / Идентификатор пациента / 患者标识号



Fragile, Handle with Care / Чупливо, Работете с необходимото внимание. / Křehké. Při manipulaci postupujte opatrně. / Forsigtig, kan gå i stykker. / Zerbrechlich, vorsichtig handhaben. / Εύθραυστο. Χειριστείτε το με προσοχή. / Frágil. Manipular con cuidado. / Öm, kásitsege ettevaatlilikult. / Fragile. Manipuler avec précaution. / Lomljivo, rukujte pažljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura. / Сыныш, абайлап пайдаланыңыз. / 조심 깨지기 쉬운 처리 / Trapu, elkitės atsargiai. / Trausls; rūkotiis uzmanīgi / Breekbaar, voorzichtig behandelen. / Ømtålig, håndter forsigtig. / Krucha zawartość, przenosić ostrożnie. / Frágil, Manuseie com Cuidado. / Frágil, manipulați cu atenție. / Хрупкое! Обращаться с осторожностью. / Křehké, vyžaduje sa opatrná manipulácia. / Lomljivo - rukujte pažljivo. / Bräckligt. Hantera försiktigt. / Kolay Kırılır, Dikkatli Taşıyın. / Тендітна, звертатися з обережністю / 易碎，小心轻放



[bd.com/e-labeling](http://bd.com/e-labeling)



Becton, Dickinson and Company  
7 Loveton Circle  
Sparks, Maryland 21152 USA



Benex Limited  
Pottery Road, Dun Laoghaire  
Co. Dublin, Ireland

#### Australian and New Zealand Sponsors:

Becton Dickinson Pty Ltd.  
66 Waterloo Road  
Macquarie Park NSW 2113, Australia  
Becton Dickinson Limited  
14B George Bourke Drive  
Mt. Wellington Auckland 1060, New Zealand

BD, the BD Logo, COR, Onclarity, PrepStain, SurePath, Totalys, and Viper are trademarks of Becton, Dickinson and company or its affiliates. All other trademarks are the property of their respective owners. © 2021 BD. All rights reserved.