BD

Veritor™ System

For Rapid Detection of Group A Strep

CLIA-waived kit configured for testing throat swab specimens

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2020-04

English

For Rapid Detection of Group A Strep

CLIA Complexity-WAIVED

For use with throat swab specimens.

For *in vitro* diagnostic use only.

A Certificate of Waiver is required to perform this test in a CLIA waived setting. To obtain a Certificate of Waiver, please contact your state health department.

Additional CLIA waiver information is available at the Centers for Medicare and Medicaid website at www.cms.hhs.gov/CLIA or from your state health department.

Failure to follow the instructions or modification to the test system instructions will result in the test no longer meeting the requirements for waived classification.

INTENDED USE

The BD Veritor™ System for Rapid Detection of Group A Strep is a rapid chromatographic immunoassay for the direct and qualitative detection of Group A *Streptococcus* antigen from throat swabs of symptomatic patients. It is intended to be used in conjunction with a BD Veritor instrument as an aid in the diagnosis of Group A Strep. All negative test results should be confirmed by bacterial culture because negative results do not preclude Group A Strep infection and should not be used as the sole basis for treatment.

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All BD Veritor System Strep A test devices are interpreted by a BD Veritor System instrument, either a BD Veritor System Reader or BD Veritor Plus Analyzer (the "Analyzer"). When using the BD Veritor Plus Analyzer, workflow steps depend on the selected operational mode and the Analyzer configuration settings. In **Analyze Now mode**, the instrument evaluates assay devices after manual timing of their development. In **Walk Away mode**, devices are inserted immediately after application of the specimen, and timing of assay development and analysis is automated. Connection of the Analyzer to a printer is possible if desired. Additional result documentation capabilities are possible with the implementation of the BD Synapsys™ Informatics Solution, and with the addition of the BD Veritor InfoScan module and BD Veritor Plus Connect. Please refer to the Analyzer *Instructions for Use* for details on these features and contact BD Technical Support for more information.

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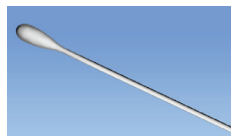
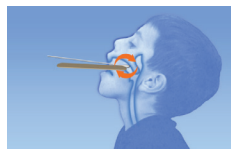
1. The BD Veritor System Group A Strep Kit includes sterile swabs with a rayon tip for throat specimen collection.

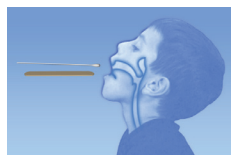
2. Have the patient open his or her mouth. Depress the tongue completely with a tongue depressor.

3. Swab the posterior pharynx, tonsils, and other inflamed areas. Avoid touching the tongue, cheeks, and teeth with the swab.

4. Withdraw the swab from the mouth. The sample is now ready for processing using the BD Veritor System Group A Strep Kit.

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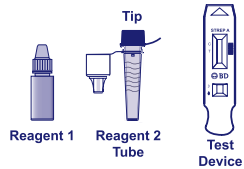
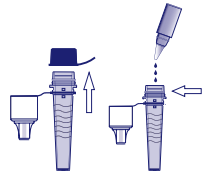
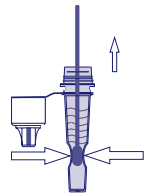
Step 3

- Insert the specimen or control swab and incubate for 1–2 minutes, then plunge the swab up and down for a minimum of 15 seconds, scrubbing inside the tube with the swab. Avoid splashing.

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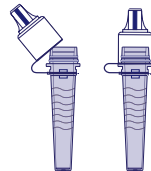
Step 5

- Snap fit the tip onto the tube containing the processed specimen or control (threading/ twisting not required).
- Note: Do not use tips from any other product, including other products from BD or other manufacturers.

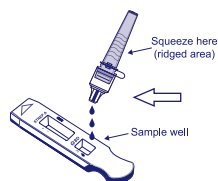
| After step 5, choose from the model and workflow option below before continuing to step 6: | | | | |
|--|---|---|--|----------|
| | BD Veritor System Reader or Analyzer in Analyze Now mode | BD Veritor Plus Analyzer in Walk Away mode | BD Veritor Plus Analyzer with the BD Veritor InfoScan module in Analyze Now mode—or Walk Away mode | |
| Instructions in section: | A | B | C | D |

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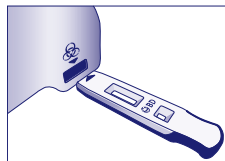
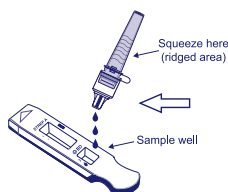
- When the display window reads "ADD SPECIMEN TO TEST DEVICE AND INSERT IMMEDIATELY":
– Invert the tube, holding it vertically (approximately one inch above the BD Veritor System Strip A device sample well).
– Gently squeeze the ridged portion of the tube, allowing three (3) drops of the processed specimen to dispense into the sample well of a labeled BD Veritor System Strip A device.
- **NOTE:** Squeezing the tube close to the tip may cause leakage.
- **CAUTION:** A countdown timer displays the time remaining for test insertion. **Walk Away mode must be activated again when this timer expires. Confirm timer is visible and Walk Away mode is activated before inserting test device.**

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| Clinical Performance Data – By Site | | | | |
|---|------------|---------|-----|-------|
| | | Culture | | Total |
| Site Code | BD Veritor | P | N | |
| Clinical Site | P | 20 | 2 | 22 |
| | N | 0 | 82 | 82 |
| Total | | 20 | 84 | 104 |
| Reference Method: Culture Sensitivity: 100% (95% CI: 83.9%, 100%) Specificity: 97.6% (95% CI: 91.7%, 99.3%) | | | | |
| POC 1 | P | 54 | 3 | 57 |
| | N | 5 | 188 | 193 |
| Total | | 59 | 191 | 250 |
| Reference Method: Culture Sensitivity: 91.5% (95% CI: 81.6%, 96.3%) Specificity: 98.4% (95% CI: 95.5%, 99.5%) | | | | |
| POC 2 | P | 21 | 9 | 30 |
| | N | 0 | 111 | 111 |

| Clinical Performance Data – By Site | | | | |
|---|------------|---------|-----|-------|
| | | Culture | | Total |
| Site Code | BD Veritor | P | N | |
| Total | | 21 | 120 | 141 |
| Reference Method: Culture Sensitivity: 100% (95% CI: 84.5%, 100%) Specificity: 92.5% (95% CI: 86.4%, 96.0%) | | | | |
| POC 3 | P | 21 | 7 | 28 |
| | N | 0 | 106 | 106 |
| Total | | 21 | 113 | 134 |
| Reference Method: Culture Sensitivity: 100% (95% CI: 84.5%, 100%) Specificity: 93.8% (95% CI: 87.8%, 97.0%) | | | | |
| POC 4 | P | 28 | 8 | 36 |
| | N | 0 | 131 | 131 |
| Total | | 28 | 139 | 167 |
| Reference Method: Culture Sensitivity: 100% (95% CI: 87.9%, 100%) Specificity: 94.2% (95% CI: 89.1%, 97.1%) | | | | |

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| Substance | Concentration Tested |
|-----------------------------------|----------------------|
| 4-Acetamidophenol | 10 mg/mL |
| Acetylsalicylic acid | 20 mg/mL |
| Albuterol | 0.083 mg/mL |
| Amantadine | 500 ng/mL |
| Ascorbic acid chewable tablets | 5% by weight |
| Beclomethasone | 500 ng/mL |
| Benzocaine throat spray (Cepacol) | 5% by volume |
| Blood, type A | 2% v/v |
| Blood, type B | 2% v/v |
| Blood, type AB | 2% v/v |
| Blood, type O | 2% v/v |
| Budesonide | 500 ng/mL |
| Chlorpheniramine maleate | 5 mg/mL |
| Dexamethasone | 10 mg/mL |
| Dextromethorphan | 10 mg/mL |
| Dyclonine HCl lozenges (Sucrets) | 5% w/v |
| Diphenhydramine HCl | 5 mg/mL |
| Fexofenadine | 500 ng/mL |
| FluMist™ | 1% v/v |
| Fluticasone | 500 ng/mL |
| Guaiacol Glyceryl Ether | 20 mg/mL |
| Ibuprofen | 10 mg/mL |
| Loratidine | 100 ng/mL |

| Substance | Concentration Tested |
|---|----------------------|
| Menthol Throat Lozenges | 5% w/v |
| Mometasone | 500 ng/mL |
| Mouthwash Listerine | 5% v/v |
| Mouthwash Scope | 5% v/v |
| Mouthwash CVS | 5% v/v |
| Mucin, salivary protein, purified | 1 mg/mL |
| Nasal Spray | 5% v/v |
| Nasal Spray | 5% v/v |
| Nasal Spray | 5% v/v |
| Oseltamivir | 500 ng/mL |
| Oxymetazoline | 0.05 mg/mL |
| Phenol throat spray (Chloraseptic) | 5% v/v |
| Phenylephrine | 1 mg/mL |
| Pseudoephedrine HCl | 20 mg/mL |
| Throat drops: CVS | 5% w/v |
| Throat drops: Pedia Care | 5% w/v |
| Throat drops: Triaminic | 5% w/v |
| Tobramycin | 500 ng/mL |
| Triamcinolone | 500 ng/mL |
| Zanamivir | 1 mg/mL |
| Zicam throat spray (Zn / benzalkonium chloride) | 5% v/v |
| Zinc Lozenges | 5% w/v |

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| BD Veritor Group A Strep Reproducibility | | | | | | | | |
|--|---------------|--------------|---------------|--------------|---------------|--------------|---------------|--------------|
| Sample | Site 1 | | Site 2 | | Site 3 | | Total | |
| | % Pos | 95% CI | % Pos | 95% CI | % Pos | 95% CI | % Pos | 95% CI |
| High negative | 3.3% (1/30) | 0.6%, 16.7% | 0.0% (0/30) | 0.0%, 11.3% | 0.0% (0/30) | 0.0%, 11.3% | 1.1% (1/90) | 0.2%, 6.0% |
| Low positive | 96.7% (29/30) | 83.3%, 99.4% | 83.3% (25/30) | 66.4%, 92.7% | 93.3% (28/30) | 78.7%, 98.2% | 91.1% (82/90) | 83.4%, 95.4% |
| Moderate positive | 100% (30/30) | 88.6%, 100% | 96.7% (29/30) | 83.3%, 99.4% | 100% (30/30) | 88.6%, 100% | 98.9% (89/90) | 94.0%, 99.8% |
| Negative | 0.0% (0/30) | 0.0%, 11.3% | 0.0% (0/30) | 0.0%, 11.3% | 0.0% (0/30) | 0.0%, 11.3% | 0.0% (0/90) | 0.0%, 4.1% |

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For Rapid Detection of Group A Strep

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Change History

| Revision | Date | Change Summary |
|----------|---------|--|
| (11) | 2019-01 | Made grammatical and content changes to provide more clarity in usage instructions. Clarified brand name references of BD Veritor Plus Analyzer. |
| (12) | 2020-04 | Changed name of BD Synapsys™ Microbiology Informatics Solution to BD Synapsys™ Informatics Solution. Additional result documentation capabilities made possible with the implementation of the BD Synapsys Informatics Solution, and with the addition of the BD Veritor InfoScan module and BD Veritor Plus Connect (replacing the BD Veritor System Reader and the BD InfoSync Module). In sections C and D, revised Steps 10C and 10D regarding the use of the BD Veritor Plus Analyzer with the BD Veritor InfoScan. Added access information to obtain the document from bd.com/e-labeling . |

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US Customers only: For symbol glossary, refer to bd.com/symbols-glossary

Manufacturer / Производител / Yörobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabricante / Аткарушы / 제조업체 / Gamintojas / Ražotājs / Tilvirkir / Producent / Producător / Производител / Vyrobca / Proizvođač / Tilverkare / Üretici / Виробник / 生产厂商

Use by / Използвайте до / Spotføjte do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / 사용 기한 / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейін пайдалануға / Naudokite iki / Izljetiot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Použite do / Upotrebiti do / Använd före / Son kullanna tarhi / Використати до / 使用截止日期

YYYY-MM-DD / YYYY-MM (MM = end of month)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = slutning af måned)
JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)
EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)
AAAA-MM-DD / AAAA-MM (MM = fin del mes)
AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)
AAAA-MM-JJ / AAAA-MM (MM = fin du mois)
GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)
EEEE-HH-NN / EEEE-HH (HH = hónap utolsó napja)
AAAA-MM-GG / AAAA-MM (MM = fine mese)
ЖЖЖЖ-АА-КК / ЖЖЖЖ-АА / (АА = айдың соңы)
YYYY-MM-DD/YYYY-MM (MM = 월말)
MMMM-MM-DD / MMMM-MM (MM = mensesio pabaiga)
GGGG-MM-DD/GGGG-MM (MM = mēneša beigas)
JJJJ-MM-DD / JJJJ-MM (MM = einde maand)
AAAA-MM-DD / AAAA-MM (MM = sluten av måneden)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = fim do mês)
AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)
RRRR-MM-DD / RRRR-MM (ММ = koniec miesiąca)
GGGG-MM-DD / GGGG-MM (ММ = kraj meseca)
AAAA-MM-DD / AAAA-MM (ММ = slutet av månaden)
YYYY-AA-GG / YYYY-AA (AA = ayın sonu)
PPPP-MM-DD / PPPP-MM (MM = кінець місяця)
YYYY-MM-DD / YYYY-MM (MM = 月末)

Catalog number / Каталоген номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalógusszám / Numero di catalogo / Каталог нөмірі / 카탈로그 번호 / Katalogo / numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalogové číslo / Kataloški broj / Katalog numarası / Номер за каталогом / 目录号

Authorized Representative in the European Community / Оторизирован представитель в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret repræsentant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatud esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizuirani predstavnik u Evropskoj uniji / Meghatalmazott képviselő az Európai Közösségekben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті өкіл / 유럽 공동체의 위임 대표 / Įgaliojasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autoriseret representant i EU / Autorizovaný zástupce predstaviteľstvo we Wspólnotie Europejskiej / Reprezentante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupca v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Avrupa Topluluğu Yetkili Temsilcisi / Упоноважений представник у країнах ЄС / 欧洲共同体授权代表

IVD

In Vitro Diagnostic Medical Device / Медицинский уред за диагностика ин витро / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διάγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiniparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnosztikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жұргізілетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietaisai / Medicinas ierices, ko lieto in vitro diagnostika / Medicisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medicinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispozitiv medical pentru diagnostic in vitro / Medicinska priobor dlya diagnostiki in vitro / Medicinska pomôcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Mediceinteknisk produkt för in vitro-diagnostik / In Vitro Diyagnostik Tibbi Cihaz / Медицинский прибор для диагностики ин витро / 体外诊断医疗设备

Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrænsning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperaturi piliang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны шектеу / 온도 제한 / Laikymo temperatūra / Temperatūras ierobežojumi / Temperatuurlimiet / Temperaturbegrænsning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ohraničenje teploty / Ograničenje temperature / Temperaturgräns / Sicaklık sınırlaması / Обмеження температури / 温度限制

LOT

Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-code (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (lot) / Codice batch (lotto) / Топтамa коды / 배치 코드 (로트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot nummer / Batch-code (parti) / Kod partii (serie) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партии / 批号 (亚批)

Contains sufficient for <n> tests / Содержитo достаточнo для <n> тестa / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Källaladane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> testzhez elegendő / Contenido suficiente per <n> test / <n> testleri için yeterli / <n> 테스트가 충분히 포함됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor "n" testen / Innholder tilstrækkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Conținut suficient pentru <n> teste / Достаточно для <n> тестов(a) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli miktarda içerir / Вистачить для аналізів: <n> / 足够进行 <n> 次检测

Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / 사용 지침 참조 / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i bruksanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se bruksanvisningen / Kullanım Talimatları'na başvurun / Див. інструкції з використання / 请参阅使用说明

Do not reuse / Не используйте отново / Neopouzívejte opakovaně / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / No reutilizar / Mitte kasutada korduvalt / Ne pas réutiliser / Ne koristiti ponovo / Egyeszer használatos / Non riutilizzare / Пайдаланбаңыз / 재사용 금지 / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolositi / Ne использовать повторно / Neopuzivajte opakovane / Ne upotrebļavajate ponovo / Får ej återanvändas / Tekrar kulanmayın / Не використовувати повторно / 请勿重复使用

SN

Serial number / Серийн номер / Sériové číslo / Seriennummer / Seriennummer / Σειριακός αριθμός / Nº de serie / Seerianummer / Numéro de série / Serijski broj / Sorozatszám / Numero di serie / Топтамалық нөмірі / Serijos numeris / Serijas numurs / Serial number / Numer serijny / Número de série / Număr de serie / Серийный номер / Seri numarasi / Номер серий / 序列号

For IVD Performance evaluation only / Само за оценка качествo на работа на IVD / Pouze pro vyhodnocení výkonu IVD / Kun til evaluering af IVD ydelse / Nur für IVD-Leistungsbewertungszwecke / Μόνο για αξιολόγηση απόδοσης IVD / Sólo para la evaluación del rendimiento en diagnóstico in vitro / Ainult IVD seadme hindamiseks / Réserve à l'évaluation des performances IVD / Samo u znanstvene svrhe za In Vitro Dijagnostiku / Kizárólag in vitro diagnosztikához / Solo per valutazione delle prestazioni IVD / Жасанды жағдайда «пробирка ішінде» диагностикада тек жұмысты бағалау үшін / IVD 성능 평가에 대해서만 사용 / Tik IVD prietaisų veikimo charakteristikoms tikrinti / Vienigi IVD darības novērtēšanai / Uitsluitend voor doeltreffendheidsonderzoek / Kun for evaluering av IVD-ytelse / Tylko do oceny wydajności IVD / Uso exclusivo para avaliação de IVD / Numai pentru evaluarea performanței IVD / Только для оценки качества диагностики ин витро / Určene iba na diagnostiku in vitro / Samo za procenu učinka u in vitro dijagnostici / Endast för utvärdering av diagnostisk användning in vitro / Yalnızca IVD Performans değerlendirmesi için / Типы для оцінювання якості діагностики ин витро / 仅限 IVD 性能评估

For US: "For Investigational Use Only"

Lower limit of temperature / Дoлeн лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inférieure de température / Almine temperaturpiir / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrænse / Dolna granica temperatury / Limite minimo de temperatura / Limitā minimā de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sicaklık alt sınırı / Мінімальна температура / 温度下限

CONTROL

Control / Контрольно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Controllo / Бақылау / 컨트롤 / Kontrolé / Kontrolle / Control / Controllo / Контроль / kontroll / Kontrolle / 对照

CONTROL+

Positive control / Положительный контроль / Pozitívni kontrol / Positiv kontrol / Positive Kontrolle / Επικτός μάρτυρας / Control positivo / Positive kontroll / Contrôle positif / Pozitívna kontrola / Pozitiv kontroll / Controllo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívna kontrola / Positive controle / Kontrola dodatna / Controllo positivo / Control pozitiv / Положительный контроль / Pozitif kontrol / Позитивный контроль / 阳性对照试剂

CONTROL-

Negative control / Отрицательный контроль / Negatívni kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negative kontroll / Contrôle négatif / Negatívna kontrola / Negativ kontroll / Controllo negativo / Негативтік бақылау / 음성 컨트롤 / Neigiama kontrolė / Negatívna kontrola / Negative controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatív kontrol / Негативный контроль / 阴性对照试剂

| | |
|--|--|
|  | <p>Method of sterilization: ethylene oxide / Метод на стерилизација: етиленов оксид / Způsob sterilizace: etylenoxid / Steriliseringsmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλenoξειδιο / Método de esterilización: óxido de etileno / Steriliseringsmetode: etüleenoksiid / Méthode de stérilisation: oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Стерилизација едичи – этилен тотгыш / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksīds / Gesteriliseerd met behulp van ethylenoxide / Steriliseringsmetode: ethylenoxid / Metoda sterylizacji: tlenek etylu / Método de esterilização: óxido de etileno / Metóda de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metóda sterilizácie: etylenoxid / Metoda sterilizacije: etilen oksid / Steriliseringsmetode: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: етиленоксидом / 灭菌方法: 环氧乙烷</p> |
|  | <p>Method of sterilization: irradiation / Метод на стерилизација: ирадиација / Způsob sterilizace: záření / Steriliseringsmetode: bestråling / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseringsmetode: klirgus / Méthode de stérilisation: irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metoda di sterilizzazione: irradiazione / Стерилизација едичи – сауле түсірі / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringsmetode: bestråling / Metoda sterylizacji: napromienianie / Método de esterilização: irradiação / Metóda de sterilizare: irradiação / Метод стерилизации: ирадиере / Метод стерилизации: облучение / Metóda sterilizácie: ožiarenie / Metoda sterilizacije: ozračevanje / Steriliseringsmetode: strålning / Sterilizasyon yöntemi: irradyasyon / Метод стерилизації: опроміненням / 灭菌方法: 辐射</p> |
|  | <p>Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologilised riskid / Risques biologiques / Biološki rizik / Biológiai veszélyes / Rischio biologico / Биологичны тауекелдер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyolojik Riskler / Биологічна небезпека / 生物学风险</p> |
|  | <p>Caution, consult accompanying documents / Внимание, направете справка в придружаващите документи / Pozor! Prostudujte si příložený dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatust! Lugeda kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Opozorenje, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dmesio, züürékite pridamdas dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацию / 小心, 请参阅附带文档。</p> |
|  | <p>Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrense / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температурни рурас етілен жоғарғы шеі / 상한 온도 / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrense / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sicaklık üst sınırı / Максимальна температура / 温度上限</p> |
|  | <p>Keep dry / Пазете сухо / Skladujte v suchém prostredí / Opbevarer tørt / Trocklagen / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / 건조 상태 유지 / Laiykite sausu / Uzglabāt sausu / Droog houden / Hoides tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávať v suchu / Držite na suvom mestu / Förvaras torrt / Kuru bir şekilde muhafaza edin / Бергити від вологи / 请保持干燥</p> |
|  | <p>Collection time / Време на събиране / Čas odběru / Opsamlingstidspunkt / Entnahmeuhrzeit / Ώρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жинау уакыты / 수집 시간 / Raėmimo laikas / Savākšanas laiks / Verzameltijd / Godzina pobrania / Hora de coleta / Ora colectării / Время сбора / Doba odboru / Vreme prikupljanja / Uppsamlingsstid / Toplama zamanı / Час забору / 采集时间</p> |
|  | <p>Peel / Обелете / Otefete zde / Ábn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Ψήκνι κάбатын алын таста / 벗기기 / Plești ă / Atilmlē / Schillen / Trekv af / Oderwać / Destaccar / Se deslizește / Отклеити / Odrhните / Oljuštiti / Dra isår / Ayırma / Відклеїти / 撕下</p> |
|  | <p>Perforation / Перфорация / Perforance / Perforering / Διήτρηση / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Tesik tesy / 결핵선 / Perforacija / Perforăcija / Perforatie / Perforacja / Perfuração / Perforare / Перфорация / Perforăcia / Perforasyon / Перфорация / 穿孔</p> |
|  | <p>Do not use if package damaged / Не използвайте, ако опаковката е повредена / Nepoužívejte, je-li obal poškozený / Má ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packungnicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά / Ne usar si el paquete está dañado / Mitte kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használni, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Eger paket buzylgan bolca, pайдaлaнбa / 패키지가 손상된 경우 사용 금지 / Jei pakuotė pažeista, nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Má ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Не использовать при повреждении упаковки / Nepoužívať, ak je obal poškodený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки / 如果包装破损, 请勿使用</p> |
|  | <p>Keep away from heat / Пазете от топлина / Nevstavujte přílišnému teplu / Má ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgeste / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Сапкын жерде сақта / 열을 피해야 함 / Laiykiti atokiau nuo šilumos šaltinių / Sargát n karstuma / Beschermen tegen warmte / Má ikke utsettes for varme / Przechowywać z dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / Не награвать / Uchovávať mimo zdroja tepla / Držite dalje od toplote / Får ej utsättas för värme / Isidan uzak tutun / Бергити від тепла / 请远离热源</p> |
|  | <p>Cut / Срежете / Odstrhněte / Klip / Schneiden / Kórt / Cortar / Lõigata / Découper / Reži / Vágja ki / Tagliare / Keciňati / 잘라내기 / Kirpti / Nogriez / Knippen / Kutt / Odciać / Cortar / Decupați / Отрезать / Odstrhните / Iseći / Klipp / Kesme / Розрізати / 剪下</p> |

Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmetermin / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuupäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жинаған тізбекүні / 수집 날짜 / Paëmimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colecta / Data colectării / Data сбора / Datum odberu / Datum prikupljanja / Urpsamlingsdatum / Toplama tarihi / Дата забору / 采集日期
 μL/test / μL/тест / μL/Test / μL/тэста / μL/prueba / μL/teszt / μL/테스트 / мкл/тест / μL/tyrimas / μL/pārbaude / μL/teste / мкл/анализ / μL/检测

Keep away from light / Παзете от светлина / Nevystavujte světlu / Må ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svetla / Fény nem érheti / Tenere al riparo dalla luce / Қазақшылған жерде ұста / 빛을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Må ikke utsettes for lys / Przechowywać z dala od źródeł światła / Manter ao abrigo da luz / Feriți de lumină / Хранить в темноте / Uchovávať mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Işıktan uzak tutun / Берегти від дії світла / 请远离光线

Hydrogen gas generated / Образован в водород газ / Možnost úniku plynného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadržaji hydrogen vodik / Hidrogen gázt fejeleszt / Produzione di gas idrogeno / Газтекес цутери пайда болды / 수소 가스 생성됨 / Išskiria vandenilio dujas / Rodas vandenilis / Waterstofgas gegenereerd / Hydrogengas generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Vyrobené použitím vodíka / Oslobađa se vodonik / Genererad vätegas / Açığa çıkan hidrojen gazı / Реакція з виділенням водню / 会产生氢气

Patient ID number / ID номер на пациента / ID pacienta / Patients ID-number / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de ID del paciente / Patsiendi ID / No d'identification du patient / Identifikacijski broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттің идентификациялық нөмірі / 환자 ID 번호 / Paciento identifikavimo numeris / Pacienta ID numurs / Identificationsnummer van de patiënt / Patients ID-number / Nummer ID pacienta / Número da ID do doente / Număr ID pacient / Идентификационный номер пациента / Identifikačné číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarasi / Идентификатор пациента / 患者标识号

Fragile, Handle with Care / Чупливо, Роботете с необходимого внимание. / Křehké. Při manipulaci postupujte opatrně. / Forsigtig, kan gå i stykker. / Zerbrechlich, vorsichtig handhaben. / Εύθραστο. Χειράριστε με προσοχή. / Frágil. Manipulær con cuidado. / Öm, k sitsege ettevaatlikult. / Fragile. Manipuler avec pr caution. / Lomljivo, rukujte pa ljivo. / T r keny!  vatosan kezelend . / Fragile, maneggiare con cura. / Сыныш, абайлап пайдаланыңыз. / 조심 깨지기 쉬운 처리 / Trapu, elikites atsargiai. / Trausis; rikoties uzmanigi / Breekbaar, voorzichtig behandelen. /  mt lig, h ndter forsigtig. / Kruha zawarto c, przenosi c ostro nie. / Fr gil, Manuseie com Cuidado. / Fr gil, manipula i cu aten ie. / Хрункоел Обращаться с осторожностью. / Křehk , vy aduje sa opatrn  manipul cia. / Lomljivo - rukujte pa ljivo. / Br ckligt. Hantera f rsiktigt. / Kolay Kırılır, Dikkatli Ta şın. / Т нд тна, звертатися з обережністю / 易碎, 小心轻放

R_x Only

This only applies to US: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / S'applique uniquement aux  tats-Unis: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Vale solo per gli Stati Uniti: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Gilt nur f r die USA: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / S lo se aplica a los EE.UU.: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner."

 bd.com/e-labeling

| Europe, CH, GB, NO: | | +800 135 79 135 |
|---------------------|----------------------|----------------------|
| International: | | +31 20 794 7071 |
| AR | +800 135 79 135 | LT 8800 30728 |
| AU | +800 135 79 135 | MT +31 20 796 5693 |
| BR | 0800 591 1055 | NZ +800 135 79 135 |
| CA | +1 855 805 8539 | RO 0800 895 084 |
| CO | +800 135 79 135 | RU +800 135 79 135 |
| EE | 0800 0100567 | SG 800 101 3366 |
| GR | 08000 161 22015 7799 | SK 0800 606 287 |
| HR | 0800 804 804 | TR 08000 142 064 866 |
| IL | +800 135 79 135 | US +1 855 236 0910 |
| IS | 800 8996 | UY +800 135 79 135 |
| LI | +31 20 796 5692 | VN 122 80297 |

Becton, Dickinson and Company
 7 Loveton Circle
 Sparks, Maryland 21152 USA

Benex Limited
 Pottery Road, Dun Laoghaire
 Co. Dublin, Ireland

BD, the BD logo, BBL, CultureSwab, Synapsys, and Veritor are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners.   2020 BD. All rights reserved.

Insert ends here

DO NOT PRINT THIS PAGE

CLSI document should not be included in print file to vendor.

NOTES:

The CLSI document is to be updated at the same time as the package insert and should be released as a separate PDF document to the web.

The package insert should also be released as a separate PDF document to the web.

The package insert and CLSI document should be released as one document to SAP.

Final art provided to Buyer/print vendor should not include this page or the CLSI portion of this document.

Proofreader: please proof the CLSI document along with the package insert.

Procedure* BD Veritor™ System For Rapid Detection of Group A Strep

For use with throat swab specimens.

| Prepared by | Date Adopted | Supersedes Procedure # |
|-------------|--------------|------------------------|
| | | |

| Review Date | Revision Date | Signature |
|-------------|---------------|-----------|
| | | |
| | | |
| | | |
| | | |

| Distributed to | # of Copies | Distributed to | # of Copies |
|----------------|-------------|----------------|-------------|
| | | | |
| | | | |
| | | | |
| | | | |

***Any modifications to this document are the sole responsibility of the facility. This “Sample Procedure” is not intended as a substitute for your facility procedure manual, instrument manual, or reagent labeling/package insert. This “Sample Procedure” is intended as a model for use by your facility to meet the needs of your laboratory.**

CLSI For Rapid Detection of Group A Strep

INTENDED USE

The BD Veritor System for Rapid Detection of Group A Strep is a rapid chromatographic immunoassay for the direct and qualitative detection of Group A *Streptococcus* antigen from throat swabs of symptomatic patients. It is intended to be used in conjunction with a BD Veritor instrument as an aid in the diagnosis of Group A Strep. All negative test results should be confirmed by bacterial culture because negative results do not preclude Group A Strep infection and should not be used as the sole basis for treatment.

The BD Veritor System for Rapid Detection of Group A Strep test is intended for use in point-of-care or laboratory settings.

SUMMARY AND EXPLANATION

Streptococcus pyogenes is a gram-positive coccus, which contains the Lancefield group A antigen that can cause serious infections such as pharyngitis, respiratory infection, impetigo, endocarditis, meningitis, puerperal sepsis, and arthritis.¹ Left untreated, these infections can lead to serious complications, including rheumatic fever and peritonsillar abscess.² Traditional identification procedures for group A streptococcal infection involve the isolation and identification of viable organisms using techniques that require 24 to 48 hours or longer.³

Rapid diagnosis and early antibiotic therapy of group A streptococcal infection appear to be the best means of preventing medical complications and reducing the spread of the disease.⁴ The BD Veritor System for Rapid Detection of Group A Strep is a digital immunoassay to qualitatively detect the presence of Strep A antigen in throat swab specimens from symptomatic patients, providing results within 5 minutes. The test utilizes antibodies specific for whole cell Lancefield group A *Streptococcus* to selectively detect Strep A antigen.

All BD Veritor System Strep A test devices are interpreted by a BD Veritor System instrument, either a BD Veritor System Reader or BD Veritor Plus Analyzer (the "Analyzer"). When using the BD Veritor Plus Analyzer, workflow steps depend on the selected operational mode and the Analyzer configuration settings. In Analyze Now mode, the instrument evaluates assay devices after manual timing of their development. In Walk Away mode, devices are inserted immediately after application of the specimen, and timing of assay development and analysis is automated. Connection of the Analyzer to a printer is possible if desired. Additional result documentation capabilities are possible with the implementation of the BD Synapsys™ Informatics Solution, and with the addition of the BD Veritor InfoScan module and BD Veritor Plus Connect. Please refer to the Analyzer *Instructions for Use* for details on these features and contact BD Technical Support for more information.

PRINCIPLES OF THE PROCEDURE

The BD Veritor System for Rapid Detection of Group A Strep is a qualitative, digital immunoassay for the detection of Strep A antigen in a throat swab. In this test, antibody specific to Strep A antigen is coated on the test line region of the Assay device. During testing, the processed throat swab specimen reacts with an antibody to Strep A that is conjugated onto detector particles. The mixture migrates up the membrane and is captured by the line of antibody on the membrane. A positive result for Strep A is determined by the BD Veritor System instrument when antigen-conjugate is deposited at the Test "T" position and the Control "C" position on the BD Veritor System Strep A assay device. The instrument analyzes and corrects for non-specific binding and detects positives not recognized by the unaided eye to provide an objective digital result.

REAGENTS

The following components are included in the BD Veritor System for Rapid Detection of Group A Strep (GAS) kit:

| | | |
|---|------------------------------|---|
| BD Veritor System Group A Strep Devices | 30 devices | Foil pouched device containing one reactive strip. Each strip has one test line of polyclonal antibody specific to Strep A antigen and a positive control line containing purified Strep A antigen. |
| BD GAS Reagent 1 | Bottle with 4 mL reagent | Dilute acetic acid solution |
| BD GAS Reagent 2 | 30 tubes with 200 µL reagent | Sodium nitrite and EDTA |
| Individually packaged swabs, sterile | 30 each | Swab for throat specimen collection |
| Positive Control Swab | 1 each | Strep A Positive Control Swab (purified Strep A antigen) with <0.1% sodium azide (preservative) |
| Negative Control Swab | 1 each | Strep A Negative Control Swab with <0.1% sodium azide (preservative) |

Materials Required But Not Provided: BD Veritor™ Plus Analyzer (Cat. No. 256066), Timer, Tube Rack for specimen testing.

Optional Equipment: BD Veritor™ InfoScan Module (Cat. No. 256068), USB Printer Cable for BD Veritor™ Analyzer (Cat. No. 443907), Epson Printer model TM-T20 II, BD Veritor Plus Connect (contact BD Technical Services for details).

Warnings and Precautions:

Gas Reagent 1

Warning

H315 Causes skin irritation.

P280 Wear protective gloves/protective clothing/eye protection/face protection. **P264** Wash thoroughly after handling. **P321** Specific treatment (see on this label). **P332+P313** If skin irritation occurs: Get medical advice/attention. **P302+P352** IF ON SKIN: Wash with plenty of soap and water. **P362+P364** Take off contaminated clothing and wash it before reuse. **P403** Store in a well-ventilated place. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

Gas Reagent 2

Warning

H302 Harmful if swallowed. **H401** Toxic to aquatic life.

P273 Avoid release to the environment. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P301+P312** IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. **P330** Rinse mouth. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

1. For *in vitro* Diagnostic Use.
2. Test results are not meant to be visually determined. **All test results must be determined using the BD Veritor System instrument.**
3. Pathogenic microorganisms, including hepatitis viruses, and Human Immunodeficiency Virus may be present in clinical specimens.⁵ "Standard Precautions"⁵⁻⁸ and institutional guidelines should be followed in handling, storing and disposing of all specimens and all items contaminated with blood and other body fluids.
4. Dispose of used BD Veritor System test devices as biohazardous waste in accordance with federal, state and local requirements.
5. Reagents contain sodium azide, which is harmful if inhaled, swallowed or exposed to skin. Contact with acids produces very toxic gas. If there is contact with skin, wash immediately with plenty of water. Sodium azide may react with lead and copper plumbing to form highly explosive metal azides. On disposal, flush with a large volume of water to prevent azide build-up.
6. Only use the reagents provided with the kit for preparation. Do not mix components from different kit lots.
7. Other than the swabs that are used for specimen collection, kit components should not make contact with the patient.
8. Do not use kit components beyond the expiration date.
9. Do not reuse the device.
10. Do not use the kit if the Positive Control Swab and Negative Control Swab do not yield appropriate results.
11. Wear protective clothing such as laboratory coats, disposable gloves and eye protection when specimens are assayed.
12. To avoid erroneous results, swab specimens must be processed as indicated in the assay procedure section.
13. Specific training or guidance is recommended if operators are not experienced with specimen collection and handling procedures.

Caution: GAS Reagent 1 may cause skin, eye and respiratory tract irritation. GAS Reagent 1 contains an acidic solution. If the solution contacts the skin or eye, flush with large volumes of water. The combination of GAS Reagent 1 and Gas Reagent 2 generates nitrous acid which may cause skin, eye and respiratory tract irritation. If this solution contacts the skin or eye, flush with large volumes of water.

Storage and Handling: Kits may be stored at 2–30 °C. DO NOT FREEZE. **Reagents and devices must be at room temperature (15–30 °C) when used for testing.**

SPECIMEN COLLECTION AND HANDLING

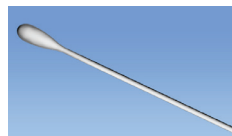
Specimen Collection:

For optimal performance, collect the throat swab with the swab that is provided in the kit. Swab the posterior pharynx, tonsils and other inflamed areas. Excess blood or mucus on the swab specimen may interfere with test performance. Avoid touching the tongue, cheeks and teeth⁹ and any bleeding areas of the mouth with the swab when collecting specimens.

Specimen Transport and Storage:

Testing should ideally be performed immediately after the specimens have been collected. Swab specimens may be stored in clean, dry plastic tubes for up to 8 hours at room temperature or 48 hours at 2–8 °C. Supplied kit swabs can be transported in Stuart's or Modified Amies Liquid Medium and stored for up to 48 hours. Nylon swabs can be transported in BD™ ESwab transport medium and be stored up to 48 hours. If a culture is desired, lightly roll the swab tip onto a blood agar plate **before** using the swab in the BD Veritor System for Rapid Detection of Group A Strep.

1. The BD Veritor System Group A Strep Kit includes sterile swabs with a rayon tip for throat specimen collection.
2. Have the patient open his or her mouth. Depress the tongue completely with a tongue depressor.
3. Swab the posterior pharynx, tonsils, and other inflamed areas. Avoid touching the tongue, cheeks, and teeth with the swab.
4. Withdraw the swab from the mouth. The sample is now ready for processing using the BD Veritor System Group A Strep Kit.

DOs and DON'Ts of Sample Collection

- Do collect sample as soon as possible after onset of symptoms
- Do test sample immediately
- BD recommends use of the throat swabs provided in this BD Veritor System Group A Strep Kit
- Do not use swabs with cotton tips and wood shafts
- Do not use calcium alginate swabs

PROCEDURE

Throat Swab Test Procedure

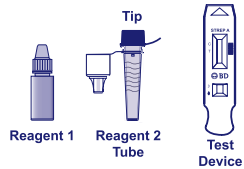
NOTE: Reagents, specimens and devices must be at room temperature (15–30 °C) for testing. The BD Veritor System instrument should be powered-on prior to use and will indicate when it is ready for insertion of the BD Veritor System Group A Strep device.

Prepare for testing

The following steps assume that users of a BD Veritor Plus Analyzer have chosen and set all configuration options, and that the Analyzer is ready to use. To choose or change these settings, see the BD Veritor Plus Analyzer *Instructions for Use*, section 4.7. A printer is not necessary to display results. However, if your facility has chosen to connect the BD Veritor Plus Analyzer to a printer, check that the printer is plugged into a power source, paper supply is adequate and any necessary network connections are enabled before testing.

Step 1 – Prepare for testing

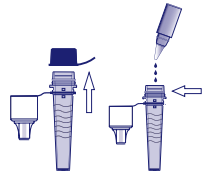
- Immediately before testing, for each control swab or patient specimen, take the bottle of GAS Reagent 1 and one GAS Reagent 2 tube/tip and one BD Veritor Group A Strep device from its foil pouch.
- Label one BD Veritor System device and one GAS Reagent 2 tube for each control or specimen to be tested.
- Place the labeled GAS Reagent 2 tube(s) in the designated area of the workstation or rack.

Prepare the Sample

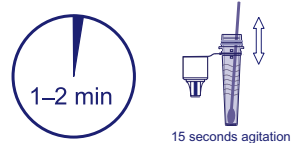
Step 2

- Remove the cap from the GAS Reagent 2 tube corresponding to the sample to be tested.
- Remove the cap from the GAS Reagent 1 bottle and add 3 drops from the GAS Reagent 1 bottle to the GAS Reagent 2 tube. GAS Reagent 2 contains a pH sensitive dye which turns from blue to yellow to confirm the addition of GAS Reagent 1. A uniform yellow color indicates complete mixing of the reagent. If any blue color remains, mix the solution by gently swirling the tube.

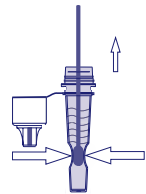
Step 3

- Insert the specimen or control swab and incubate for **1–2** minutes, then plunge the swab up and down for a minimum of **15** seconds, scrubbing inside the tube with the swab. Avoid splashing.

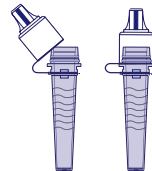
Step 4

- Remove the swab while squeezing the sides of the tube to extract the liquid.

Step 5

- Snap fit the tip onto the tube containing the processed specimen or control (threading/twisting not required).
- Note: Do not use tips from any other product, including other products from BD or other manufacturers.

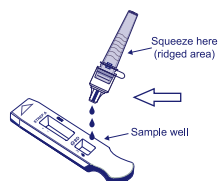
| After step 5, choose from the model and workflow option below before continuing to step 6: | | | | |
|--|---|---|---|----------|
| | BD Veritor System Reader or Analyzer in Analyze Now mode | BD Veritor Plus Analyzer in Walk Away mode | BD Veritor Plus Analyzer with the BD Veritor InfoScan module in Analyze Now mode---or--- Walk Away mode | |
| Instructions in section: | A | B | C | D |

Step 6A: Adding the specimen

- Invert the tube and hold the tube vertically (approximately one inch above the labeled BD Veritor System Strep A device sample well).
- Gently squeeze the ridged body of the tube, dispensing three (3) **drops** of the processed specimen into the sample well of a labeled BD Veritor System Strep A device.

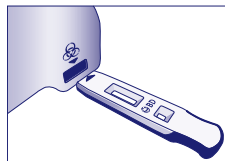
NOTE: Squeezing the tube too close to the tip may cause leakage.

**Step 7A: - Timing development**

- After adding the sample, allow the test to run for **5 minutes** before inserting into the BD Veritor instrument.
- **CAUTION: incorrect results may occur if development time is less than 5 minutes.** Some lines may appear on the device sooner. Do not read device visually.
- **NOTE:** If running test under laminar flow hood or in an area with heavy ventilation, cover test device to avoid inconsistent flow.

**Step 8A: Using the BD Veritor instrument**

- During incubation time, turn the BD Veritor instrument on by pressing the power button once.
- Insert assay device when 5 minute assay development time is complete.
- Follow the on-screen prompt to complete the procedure. The status of the assay analysis process appears in the display window, followed by the result display.

**Step 9A: Record the Result**

- When analysis is complete, the test result appears in the display window.
- Record the result and discard the test device appropriately.

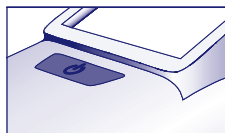
ATTENTION: TEST Results are NOT maintained in the display window when the device is removed or if the Analyzer is left unattended for more than 60 minutes (if the AC power adapter is connected).

Using a BD Veritor Plus Analyzer in "Walk Away" mode: (with no bar code scanning module installed)

To use **Walk Away mode** – connect the AC power adapter to the Analyzer and a power source

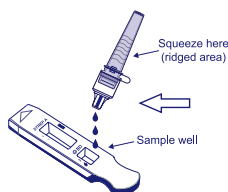
Step 6B: Starting Walk Away mode

- Turn the Analyzer on by pressing the blue power button once.
- When the display window reads:
"INSERT TEST DEVICE OR DOUBLE-CLICK FOR WALK AWAY MODE,"
– **Double-click** the blue power button.

Step 7B: Add the specimen to the test device

- When the display window reads "ADD SPECIMEN TO TEST DEVICE AND INSERT IMMEDIATELY":
– Invert the tube, holding it vertically (approximately one inch above the BD Veritor System Strip A device sample well).
– Gently squeeze the ridged portion of the tube, allowing three (3) drops of the processed specimen to dispense into the sample well of a labeled BD Veritor System Strip A device.

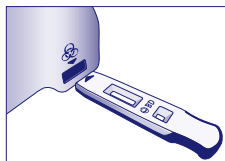
NOTE: Squeezing the tube close to the tip may cause leakage.

CAUTION: A countdown timer displays the time remaining for test insertion. **Walk Away mode must be activated again when this timer expires. Confirm timer is visible and Walk Away mode is activated before inserting test device.**

Step 8B: Start the development and reading sequence

- Insert the test device into the slot on the right side of the Analyzer.
The test device must remain horizontal to prevent spilling the specimen out of the sample well.
- "DO NOT DISTURB TEST IN PROGRESS" appears in the display window. Automatic timing of the assay development, image processing and result analysis begins.
- The display window shows the remaining analysis time.
- **Do not touch the Analyzer or remove the test device during this process. Doing so will abort the assay analysis.**

Step 9B: Record the Result

- When analysis is complete, the test result appears in the display window. Record the result and discard the test device appropriately.

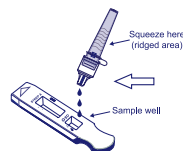
ATTENTION: TEST Results are NOT maintained in the display window when the device is removed or if the Analyzer is left unattended for more than 60 minutes (when AC power adapter is connected).

Step 6C: Add the specimen to the test device

- Invert the tube, holding it vertically (approximately one inch above the BD Veritor System Group A Strep device sample well).

- Gently squeeze the ridged body of the tube, dispensing three (3) drops of the processed specimen into the sample well of a labeled BD Veritor System Group A Strep A device.

NOTE: Squeezing the tube close to the tip may cause leakage.

Step 7C: Timing development

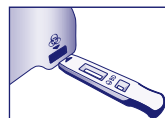
- Allow the test to develop for 5 minutes. BD recommends the use of a digital timer or stopwatch.
- **CAUTION: incorrect results may occur if development time is less than 5 minutes.** Some lines may appear on the device sooner. Do not read device visually.
- If running the test in a laminar flow hood or in an area with heavy ventilation, cover test device to avoid inconsistent flow.

Step 8C: Using the BD Veritor Plus Analyzer

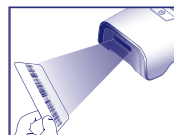
During the incubation time, turn the BD Veritor Plus Analyzer on by pressing the blue button once.

- The display window will briefly display “SCAN CONFIG BARCODE.” This is an opportunity to change the configuration of the Analyzer. Please refer to the Analyzer *Instructions for Use* for configuration instructions. Ignore this message and postpone this process when an assay is awaiting analysis.
- When assay development time is complete and the Analyzer display window reads: “INSERT TEST DEVICE OR DOUBLE-CLICK FOR WALK AWAY MODE”:
 - Insert the BD Veritor System Group A Strep device into the BD Veritor Plus Analyzer.

Step 9C: Using the Bar Code scanner

- Follow the prompts on the display window to complete any required barcode scans of:
 - OPERATOR ID
 - SPECIMEN ID and/or
 - KIT LOT NUMBER

- Prompts for each scanning step appear in the display window for only 30 seconds. Failure to complete scans during that time will cause the Analyzer to default to the beginning of step 8C. To restart this step, remove and reinsert the test device to initiate a new sequence.
- Move the barcode slowly toward the window until a confirmation tone sounds. The scanned barcode value appears in the next display window.
- The Analyzer can record the Kit Lot Number in the test record but does not restrict the use of expired or inappropriate reagents. Management of expired materials is the responsibility of the user. BD recommends against the use of expired materials.

After required scans are completed, the Analyzer displays a countdown timer and test analysis begins.

- **Do not touch the Analyzer or remove the test device during this process. Doing so will abort the assay analysis.**
- When analysis is complete, a result appears in the display window. If configured to display, the specimen ID barcode value also appears. If a printer is connected, specimen ID and result are automatically printed.

If a printer is not connected, record the result before removing the assay device.

ATTENTION: TEST Results are NOT maintained in the display window when the device is removed or if the Analyzer is left unattended for more than 15 minutes (60 minutes if the AC power adapter is connected).

Step 10C: Remove the test device

- Remove and then discard the test device appropriately. The display will show INSERT TEST DEVICE OR DOUBLE-CLICK BUTTON FOR WALK AWAY MODE to indicate the Analyzer is ready to perform another test. Note that the Analyzer returns to Analyze Now mode at the conclusion of each read sequence.

If the Veritor Plus Analyzer is connected to an LIS, a steady ENVELOPE symbol will appear to indicate that results are awaiting transmission. In the event that a network connection is not detected while the ENVELOPE symbol is still displayed, the Analyzer will queue all untransmitted results and attempt to transmit them when reconnected. If it is powered off during this time, it will attempt to transmit as soon as power is restored, and connection is re-established. A flashing envelope indicates that data are in the process of being transmitted.

Using the BD Veritor Plus Analyzer In “Walk Away” mode: with the BD Veritor InfoScan module installed

To use Walk Away mode - connect the AC power adapter to the Analyzer and a power source

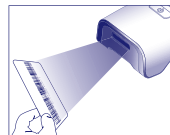
Step 6D: Starting Walk Away mode

- Turn the Analyzer on by pressing the blue power button once.
- The display window will briefly display “SCAN CONFIG BARCODE.” This is an opportunity to change the configuration of the Analyzer. Please refer to the Analyzer *Instructions for Use* for configuration instructions. Ignore this message and postpone this process when an assay is awaiting analysis.
- When the display window reads: “INSERT TEST DEVICE OR DOUBLE-CLICK FOR WALK AWAY MODE”
 - **Double-click** the blue power button.

Step 7D: Using the Bar Code scanner

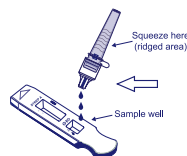
- Follow the prompts on the display window to complete any required barcode scans of:
 - OPERATOR ID
 - SPECIMEN ID and/or
 - KIT LOT NUMBER

- Prompts for each scanning step appear in the display window for only 30 seconds. Failure to complete scans during that time will cause the Analyzer to default to the beginning of step 6D. To restart this step, double-click the power button.
- Move the barcode slowly toward the window until a confirmation tone sounds. The scanned barcode value appears in the next display window.
- The Analyzer can record the Kit Lot Number in the test record but does not restrict the use of expired or inappropriate reagents. Management of expired materials is the responsibility of the user. BD recommends against the use of expired materials.

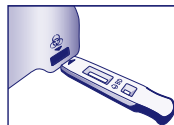
Step 8D: Add the specimen to the test device:

- When the display window reads: “ADD SPECIMEN TO TEST DEVICE AND INSERT IMMEDIATELY”:
 - Invert the tube, holding it vertically (approximately one inch above the BD Veritor System Group A Strep device sample well).
 - Gently squeeze the ridged portion of the tube, allowing three (3) drops of the processed specimen to dispense into the sample well of a labeled BD Veritor System Group A Strep A device. **NOTE: Squeezing the tube close to the tip may cause leakage.**
- **CAUTION: A countdown timer displays the time remaining for test insertion. Walk Away mode must be activated again when this timer expires. Confirm timer is visible and Walk Away mode is activated before inserting test device.**

Step 9D: Start the development and reading sequence

- Insert the test device into the slot on the right side of the Analyzer. **The test device must remain horizontal to prevent spilling the specimen out of the sample well.**
- **DO NOT DISTURB TEST IN PROGRESS** appears in the display window. Automatic timing of the assay development, image processing and result analysis begins.
- The display window shows the remaining analysis time.

Do not touch the Analyzer or remove the test device during this process. Doing so will abort the assay analysis.

- When analysis is complete, a result appears in the display window. If configured to display, the specimen ID barcode value also appears. If a printer is connected, specimen ID and result are automatically printed. **IF a printer is not connected, note the result before removing the assay device.**

ATTENTION: TEST Results are NOT maintained in the display window when the device is removed or if the Analyzer is left unattended for more than 60 minutes (when AC power adapter is connected).

Step 10D: Remove the test device

- Remove and then discard the test device appropriately. The display will show INSERT TEST DEVICE OR DOUBLE-CLICK BUTTON FOR WALK AWAY MODE to indicate the Analyzer is ready to perform another test. Note that the Analyzer returns to Analyze Now mode at the conclusion of each read sequence.

If the Veritor Plus Analyzer is connected to an LIS, a steady ENVELOPE symbol will appear to indicate that results are awaiting transmission. In the event that a network connection is not detected while the ENVELOPE symbol is still displayed, the Analyzer will queue all untransmitted results and attempt to transmit them when reconnected. If it is powered off during this time, it will attempt to transmit as soon as power is restored, and connection is re-established. A flashing envelope indicates that data are in the process of being transmitted.

INTERPRETATION OF RESULTS

Due to the technologies incorporated in the BD Veritor System, operators should not attempt to interpret assay results visually from the test strip contained within the BD Veritor System Strep A assay device. The BD Veritor System instrument must perform interpretation of results of all BD Veritor Assays.

| Instrument Display | Interpretation |
|--------------------|---|
| STREP: + | Positive Test for Strep A (Strep A antigen present) |
| STREP: - | Negative Test for Strep A (no antigen detected) |
| CONTROL INVALID | Control line error. Repeat the test. |

Invalid Test If the test is invalid, the BD Veritor System instrument will display "CONTROL INVALID" and the test or control must then be repeated. If "CONTROL INVALID" reading results, contact BD.

REPORTING OF RESULTS

Positive Test Positive for the presence of Strep A antigen. A positive result may occur in the absence of viable bacteria.

Negative Test Negative for the presence of Strep A antigen. Infection due to Strep A cannot be ruled-out because the antigen present in the sample may be below the detection limit of the test. Negative results should be confirmed by culture. Outside the U.S., additional follow-up of negative results should be conducted according to applicable clinical practice guidelines.

Invalid Test Test result is inconclusive. Do not report results.

QUALITY CONTROL:

To utilize the Analyzer's QC documentation capability, specimen barcode scanning must be enabled on an Analyzer equipped with the BD Veritor InfoScan module. Please refer to the *Analyzer Instructions for Use*, section 4, to choose or change this configuration.

Each BD Veritor System Group A Strep device contains both positive and negative internal/procedural controls:

1. The internal positive control validates the immunological integrity of the device, proper reagent function, and verifies correct test procedure.
2. The membrane area surrounding the test lines functions as a background check on the assay device.

The BD Veritor System instrument evaluates the positive and negative internal/procedural controls after insertion of the BD Veritor System test device. The BD Veritor System instrument will prompt the operator should a quality issue occur. Failure of the internal/procedural controls will generate an invalid test result. NOTE: The internal controls do not assess proper sample collection.

External Positive and Negative Controls:

Strep A Positive and Strep A Negative control swabs are supplied with each kit to ensure that the test reagents work properly and that the test procedure is performed correctly. **Prepare and test kit control swabs using the same workflow procedure as used for patient specimen swabs.**

Your laboratory's standard Quality Control procedures and applicable local, state and/or federal regulations or accreditation requirements dictate the performance of external quality control procedures.

BD recommends controls be run once for:

- each new kit lot,
- each new operator,
- each new shipment of test kits,
- and at periodic intervals as required by your facility.

If the kit controls do not perform as expected, do not report patient results. Contact BD Technical Services at 1.800.638.8663.

LIMITATIONS OF THE PROCEDURE

1. This test will indicate the presence of Strep A antigen in the throat swab specimen from both viable and non-viable group A *Streptococcus* bacteria. It does not determine the qualitative concentration of Strep A antigen.
2. Respiratory infections can be caused by Streptococci of serogroups other than A as well as other pathogens. This test does not differentiate between carriers and infected individuals.
3. Excess blood or mucus on the swab specimen may interfere with test performance.
4. Avoid touching the tongue, cheeks, and teeth⁹ and any bleeding areas of the mouth with the swab when collecting specimens.
5. False negative results can occur from inadequate or improper specimen collection, or from antigen levels that are below the limit of detection for this test.
6. As with all diagnostic tests, all results must be interpreted together with other clinical information available to the physician.

7. All negative results obtained with the BD Veritor System for Rapid Detection of Group A Strep should be confirmed by culture. Outside the U.S., additional follow-up of negative results should be conducted according to applicable clinical practice guidelines.
8. The use of antibiotics or over-the-counter medications may suppress the growth of Group A *Streptococcus* in culture despite the presence of organisms detectable by rapid antigen tests.

EXPECTED VALUES

Approximately 15% of pharyngitis in children ages 3 months to 5 years is caused by group A beta-hemolytic *Streptococcus*.¹⁰ In school-aged children and adults, the incidence of Strep throat infection is about 40%.¹¹ This disease usually occurs in the winter and early spring in temperate climates.³

PERFORMANCE CHARACTERISTICS

Explanation of Terms:

- P: Positive
- N: Negative
- C.I.: Confidence Interval

CLINICAL PERFORMANCE

Performance characteristics for the BD Veritor System for Rapid Detection of Group A Strep were established in a multi-center clinical trial conducted at one clinical laboratory site and four POC sites during the 2011-2012 respiratory season. A total of 796 prospectively collected specimens were evaluated using the BD Veritor System for Rapid Detection of Group A Strep and bacterial culture. Throat swabs from symptomatic patient were obtained, 51.8% from females and 48.2% were from males. Specimens from patients five years old or younger comprised 39.1% of the total, with 59.3% from patients 6 to 21 years of age and 1.6% from patients 22 years of age or older.

The performance of the BD Veritor System for Rapid Detection of Group A Strep was determined by comparison to bacterial culture and is presented in the table below.

| Clinical Performance Data – All Sites | | | |
|---|---------|-----|-------|
| | Culture | | Total |
| BD Veritor | P | N | |
| P | 144 | 29 | 173 |
| N | 5 | 618 | 623 |
| Total | 149 | 647 | 796 |
| Reference Method: Culture Sensitivity: 96.6% (95% CI: 92.4%, 98.6%) Specificity: 95.5% (95% CI: 93.6%, 96.9%) | | | |

| Clinical Performance Data – By Site | | | | |
|---|------------|---------|-----|-------|
| | | Culture | | Total |
| Site Code | BD Veritor | P | N | |
| Clinical Site | P | 20 | 2 | 22 |
| | N | 0 | 82 | 82 |
| Total | | 20 | 84 | 104 |
| Reference Method: Culture Sensitivity: 100% (95% CI: 83.9%, 100%) Specificity: 97.6% (95% CI: 91.7%, 99.3%) | | | | |
| POC 1 | P | 54 | 3 | 57 |
| | N | 5 | 188 | 193 |
| Total | | 59 | 191 | 250 |
| Reference Method: Culture Sensitivity: 91.5% (95% CI: 81.6%, 96.3%) Specificity: 98.4% (95% CI: 95.5%, 99.5%) | | | | |
| POC 2 | P | 21 | 9 | 30 |
| | N | 0 | 111 | 111 |
| Total | | 21 | 120 | 141 |

| Clinical Performance Data – By Site | | | | |
|---|------------|---------|-----|-------|
| | | Culture | | Total |
| Site Code | BD Veritor | P | N | |
| Reference Method: Culture Sensitivity: 100% (95% CI: 84.5%, 100%) Specificity: 92.5% (95% CI: 86.4%, 96.0%) | | | | |
| POC 3 | P | 21 | 7 | 28 |
| | N | 0 | 106 | 106 |
| Total | | 21 | 113 | 134 |
| Reference Method: Culture Sensitivity: 100% (95% CI: 84.5%, 100%) Specificity: 93.8% (95% CI: 87.8%, 97.0%) | | | | |
| POC 4 | P | 28 | 8 | 36 |
| | N | 0 | 131 | 131 |
| Total | | 28 | 139 | 167 |
| Reference Method: Culture Sensitivity: 100% (95% CI: 87.9%, 100%) Specificity: 94.2% (95% CI: 89.1%, 97.1%) | | | | |

Analytical Studies

Analytical Sensitivity (Limit of Detection)

The limit of detection for *Streptococcus pyogenes* was established with the BD Veritor System for Rapid Detection of Group A Strep test. The limit of detection (LOD) is defined as the lowest concentration that produces an approximate 95% positive reaction when tested with 60 replicates.

| Strain | LOD | Results | % Positivity |
|--------|---------------------|----------------|--------------|
| | (CFU/mL) | | |
| 12384 | 1 x 10 ⁵ | 57/60 Positive | 95.0% |
| 19615 | 5 x 10 ⁴ | 58/60 Positive | 96.7% |
| 25663 | 2 x 10 ⁵ | 57/60 Positive | 95.0% |

Analytical Specificity (Cross Reactivity)

The reactivity of various Streptococcal strains was determined with the BD Veritor System for Rapid Detection of Group A Strep test. Lancefield Groups B, C, D, F and G were tested at 1 X 10⁹ CFU/mL in triplicate and yielded negative results.

Various microorganisms (including bacteria and yeasts) that might be found in specimens were evaluated for potential cross reactivity with the BD Veritor System for Rapid Detection of Group A Strep test. Of the microorganisms tested, none demonstrated cross-reactivity with the BD Veritor System for Rapid Detection of Group A Strep test.

| Microorganism Name | Concentration Tested | Microorganism Name | Concentration Tested |
|--|------------------------------|---|----------------------------|
| <i>Arcanobacterium haemolyticum</i> | 1 x 10 ⁹ CFU/mL | <i>Lactobacillus</i> sp. (<i>Lactobacillus casei</i>) | 1 x 10 ⁹ CFU/mL |
| <i>Bordetella pertussis</i> | 5 x 10 ⁸ CFU/mL | <i>Moraxella catarrhalis</i> | 1 x 10 ⁹ CFU/mL |
| <i>Candida albicans</i> | 1 x 10 ⁹ CFU/mL | <i>Moraxella lacunata</i> | 1 x 10 ⁹ CFU/mL |
| <i>Corynebacterium diphtherium</i> sp. (<i>Corynebacterium</i> sp.) | 1 x 10 ⁹ CFU/mL | <i>Mycobacterium tuberculosis avirulent</i> | 5 x 10 ⁶ CFU/mL |
| <i>Enterococcus faecalis</i> | 1 x 10 ⁹ CFU/mL | <i>Neisseria gonorrhoeae</i> | 1 x 10 ⁹ CFU/mL |
| <i>Enterococcus faecium</i> | 1 x 10 ⁹ CFU/mL | <i>Neisseria lactamica</i> | 1 x 10 ⁹ CFU/mL |
| <i>Escherichia coli</i> | 1.5 x 10 ⁹ CFU/mL | <i>Neisseria meningitidis</i> | 1 x 10 ⁹ CFU/mL |
| <i>Fusobacterium necrophorum</i> | 1 x 10 ⁹ CFU/mL | <i>Neisseria mucosa</i> | 1 x 10 ⁶ CFU/mL |
| <i>Haemophilus influenzae</i> | 1 x 10 ⁹ CFU/mL | <i>Neisseria sicca</i> | 1 x 10 ⁹ CFU/mL |
| <i>Haemophilus parahaemolyticus</i> | 1.2 x 10 ⁵ CFU/mL | <i>Neisseria subflava</i> | 1 x 10 ⁹ CFU/mL |
| <i>Haemophilus parainfluenzae</i> | 1 x 10 ⁹ CFU/mL | <i>Proteus vulgaris</i> | 1 x 10 ⁹ CFU/mL |
| <i>Klebsiella pneumoniae</i> | 1.5 x 10 ⁹ CFU/mL | <i>Pseudomonas aeruginosa</i> | 1 x 10 ⁹ CFU/mL |
| | | <i>Serratia marcescens</i> | 1 x 10 ⁹ CFU/mL |

| Microorganism Name | Concentration Tested | Microorganism Name | Concentration Tested |
|---|----------------------------|--|---|
| <i>Staphylococcus aureus</i> | 1 x 10 ⁹ CFU/mL | <i>Yersinia enterocolitica</i> | 1 x 10 ⁹ CFU/mL |
| <i>Staphylococcus epidermidis</i> | 1 x 10 ⁹ CFU/mL | Adenovirus Type 1 | 1.6 x 10 ⁶ TCID ₅₀ /mL |
| <i>Staphylococcus haemolyticus</i> | 1 x 10 ⁹ CFU/mL | Adenovirus Type 7 | 2.81 x 10 ⁵ TCID ₅₀ /mL |
| <i>Staphylococcus oralis</i> | 1 x 10 ⁹ CFU/mL | Cytomegalovirus | 8.9 x 10 ³ TCID ₅₀ /mL |
| <i>Staphylococcus sanguis</i> | 1 x 10 ⁹ CFU/mL | Enterovirus (VR-28 Human Coxsackievirus) | 8.9 x 10 ⁶ TCID ₅₀ /mL |
| <i>Streptococcus anginosus</i> | 1 x 10 ⁹ CFU/mL | Epstein Barr Virus | N/A |
| <i>Streptococcus mitis</i> | 1 x 10 ⁹ CFU/mL | HSV Type 1 (HF) | 8.89 x 10 ⁶ TCID ₅₀ /mL |
| <i>Streptococcus mutans</i> ATCC 25173 | 3 x 10 ⁹ CFU/mL | Human coronavirus OC43 | 2.81 x 10 ⁴ TCID ₅₀ /mL |
| <i>Streptococcus pneumoniae</i> | 1 x 10 ⁹ CFU/mL | Human metapneumovirus (HMPV-27 A2) | 2.8 x 10 ⁶ TCID ₅₀ /mL |
| <i>Streptococcus salivarius</i> | 1 x 10 ⁹ CFU/mL | Human parainfluenza | 2.8 x 10 ⁶ TCID ₅₀ /mL |
| <i>Streptococcus</i> sp. Group B | 1 x 10 ⁹ CFU/mL | Measles | 1.6 x 10 ⁴ TCID ₅₀ /mL |
| <i>Streptococcus</i> sp. Group C | 1 x 10 ⁹ CFU/mL | Mumps virus | 1.6 x 10 ⁵ TCID ₅₀ /mL |
| <i>Streptococcus</i> sp. (<i>bovis</i> II) Group D | 1 x 10 ⁹ CFU/mL | Respiratory syncytial virus VR-26 | 1.6 x 10 ⁷ TCID ₅₀ /mL |
| <i>Streptococcus</i> sp. Group F | 1 x 10 ⁹ CFU/mL | Rhinovirus | 2.8 x 10 ⁶ TCID ₅₀ /mL |
| <i>Streptococcus</i> sp. Group G | 1 x 10 ⁹ CFU/mL | | |

Interfering Substances

Various substances were evaluated for potential interference with the BD Veritor System for Rapid Detection of Group A Strep test at concentrations comparable to or greater than levels that may be present in patient respiratory samples. Of the substances tested in this study, none exhibited interference when either Group A positive or Group A negative samples were tested with the BD Veritor System for Rapid Detection of Group A Strep test.

| Substance | Concentration Tested | Substance | Concentration Tested |
|-----------------------------------|----------------------|---|----------------------|
| 4-Acetamidophenol | 10 mg/mL | Menthol Throat Lozenges | 5% w/v |
| Acetylsalicylic acid | 20 mg/mL | Mometasone | 500 ng/mL |
| Albuterol | 0.083 mg/mL | Mouthwash Listerine | 5% v/v |
| Amantadine | 500 ng/mL | Mouthwash Scope | 5% v/v |
| Ascorbic acid chewable tablets | 5% by weight | Mouthwash CVS | 5% v/v |
| Beclomethasone | 500 ng/mL | Mucin, salivary protein, purified | 1 mg/mL |
| Benzocaine throat spray (Cepacol) | 5% by volume | Nasal Spray | 5% v/v |
| Blood, type A | 2% v/v | Nasal Spray | 5% v/v |
| Blood, type B | 2% v/v | Nasal Spray | 5% v/v |
| Blood, type AB | 2% v/v | Oseltamivir | 500 ng/mL |
| Blood, type O | 2% v/v | Oxymetazoline | 0.05 mg/mL |
| Budesonide | 500 ng/mL | Phenol throat spray (Chloraseptic) | 5% v/v |
| Chlorpheniramine maleate | 5 mg/mL | Phenylephrine | 1 mg/mL |
| Dexamethasone | 10 mg/mL | Pseudoephedrine HCl | 20 mg/mL |
| Dextromethorphan | 10 mg/mL | Throat drops: CVS | 5% w/v |
| Dyclonine HCl lozenges (Sucrets) | 5% w/v | Throat drops: Pedia Care | 5% w/v |
| Diphenhydramine HCl | 5 mg/mL | Throat drops: Triaminic | 5% w/v |
| Fexofenadine | 500 ng/mL | Tobramycin | 500 ng/mL |
| FluMist™ | 1% v/v | Triamcinolone | 500 ng/mL |
| Fluticasone | 500 ng/mL | Zanamivir | 1 mg/mL |
| Guaiaicol Glyceryl Ether | 20 mg/mL | Zicam throat spray (Zn / benzalkonium chloride) | 5% v/v |
| Ibuprofen | 10 mg/mL | Zinc Lozenges | 5% w/v |
| Loratidine | 100 ng/mL | | |

Reproducibility

The reproducibility of the BD Veritor System for Rapid Detection of Group A Strep was evaluated at one clinical and two POC sites. The panel was composed of 4 simulated Group A Strep samples. These included high negative samples (i.e., samples containing a very low concentration of Group A Strep), a low positive sample (near the limit of detection), a moderate positive sample and a negative sample. The panel was tested by two operators at each site over five days. The results are summarized below.

| BD Veritor Group A Strep Reproducibility | | | | | | | | |
|--|------------------|--------------|------------------|--------------|------------------|--------------|------------------|--------------|
| Sample | Site 1 | | Site 2 | | Site 3 | | Total | |
| | % Pos | 95% CI | % Pos | 95% CI | % Pos | 95% CI | % Pos | 95% CI |
| High negative | 3.3% (1/30) | 0.6%, 16.7% | 0.0% (0/30) | 0.0%, 11.3% | 0.0% (0/30) | 0.0%, 11.3% | 1.1% (1/90) | 0.2%, 6.0% |
| Low positive | 96.7% (29/30) | 83.3%, 99.4% | 83.3% (25/30) | 66.4%, 92.7% | 93.3% (28/30) | 78.7%, 98.2% | 91.1% (82/90) | 83.4%, 95.4% |
| Moderate positive | 100% (30/30) | 88.6%, 100% | 96.7% (29/30) | 83.3%, 99.4% | 100% (30/30) | 88.6%, 100% | 98.9% (89/90) | 94.0%, 99.8% |
| Negative | 0.0% (0/30) | 0.0%, 11.3% | 0.0% (0/30) | 0.0%, 11.3% | 0.0% (0/30) | 0.0%, 11.3% | 0.0% (0/90) | 0.0%, 4.1% |

CLIA WAIVER STUDY

As part of the larger prospective study, as described in the Performance Characteristics section above, the accuracy of the BD Veritor System for Rapid Detection of Group A Strep test was evaluated at four non-laboratory intended use Point of Care (POC) sites. A total of 14 operators representative of CLIA waived site personnel (intended users) participated in the study. No training on the use of the test was provided. A total of 692 prospectively collected specimens were tested.

Results below represent the performance of untrained operators at the POC sites of the BD Veritor Group A Strep test compared to culture. The results of the BD Veritor GAS test were compared with the results obtained from bacterial culture. Results are shown in the table below.

| CLIA Waiver Clinical Performance Data | | | |
|---|---------|-----|-------|
| | Culture | | |
| BD Veritor | P | N | Total |
| P | 124 | 24 | 148 |
| N | 6 | 538 | 544 |
| Total | 130 | 562 | 692 |
| Reference Method: Culture Sensitivity: 95.4% (95% CI: 90.3%, 97.9%) Specificity: 95.7% (95% CI: 93.7%, 97.1%) | | | |

Another study was designed to assess the capability of untrained users to test weakly reactive samples and deliver results with accuracy. The study was conducted at three non-laboratory intended use POC sites and one clinical laboratory site using panels of simulated samples. The panels were composed of swabs that were masked and randomized prior to shipment to the sites.

The positive samples were formulated at three levels (high negative ~5% positivity, low positive ~95% positivity, and moderate positive ~100%).

| Reproducibility Panel | |
|-----------------------|-------------------|
| Number of Samples | Group A Strep |
| 1 | High negative |
| 1 | Low positive |
| 1 | Moderate positive |

Each site had two operators who each tested the panel on each of ten days. Each swab was processed and tested in a single device according to the test procedure.

The table below shows performance of the test with samples near the cutoff of the assay for Group A Strep in the hands of untrained intended users across the POC sites and in the hands of laboratory personnel at one clinical laboratory site.

| BD Veritor Group A Strep Reproducibility | | | | | | | | |
|--|------------------|--------------|------------------|--------------|------------------|--------------|------------------|--------------|
| Sample | POC 1 | | POC 2 | | POC 3 | | Clinical Site | |
| | % Pos | 95% CI | % Pos | 95% CI | % Pos | 95% CI | % Pos | 95% CI |
| High negative | 5.0% (1/20) | 0.9%, 23.6% | 5.0% (1/20) | 0.9%, 23.6% | 10.0% (2/20) | 2.8%, 30.1% | 0.0% (0/20) | 0.0%, 16.1% |
| Low positive | 80.0% (16/20) | 58.4%, 91.9% | 90.0% (18/20) | 69.9%, 97.2% | 80.0% (16/20) | 58.4%, 91.9% | 80.0% (16/20) | 58.4%, 91.9% |
| Moderate positive | 100% (20/20) | 83.9%, 100% | 100% (20/20) | 83.9%, 100% | 100% (20/20) | 83.9%, 100% | 100% (20/20) | 83.9%, 100% |

Using risk analysis as a guide, analytical flex studies were conducted. The studies demonstrated that the test is insensitive to stresses of environmental conditions and potential user errors.

Technical Support

Technical Information: In the United States, contact BD Technical Service and Support at 1.800.638.8663 or bd.com.

Test system problems may also be reported to the FDA using the MedWatch reporting system (phone: 1-800 FDA-1088; fax: 1-800 FDA-1078; or <http://www.fda.gov/medwatch>).

AVAILABILITY

Cat. No. Description

| | |
|--------|--|
| 256040 | BD Veritor™ System for Rapid Detection of Group A Strep, 30 tests |
| 220093 | BD BBL™ CultureSwab™ Liquid Amies, 50 Single Swabs |
| 220099 | BD BBL™ CultureSwab™ Liquid Stuart, 50 Single Swabs |
| 220105 | BD BBL™ CultureSwab™ Liquid Amies, 50 Double Swabs |
| 220109 | BD BBL™ CultureSwab™ Liquid Stuart, 50 Double Swabs |
| 256049 | BD Veritor™ System Group A Strep Control Swab Set, 10 pairs of swabs |
| 220245 | BD™ ESwab Regular Collection Kit – White polypropylene screw-cap tube filled with 1 mL of Liquid Amies Medium and one regular size flocked applicator swab, 50 units |
| 220246 | BD™ ESwab Minitip Collection Kit – Green polypropylene screw-cap tube filled with 1 mL of Liquid Amies Medium and one minitip flocked applicator swab, 50 units |
| 220532 | BD™ ESwab Flexible Minitip Collection Kit – Blue polypropylene screw-cap tube filled with 1 mL of Liquid Amies Medium and one flexible minitip flocked applicator swab, 50 units |
| 256066 | BD Veritor™ Plus Analyzer |
| 256068 | BD Veritor™ InfoScan Module |
| 443907 | USB Printer Cable for BD Veritor™ Analyzer |

To network a BD Veritor Plus Analyzer to an LIS, contact BD Technical Services for details.

REFERENCES

1. Versalovic, J., K.C. Carroll, G. Funke, J.H. Jorgensen, M.L. Landry, and D.W. Warnock (ed.). 2011. Manual of Clinical Microbiology, 10th ed. American Society for Microbiology, Washington, D.C.
2. Webb, K.H. 1998. *Pediatrics*, 101:2, 2.
3. Bisno A.L., M.A. Gerber, J.M. Gwaltney, E.L. Kaplan, and R.H. Schwartz. 1997. Clin. Infect. Dis., 25: 574–83.
4. Needham, C.A., K.A. McPherson, K.H. Webb. 1998. Clin. Microbiol., 3468–3473.
5. Clinical and Laboratory Standards Institute. 2005. Approved Guideline M29-A3. Protection of laboratory workers from occupationally acquired infections, 3rd ed. CLSI, Wayne, Pa.
6. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53–80.
7. U.S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC) 5th ed. U.S. Government Printing Office, Washington, D.C.
8. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021–0045.
9. Garcia, L.S. (ed.). 2007. Specimen collection and transport. In Clinical microbiology procedures handbook, 3rd ed. American Society for Microbiology, Washington, D.C.
10. Nussinovitch, M., Y. Finkelstein, J. Amir, and I. Varsano. 1999. Clinical Pediatrics, 357–360.
11. Woods, W.A., C.T. Carter, M. Stack, A.F. Connors Jr., and T.A. Schlager. 1999. Southern Medical Journal, 491–492.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or bd.com.