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231748    **BBL DrySlide** Indole, 25 x 3 lamele.

## REFERINȚE

1. MacFadden, J.F. 1980. Biochemical tests for the identification of medical bacteria, 2nd ed. Williams and Wilkins, Baltimore.
2. Vracko, R., and J.C. Sherris. 1963. Indole spot-test in bacteriology. Am. J. Clin. Pathol. 39: 429-432.
3. Miller, J.M., and J.W. Wright. 1982. Spot indole test: evaluation of four reagents. J. Clin. Microbiol. 15: 589-592.
4. Lowrance, B.L., P. Reich, and W.H. Traub. 1969. Evaluation of two spot-indole reagents. Appl. Microbiol. 17: 923-924.
5. Peterson, W.C., D.C. Hale, and J.M. Matsin. 1982. An evaluation of the practicality of the spot-indole test for the identification of *Escherichia coli* in the clinical laboratory. Am. J. Clin. Pathol. 78: 755-758.
6. Isenberg, H.D. (ed.) 1994. Clinical microbiology procedures handbook, vol. 1. American Society for Microbiology, Washington, D.C.
7. Murray, P.R., E.J. Baron, M.A. Pfaller, F.C. Tenover, and R.H. Tenover (ed.). 1995. Manual of clinical microbiology, 6th ed. American Society for Microbiology, Washington, D.C.
8. Baron, E.J., L.R. Peterson, and S.M. Finegold. 1994. Bailey & Scott's diagnostic microbiology, 9th ed. Mosby-Year Book, Inc., St. Louis.

Manufacturer / Výrobce / Producent / Fabrikant / Tootja / Valmistaja / Fabricant / Hersteller / Κατασκευαστής / Gyártó / Ditta produttrice / Gamintojas /  
 Producent / Fabricante / Výrobca / Tilverkare / Producător / Üretici / Proizvođač / Производител / Атқарушы

Use by / Spottebujtje do / Anvendes for / Houdbaar tot / Kasutada enne / Viimeinkäyttöpäivä / A utiliser avant / Verwendbar bis / Ημερομηνία λήξης /  
 Felhasználhatósági dátuma / Usare entro / Naudokite iki / Brukes for / Stosować do / Utilizar em / Pouzít do / Usar antes de / Använd före / Използвайте  
 до / A se utiliza până la / Son kulanma tarihi / Upotrebiti do / Използовать до / дейін пайдалануға /  
 YYYY-MM-DD / YYYY-MM (MM = end of month) /  
 RRRR-MM-DD / RRRR-MM (MM = konec měsíce) /  
 AAAA-MM-DD / AAAA-MM (MM = slutning af måned) /  
 JJJJ-MM-DD / JJJJ-MM (MM = einde maand) /  
 AAAA-KK-PP / AAAA-KK (KK = kuu lõpp) /  
 VVVV-KK-PP / VVVV-KK (kuukauden loppuun mennessä) /  
 AAAA-MM-JJ / AAAA-MM (MM = fin du mois) /  
 JJJJ-MM-TT / JJJJ-MM (MM = Monatsende) /  
 EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα) /  
 EEEE-HH-NN / EEEE-HH (HH = hónap utolsó napja) /  
 AAAA-MM-GG / AAAA-MM (MM = fine mese) /  
 MMMM-MM-DD / MMMM-MM (MM = mēnesio pabaiga) /  
 AAAA-MM-DD / AAAA-MM (MM = slutten av måneden) /  
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) /  
 AAAA-MM-DD / AAAA-MM (MM = fim do mês) /  
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) /  
 aaaa-mm-dd / aaaa-mm (mm = fin del mes) /  
 AAAA-MM-DD / AAAA-MM (MM = slutet på månaden) /  
 TTTT-MM-DD / TTTT-MM (MM = края на месеца) /  
 AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii) /  
 YYYY-AA-GG / YYYY-AA (AA = аяы соңу) /  
 GGGG-MM-DD / GGGG-MM (MM = kraj meseca) /  
 TTTT-MM-DD / TTTT-MM (MM = конец месяца) /  
 ЖЖЖЖ-АА-КК / ЖЖЖЖ-АА (АА = айдың соңы)

Catalog number / Katalogové číslo / Katalognummer / Catalogusnummer / Kataloogi number / Tuotenumero / Numéro catalogue / Bestellnummer /  
 Αριθμός καταλόγου / Katalógusszám / Numero di catalogo / Katalogo numeris / Numer katalogowy / Número do catálogo / Katalógové číslo / Número de  
 catálogo / Каталоген номер / Număr de catalog / Katalog numerasi / Kataloški broj / Номер по каталогу / Каталог номери

Authorized Representative in the European Community / Autorizovaný zástupce pro Evropskou unii / Autoriseret repræsentant i EU / Erkend  
 vertegenwoordiger in de Europese Unie / Voliatud esindaja Euroopa Liidukogus / Valtuutettu edustaja Euroopan yhteisössä / Représentant agréé  
 pour la C.E.E. / Autorisierte EG-Vertretung / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Hivatalos képviselő az Európai Unióban /  
 Rappresentante autorizzato nella Comunità europea / Įgaliojasis atstovas Europos Bendrijoje / Autoriseret representant i EU / Autoryzowane  
 przedstawicielstwo w Unii Europejskiej / Reprezentante autorizado na União Europeia / Autorizovaný zástupca v Európskom spoločenstve / Reprezentante  
 autorizado en la Comunidad Europea / Auktoriserad representant i EU / Оторизирани представител в ЕС / Reprezentant autorizat în Uniunea Europeană /  
 Ауторизовану Yetkilil Temsilcisi / Ovlašteni predstavnik u Evropskoj zajednici / Уполномоченный представитель в Европейском сообществе / Европа  
 қауымдастығындағы уәклетті өкіл

In Vitro Diagnostic Medical Device / Lékařské zařízení určeno pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medisch hulpmiddel  
 voor in vitro diagnose / In vitro diagnostika meditsiniaparatuur / Lääkinnällinen in vitro -diagnostiikkalaitte / Dispositif médical de diagnostic in vitro /  
 Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / In vitro diagnostikai orvosi eszköz / Dispositivo médico diagnóstico in vitro /  
 In vitro diagnostikos prietais / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico  
 in vitro / Medicinska pomůcka na diagnostiku in vitro / Dispositivo médico de diagnóstico in vitro / Medicinsk anordning för in vitro-diagnostik /  
 Медицински уред за диагностика ин витро / Aparatură medicală de diagnosticare in vitro / In Vitro Diyagnostik Tibbi Cihaz / Medicinski uređaj za in vitro  
 dijagnostiku / Медицинский прибор для диагностики in vitro / Жасанды жағдайда жүргізілетін медициналық диагностика аспабы

Temperature limitation / Teplotní omezení / Temperaturbegrænsning / Temperaturlimit / Temperaturi piirang / Lämpötilarajoitus / Température  
 limite / Zulässiger Temperaturbereich / Οριο θερμοκρασίας / Hömersékleti határ / Temperatura limite / Laikymo temperatūra / Temperaturbegrænsning /  
 Ograniczenie temperatury / Limitação da temperatura / Ograničenje toplote / Limitación de temperatura / Temperaturbegrænsning / Температури  
 ограничения / Limitare de temperatură / Sıcaklık sınırlaması / Ograničenje temperature / Ограничение температуры / Температураны шектеу

Batch Code (Lot) / Kód (číslo) šarže / Batch code (Lot) / Chargennummer (lot) / Partii kood / Eräkoodi (LOT) / Code de lot (Lot) / Chargencode  
 (Chargenbezeichnung) / Кодікός партії (Партія) / Tetel száma (Lot) / Codice del lotto (partita) / Partijos numeris (Lot) / Batch-kode (Serie) / Kod partii  
 (serie) / Código do lote (Lote) / Kód série (šarža) / Código de lote (Lote) / Satskod (parti) / Код (Партія) / Număr lot (Lotul) / Parti Kodu (Lot) / Kod serije /  
 Код партии (lot) / Топтама коды

Contains sufficient for <n> tests / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> test / Voldoende voor <n> tests / Kõllaldane <n>  
 testide jaoks / Sisältöön riittävä <n> testejä varten / Contenu suffisant pour <n> tests / Ausreichend für <n> Tests / Παρύχει επαρκή ποσότητα <n>  
 εξετάσεων / <n> testhez elegendő / Contenido suficiente per <n> test / Pakankamas kiekis atlikti <n> testų / Innholder tilstrækkelig for <n> tester /  
 Zawiera ilość wystarczającą do <n> testów / Contém suficiente para <n> testes / Obsah vystačí na <n> testov / Contenido suficiente para <n> pruebas /  
 Räckertill <n> antal tester / Содержимого в достаточно за <n> теста / Conține suficient pentru <n> teste / <n> testleri için yeterli miktarda içerir / Sadržaj  
 dovoljan za <n> testova / Достаточно для <n> тестов(а) / <n> тесттері үшін жеткілікті

Becton, Dickinson and Company  
 7 Loveton Circle  
 Sparks, MD 21152 USA

Benex Limited  
 Pottery Road, Dun Laoghaire  
 Co. Dublin, Ireland

ATCC is a trademark of the American Type Culture Collection.

DrySlide is a trademark of Difco Laboratories, a subsidiary of Becton, Dickinson and Company.

BD, BD Logo and BBL are trademarks of Becton, Dickinson and Company. © 2013 BD.