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1. MacFadden, J.F. 1980. Biochemical tests for the identification of medical bacteria, 2nd ed. Williams and Wilkins, Baltimore.
2. Vracko, R., and J.C. Sherris. 1963. Indole spot-test in bacteriology. Am. J. Clin. Pathol. 39: 429-432.
3. Miller, J.M., and J.W. Wright. 1982. Spot indole test: evaluation of four reagents. J. Clin. Microbiol. 15: 589-592.
4. Lowrance, B.L., P. Reich, and W.H. Traub. 1969. Evaluation of two spot-indole reagents. Appl. Microbiol. 17: 923-924.
5. Peterson, W.C., D.C. Hale, and J.M. Matsin. 1982. An evaluation of the practicality of the spot-indole test for the identification of *Escherichia coli* in the clinical laboratory. Am. J. Clin. Pathol. 78: 755-758.
6. Isenberg, H.D. (ed.) 1994. Clinical microbiology procedures handbook, vol. 1. American Society for Microbiology, Washington, D.C.
7. Murray, P.R., E.J. Baron, M.A. Pfaller, F.C. Tenover, and R.H. Tenover (ed.). 1995. Manual of clinical microbiology, 6th ed. American Society for Microbiology, Washington, D.C.
8. Baron, E.J., L.R. Peterson, and S.M. Finegold. 1994. Bailey & Scott's diagnostic microbiology, 9th ed. Mosby-Year Book, Inc., St. Louis.

Manufacturer / Výrobce / Producent / Fabrikant / Tootja / Valmistaja / Fabricant / Hersteller / Κατασκευαστής / Gyártó / Ditta produttrice / Gamintojas / Producent / Fabricante / Výrobca / Tillverkare / Производител / Producător / Üretici / Proizvodač / Производител / Атқарушы

Use by / Spotföreljutt do / Anvendes for / Houdbaar tot / Kasutada enne / Viimeinkäyttöpäivä / A utiliser avant / Verwendbar bis / Ημερομηνία λήξης / Felhasználhatóság dátuma / Usare entro / Naudokite iki / Brukes for / Stosować do / Utilizar em / Použít do / Usar antes de / Använd före / Исползуйте до / A se utiliza până la / Son kullanna tarhi / Upotrebiti do / Исползовать до / дейн пайдалану / YYYY-MM-DD / YYYY-MM (MM = end of month) / RRRR-MM-DD / RRRR-MM (MM = konec měsíce) / ÁÁÁÁ-MM-DD / ÁÁÁÁ-MM (MM = slutning af måned) / JJJJ-MM-DD / JJJJ-MM (MM = einde maand) / AAAA-KK-PP / AAAA-KK (KK = kuu lõpp) / VVVV-KK-PP / VVVV-KK (kuukauden loppuun mennessä) / AAAA-MM-JJ / AAAA-MM (MM = fin du mois) / JJJJ-MM-TT / JJJJ-MM (MM = Monatsende) / EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα) / EEEE-HH-NN / EEEE-HH (HH = hónap utolsó napja) / AAAA-MM-GG / AAAA-MM (MM = fine mese) / MMMM-MM-DD / MMMM-MM (MM = mêsio pabaiga) / ÁÁÁÁ-MM-DD / ÁÁÁÁ-MM (MM = slutten av månaden) / RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) / AAAA-MM-DD / AAAA-MM (MM = fim do mês) / RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) / aaaa-mm-dd / aaaa-mm (mm = fin del mes) / ÁÁÁÁ-MM-DD / ÁÁÁÁ-MM (MM = slutet på månaden) / ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца) / AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii) / YYYY-AA-GG / YYYY-AA (AA = ayin sonu) / GGGG-MM-DD / GGGG-MM (MM = kraj meseca) / ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца) / ЖЖЖЖ-АА-КК / ЖЖЖЖ-АА (АА = айдын соңы)

Catalog number / Katalogové číslo / Katalognummer / Catalogusnummer / Kataloogi number / Tuotenumero / Numéro catalogue / Bestellnummer / Αριθμός καταλόγου / Katalógusszám / Numero di catalogo / Katalogo numeris / Numer katalogowy / Número do catálogo / Katalogové číslo / Número de catálogo / Каталоген номер / Număr de catalog / Katalog numarası / Kataloški broj / Номер по каталогу / Каталог номери

Authorized Representative in the European Community / Autorizovaný zástupce pro Evropskou unii / Autoriseret repræsentant i EU / Erkend vertegenwoordiger in de Europese Unie / Volitatud esindaja Euroopa Nõukogus / Valtuutettu edustaja Euroopan yhteisössä / Représentant agréé pour la C.E.E. / Autorisierter EG-Vertretung / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Hivatalos képviselő az Európai Unióban / Rappresentante autorizzato nella Comunità europea / Įgaliojasis atstovas Europos Bendrijoje / Autoriseret representant i EU / Autoryzowane przedstawicielstwo w Unii Europejskiej / Representante autorizado na União Europeia / Autorizovaný zástupca v Európskom spoločenstve / Representante autorizado en la Comunidad Europea / Auktoriserad representant i EU / Оторизован представитель в EU / Représentant autorizat în Uniunea Europeană / Авторизован представитель в EU / Ovlašteni predstavnik u Evropskoj zajednici / Уполномоченный представитель в Европейском сообществе / Европа қауымдастығындағы уәкілетті өкіл

In Vitro Diagnostic Medical Device / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medisch hulpmiddel voor in vitro diagnose / In vitro diagnostika meditsiiniaparatuur / Lääkinnällinen in vitro -diagnostiikkalaitte / Dispositif médical de diagnostic in vitro / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / In vitro diagnosztikai orvosi eszköz / Dispositivo medico diagnostico in vitro. / In vitro diagnostikos prietaisas / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Medicinska pomůcka na diagnostiku in vitro / Dispositivo médico de diagnóstico in vitro / Medicinsk anordning för in vitro-diagnostik / Медицински уред за диагностика ин витро / Aparatură medicală de diagnosticare in vitro / In Vitro Diagnostik Tibbi Cihaz / Medicinski uređaj za in vitro dijagnostiku / Медицинский прибор для диагностики in vitro / Жасанды жағдайда жүзінетін медициналық диагностика аспабы

Temperature limitation / Teplotní omezení / Temperaturbegrænsning / Temperaturlimit / Temperaturi piirang / Lämpötilarajoitus / Température limite / Zulässiger Temperaturenbereich / Όριο θερμοκρασίας / Hőmérsékleti határ / Temperatura limite / Laikymo temperatūra / Temperaturbegrensning / Ograniczenie temperatury / Limitação da temperatura / Ograničenje teploty / Limitación de temperatura / Temperaturbegränsning / Температурни ограничения / Limitare de temperatură / Sıcaklık sınırlaması / Ograničenje temperature / Ограничение температуры / Температураны шектеу

Batch Code (Lot) / Kód (číslo) šarže / Batch kode (Lot) / Chargennummer (lot) / Partii kood / Eräkoodi (LOT) / Code de lot (Lot) / Chargencode (Chargenbezeichnung) / Κωδικός παρτίδας (Παρτίδα) / Tétel száma (Lot) / Codice del lotto (partita) / Partijos numeris (Lot) / Batch-kode (Serie) / Kod partii (serie) / Código do lote (Lote) / Kód série (šarža) / Código de lote (Lote) / Satskod (parti) / Код (Партия) / Număr lot (lotul) / Parti Kodu (Lot) / Kod serije / Код партии (лот) / Топтама коды

Contains sufficient for <n> tests / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> test / Voldoende voor <n> tests / Küllaldane <n> testide jaoks / Sisaldoon riitavä <n> testejä varten / Contenu suffisant pour <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα <n> εξετάσεων / <n> teszthez elegendő / Contenuto sufficiente per <n> test / Pakankamas kiekis atlikti <n> testų / Innholder tilstrekkelig for <n> tester / Zawiera ilość wystarczającą do <n> testów / Contém suficiente para <n> testes / Obsah vystačí na <n> testov / Contenido suficiente para <n> pruebas / Räckertill <n> antal tester / Съдържа много е достатъчно за <n> теста / Conține suficient pentru <n> teste / <n> testleri için yeterli miktarda içerir / Sadržaj dovoljan za <n> testova / Достаточно для <n> тестов(а) / <n> тесттери үшін жеткілікті

Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA
800-638-8663
www.bd.com/ds

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

ATCC is a trademark of the American Type Culture Collection.

DrySlide is a trademark of Difco Laboratories, a subsidiary of Becton, Dickinson and Company.

BD, BD Logo and BBL are trademarks of Becton, Dickinson and Company. © 2013 BD.