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
Revisions

BALTSO0191 Version 10.0 Template 4
Inserts

Rev from	Rev to	JOB #
NA	01	8966-16

NOTES:

- BD Catalog Number: 491253, 491324, 491438, 491439, 491440
- Blank (Sheet) Size: Length: 11" Width: 8.5"
- Number of Pages: 16 Number of Sheets: 8
- Page Size: Length: 11" Width: 8.5" Final Folded Size: 8.5" x 11"
- Ink Colors: No. of Colors: 2 PMS#: 032 Red, PMS Standard Black
- Printed two sides: Yes ☒ No ☐
- Style (see illustrations below): # 1 (3-hole punch); Stapled top left corner

- Vendor Printed ☒ Online/In House Printed ☒ Web: ☐
- See specification control no. N/A for material information.
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Label Design	REVISED BY By Nancy Carlsen at 9:53 am, Jan 25, 2017	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION.	 Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA
Proofer	PROOFING APPROVED BY By Natalie Morio at 2:05 pm, Jan 25, 2017		
Checked By	THIRD EYE BY By Nichole Graham at 4:55 pm, Jan 26, 2017		
Part Number:	500019392	Category and Description Product Insert BD SurePath Collection Vial	Sheet: 1 of 17 Scale: N/A A

BD SurePath™ Collection Vial

For use with the BD PrepStain™ and
BD Totalys™ Systems

REF	491253	BD SurePath™ Collection Vial Kit	▽	500
REF	491324	BD SurePath™ Collection Vial Kit	▽	25
REF	491438	BD SurePath™ Vial and Cervix Brush Kit	▽	500
REF	491439	BD SurePath™ Vial and Combi Brush Kit	▽	500
REF	491440	BD SurePath™ Vial and Spatula/ Brush Kit	▽	500

English: pages 1 – 2 Deutsch: Seiten 5 – 6 Español: páginas 9 – 10
Français: pages 3 – 4 Italiano: pagine 7 – 8 Português: páginas 11 – 12

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokyny vám poskytne místní zástupce společnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteyts lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőjétől. / Нұсқаулар үшін жергілікті BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielem BD. / Contacte o reprezentante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указаний обратитесь к местному представителю компании BD. / Instrukcie ziskate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD.

INTENDED USE FOR GYNECOLOGIC APPLICATIONS

The BD SurePath™ Collection Vial is designed for use with the BD PrepStain™ and BD Totalys™ Systems for the processing of BD SurePath Liquid-based Pap Tests. The BD SurePath Collection Vial contains an alcohol-based preservation solution that serves as a transport, preservative and antibacterial medium for gynecologic specimens. Refer to the BD PrepStain or the BD Totalys SlidePrep package inserts for additional information and clinical data regarding the BD SurePath Liquid-based Pap Test.

SUMMARY AND EXPLANATION

Refer to the SAMPLE COLLECTION USING CERVICAL SAMPLING DEVICE(S) WITH DETACHABLE HEAD(S) section of this package insert for instructions on using the BD SurePath Collection Vial.

For information on using the BD SurePath Collection Vial with different platforms, refer to the following documentation:

- BD Totalys SlidePrep Product Insert
- BD Totalys MultiProcessor Instrument User's Manual
- BD PrepMate™ Instrument User's Manual
- BD PrepStain Product Insert
- BD SurePath Manual Method Product Insert

REAGENTS

Materials Provided: Each package of BD SurePath Collection Vials contains: 25 or 500 BD SurePath Collection Vials with 10 mL of BD SurePath Preservative Fluid in each vial.

Materials Required But Not Provided: Specimen collection device. Use either a broom-type sampling device with detachable head (e.g., Rovers® Cervex Brush®) or a combination endocervical brush/plastic spatula device with detachable head(s) (e.g., Cytobrush Plus® GT and Pap Perfect® spatula).

WARNINGS AND PRECAUTIONS

Cytologic specimens may contain infectious agents. Wear suitable protective clothing, gloves, and eye/face protection. Follow appropriate biohazard precautions when handling samples.

BD SurePath Preservative Fluid contains an aqueous solution of denatured ethanol. The mixture contains small amounts of methanol and isopropanol. Do not ingest.

Warning

H226 Flammable liquid and vapour.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P241** Use explosion-proof electrical/ventilating/lighting/equipment. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P240** Ground/bond container and receiving equipment. **P233** Keep container tightly closed. **P242** Use only non-sparking tools. **P243** Take precautionary measures against static discharge. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P370+P378** In case of fire: Use for extinction: CO₂, powder or water spray. **P403+P235** Store in a well-ventilated place. Keep cool. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

PRECAUTIONS

1. For *in vitro* Diagnostic Use.
2. Good laboratory practices should be followed and all procedures for use of the BD PrepStain, the BD Totalys SlidePrep, and the BD Totalys MultiProcessor systems should be strictly observed.
3. Avoid splashing or generating aerosols. Operators should use appropriate hand, eye and clothing protection.
4. The preservative fluid contained in the BD SurePath Collection Vial was tested for antimicrobial effectiveness against *Escherichia coli*, *Pseudomonas aeruginosa*, *Staphylococcus aureus*, *Candida albicans*, *Mycobacterium tuberculosis* and *Aspergillus niger* and found to be effective. BD SurePath samples inoculated with 10⁶ CFU/mL of each species yielded no growth after 14 days (28 days for *Mycobacterium tuberculosis*) of incubation under standard conditions. However, universal precautions for safe handling of biological fluids should be practiced at all times.

General precautions on ancillary testing from the BD SurePath Collection Vial

While there is no evidence that removal of an aliquot from the BD SurePath Collection Vial affects the quality of the specimen for cytology testing, rare instances of misallocation of pertinent diagnostic material may occur during this process. Healthcare providers may need to acquire a new specimen if the results do not correlate with the clinical history of the patient. Furthermore, cytology often addresses different clinical questions than molecular testing; therefore, aliquot removal may not be suitable for all clinical situations. If necessary, a separate specimen may be collected for ancillary testing rather than taking an aliquot from the BD SurePath Collection Vial.

Aliquot removal from low-cellularity specimens may leave insufficient material in the BD SurePath Collection Vial for preparation of a satisfactory BD SurePath Liquid-based Pap Test.

FIRST AID

Call a physician immediately. If swallowed, do not induce vomiting. Give plenty of water to drink. Never give anything by mouth to an unconscious person. If inhaled, remove person to fresh air. In case of contact, immediately flush skin with water; immediately flush eyes with plenty of water for at least 15 minutes.

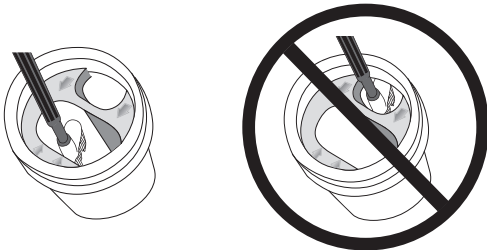
STORAGE AND DISPOSAL

1. Store the **BD SurePath™** Collection Vial without cytologic samples at room temperature (15–30 °C) in the vials provided.
2. The **BD SurePath** Collection Vial preserves cells for up to six months at refrigerated temperatures (2–10 °C) or up to 4 weeks at room temperature (15–30 °C).
3. Do not use the **BD SurePath** Collection Vial beyond the expiration date printed on the vial.
4. Expired and used supplies and products should be disposed of properly in accordance with institutional and local government regulations.

SAMPLE COLLECTION USING CERVICAL SAMPLING DEVICE(S) WITH DETACHABLE HEAD(S):

NOTE: For specimen collection, use either a broom-type sampling device with detachable head (e.g., Rovers Cervex-Brush) or a combination endocervical brush/plastic spatula device with detachable head(s) (e.g., Cytobrush Plus GT and Pap Perfect spatula).

1. Obtain a sample from the cervix according to the standard collection procedure (e.g., CLSI guideline GP15-A3).¹
2. Holding the collection vial firmly down on a flat surface, insert the head(s) of the collection device(s) into the larger of the two openings in the **BD SurePath** Collection Vial using one of three methods:
 - (1) Using the thumb and forefinger of a gloved hand, disconnect the head of the broom-type device from the handle and deposit into the larger of the two vial openings.
 - (2) Insert the head of the broom-type device into the larger of the two vial openings. Rotate the handle of the collection device while gently pulling up to detach the device head from the handle, depositing the device head into the larger of the two vial openings.
 - (3) When using combination brush/spatula collection devices with detachable heads, insert the first device so that the break point is above the top of the light blue vial insert and the head is below the insert. Bend the device back and forth until the device breaks, depositing the collection head into the larger of the two vial openings. Repeat for the second device.Discard the handle(s) of the sampling device(s). Do not touch the head(s) of the sampling device(s).

NOTE: Arrows on top of the light blue vial insert point to where the collection device head(s) should be deposited. **Always deposit collection device head(s) into the larger vial opening.**

DO NOT place collection device head(s) into the smaller vial opening.

3. Cap the vial tightly.
4. Optional: To utilize pre-labeled 2-D barcodes for positive sample identification, remove the upper barcode on the vial label by peeling from the top left corner. Attach the barcode to the requisition form to be sent to the lab. The second barcode remains on the vial, linking the patient sample to the requisition form.
5. Send the specimen containing the head(s) of the sampling device(s), with appropriate paperwork, to the laboratory for processing.

Collection Vial Transport

For domestic and international shipments, specimens should be packaged and labeled in compliance with applicable state, federal, and international regulations covering the transport of clinical specimens and etiologic agents/infectious substances. Time and temperature conditions for storage must be maintained during transport.

LIMITATIONS OF THE PROCEDURE

Always use good sampling techniques when collecting specimens. Poor sample collection techniques will produce inadequate preparations. Gynecologic specimens should be collected using a broom-type sampling device or a combination endocervical brush/plastic spatula device with detachable head(s) according to the standard collection procedure provided by the manufacturer. Wooden spatulas should not be used to collect specimens with the **BD SurePath** Collection Vial. Endocervical brush/plastic spatula devices that are not detachable should not be used with the **BD SurePath** Collection Vial.

After the first vortex step, a maximum of 0.5 mL of the homogeneous mixture of cells and fluid may be removed for ancillary testing prior to further processing of the **BD SurePath** Liquid-based Pap Test. Refer to the **BD PrepStain**, **BD Totalys** SlidePrep, or Manual Method package inserts or the **BD Totalys** MultiProcessor Instrument User's Manual for instructions on aliquot removal.

REFERENCE

1. Clinical and Laboratory Standards Institute (CLSI). *Cervicovaginal Cytology Based on the Papanicolaou Technique; Approved Guideline – Third Edition*. CLSI document GP15-A3 (ISBN 1-56238-679-4). CLSI 2008.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

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TriPath Imaging, Inc.
780 Plantation Drive
Burlington, NC 27215 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:

Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

U.S. Patent Number: 8,617,895.

Rovers Cervex-Brush is a registered trademark of Rovers Medical Devices B.V.
Cytobrush Plus and Pap Perfect are registered trademarks of CooperSurgical, Inc.

© 2017 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.