REF 491336	BD CytoRich Red Preservative
REF 491303	BD PrepStain Non-GYN Test Kit
REF 491304	BD Totalys SlidePrep Non-GYN Test Kit
REF 491333	BD Non-GYN Test Kit

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1. Hansen T, Pedersen H, Brauner V, Hariri J. Control specimens for immunocytochemistry in liquid-based cytology. *Cytopathology*. 2011 Aug;22(4):243-6.
2. Raistrick J, Shambayati B, Dunsmuir W. Collection fluid helps preservation in voided urine cytology. *Cytopathology*. 2008 Apr;19(2):111-7.
3. Gottschall EB, McGinley JN, Spoelstra N, Knott K, Wolfe P, Rose C, Singh M, Thompson HJ. Effect of cytological fixative and environmental conditions on nuclear morphometric characteristics of squamous epithelial cells in sputum. *Cytometry B Clin Cytom*. 2005 Sep;67(1):19-26.
4. Davis-Devine S, Day SJ, Freund GG. New red blood cell lysing fixative for use in fine needle aspiration and fluid cytology. *Acta Cytol*. 2003 Jul-Aug;47(4):630-6.
5. Tao LC, Smith JW. Fine-needle aspiration biopsy using a newly-developed pencil-grip syringe holder. *Diagn Cytopathol*. 1999 Feb;20(2):99-104.
6. Yang GC, Wan LS, Papella J, Waisman J. Compact Cell Blocks. Use for Body Fluids, Fine Needle Aspirations and Endometrial Brush Biopsies. *Acta Cytol* 1998. May-Jun;42(3):703-706.
7. Weidmann J, Chaubal A, Bibbo M. Cellular fixation. A study of CytoRich Red and Cytospin Collection Fluid. *Acta Cytol*. 1997 Jan-Feb;41(1):182-7.

BD Diagnostics teknisk service: Kontakta närmaste BD-representant eller besök www.bd.com/ds.

Do not reuse / Не используйте повторно / Nepoužívejte opakovaně / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / No reutilizar / Mitte kasutada korduvalt / Ne pas réutiliser / Ne koristiti ponovo / Egyszer használatos / Non riutilizzare / Пайдаланбаңыз / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolositi / He использовать повторно / Nepoužívejte opakovaně / Ne upotrebljavajte ponovo / Får ej återanvändas / Tekrar kullannatma / Не використовувати повторно

Use by / Используйте до / Spotřebujte do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейін пайдаланура / Naudokite iki / Izlietot līdz / Houdbaar tot / Brukes før / Stosować do / Prazo de validade / A se utiliza până la / Исползовать до / Použite do / Upotrebiti do / Använd före / Son kullanna tarihi / Використати до

YYYY-MM-DD / YYYY-MM (MM = end of month)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = slutning af måned)
JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)
EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)
AAAA-MM-DD / AAAA-MM (MM = fin del mes)
AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)
AAAA-MM-JJ / AAAA-MM (MM = fin du mois)
GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)
EEEE-HH-NN / EEEE-HH (HH = hónap utolsó napja)
AAAA-MM-GG / AAAA-MM (MM = fine mese)
ЖОЖОЖ-АА-КК / ЖОЖОЖ-АА / (АА = айдың соңы)
MMMM-MM-DD / MMMM-MM (MM = mēnesio pabaiga)
GGGG-MM-DD / GGGG-MM (MM = mēneša beigās)
JJJJ-MM-DD / JJJJ-MM (MM = einde maand)
AAAA-MM-DD / AAAA-MM (MM = sluten av måneden)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = fim do mês)
AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)
RRRR-MM-DD / RRRR-MM (ММ = koniec miesiąca)
GGGG-MM-DD / GGGG-MM (ММ = kraj meseca)
AAAA-MM-DD / AAAA-MM (ММ = slutet av månaden)
YYYY-AA-GG / YYYY-AA (AA = ayın sonu)
PPPP-MM-ДД / PPPP-MM (ММ = кінець місяця)

Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Топтама коды / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot nummer / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партії

Catalog number / Католожен номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalógusszám / Numero di catalogo / Kataloγ αριθμός / Katalogo numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalogové číslo / Kataloški broj / Katalog numarası / Номер за каталог

Caution, consult accompanying documents / Внимание, направьте справку в придружаващите документи / Pozor! Pristudujte si přiloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatust! Lugeka kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Upozorenje, koristite prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / Dėmesio, žiūrėkite pridedamus dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Vystřaha, pozri spřívodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацію

Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbicante / Atқарушы / Gamintojas / Ražotājs / Tilvirker / Producent / Producător / Производитель / Výrobca / Proizvođač / Tillverkare / Üretici / Виробник

In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsinaparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnosztikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жүргізетін медициналық диагностика аспабы / In vitro diagnostikos prietaisai / Medicinas ierices, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispozitiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики in vitro / Medicinska pomůcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diyagnostik Tibbi Cihaz / Медичний пристрій для діагностики in vitro

Consult Instructions for Use / Направете справка в инструкциите за употреба /
 Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten
 / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lúgeda
 kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el
 a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен
 танысып алыңыз / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību
 / Raadpleeg de gebruiksaanwijzing / Se i bruksanvisningen / Zobacz instrukcja
 użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile
 de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie /
 Pogledajte uputstvo za upotrebu / Se bruksanvisningen / Kullanım Talimatları'na
 başvurun / Див. інструкції з використання

Temperature limitation / Температурни ограничения / Teplotní omezení /
 Temperaturbegrænsning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας
 / Limitación de temperatura / Temperatuuri piirang / Limites de température /
 Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны
 шектеу / Laikymo temperatūra / Temperatūras ierobežojumi / Temperatuurilimiet /
 Temperaturbegrensning / Ograniczenie temperatury / Limites de temperatura / Limite
 de temperatură / Ограничение температуры / Ohraničenje teploty / Ograničenje
 temperature / Temperaturgräns / Sicaklık sınırlaması / Обмеження температури

Contains sufficient for <n> tests / Съдържанието е достатъчно за <n> теста / Dostatečné
 množství pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests /
 Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas /
 Küllaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova /
 <n> teszthez elegendő / Contenuto sufficiente per <n> test / <n> тесттегі үшін жеткілікті /
 Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende
 voor "n" testen / Innholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do
 <n> testów / Conteúdo suficiente para <n> testes / Conținut suficient pentru <n> teste
 / Достаточное для <n> тестов(a) / Obsah vystačí na <n> testov / Sadržaj dovoljan za
 <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli malmaze içerir /
 Вистачить для аналізів: <n>

Authorized Representative in the European Community / Оторизирани представител
 в Европейската общност / Autorizovaný zástupce pro Evropském společenství /
 Autoriseret repræsentant i De Europæiske Fællesskaber / Autorisierter Vertreter in
 der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή
 Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatud esindaja
 Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne /
 Autorizirani predstavnik u Evropskoj uniji / Meghatalmazott képviselő az Európai
 Közösségekben / Rappresentante autorizzato nella Comunità Europea / Европа
 қауымдастығындағы уәкілетті өкіл / Įgaliotasis atstovas Europos Bendrijoje /
 Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese
 Gemeenschap / Autorisert representant i EU / Autoryzowane przedstawicielstwo we
 Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia /
 Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель
 в Европейском сообществе / Autorizovaný zástupca v Európskom spoločenstve /
 Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska
 gemenskapen / Avrupa Topluluğu Yetkili Temsilcisi / Уповноважений представник у країнах
 ЄС

Fragile, Handle With Care / Чупливо, Работете с необходимото внимание. /
 Krehké, Při manipulaci postupujte opatrně. / Forsigtig, kan gå i stykker. / Zerbrechlich,
 vorsichtig handhaben. / Εύθραυστο. Χειριστείτε το με προσοχή. / Frágil. Manipular con
 cuidado. / Örn, kæsitlege ettevaatlikult. / Fragile. Manipuler avec précaution. / Lomljivo,
 rukujte pažljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura. /
 Сынгыш, абайлап пайдаланыңыз. / Traпу, elkítés atsargiai. / Breekbaar, voorzichtig
 behandelen. / Ømtålig, håndter forsigtig. / Krucha zawartość, przenosić ostrożnie. /
 Frágil, Manuseie com Cuidado. / Frágil, manipulați cu atenție. / Хрупкое! Обращаться
 с осторожностью. / Krehké, vyžaduje sa opatrná manipulácia. / Lomljivo - rukujte
 pažljivo. / Bräckligt. Hantera försiktigt. / Kolay Kırılır, Dikkatli Taşıyın

TriPath Imaging, Inc.
 780 Plantation Drive
 Burlington, NC 27215 USA

Benex Limited
 Pottery Road, Dun Laoghaire
 Co. Dublin, Ireland

Australian Sponsor:

Becton Dickinson Pty Ltd.
 4 Research Park Drive
 Macquarie University Research Park
 North Ryde, NSW 2113
 Australia

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. © 2015 BD