
Rev from	Rev to	JOB #
03	04	7842-15

NOTES:

- BD Catalog Number: 490518
- Blank (Sheet) Size: Length: 11" Width: 8.5"
- Number of Pages: 7 Number of Sheets: 4
- Page Size: Length: 11" Width: 8.5" Final Folded Size: 11"L x 8.5"W
- Ink Colors: No. of Colors: 2 PMS#: 032 Red, Standard Black
- Printed two sides: Yes ☒ No ☐
- Style (see illustrations below): # #1: 3-hole punch; stapled at top left corner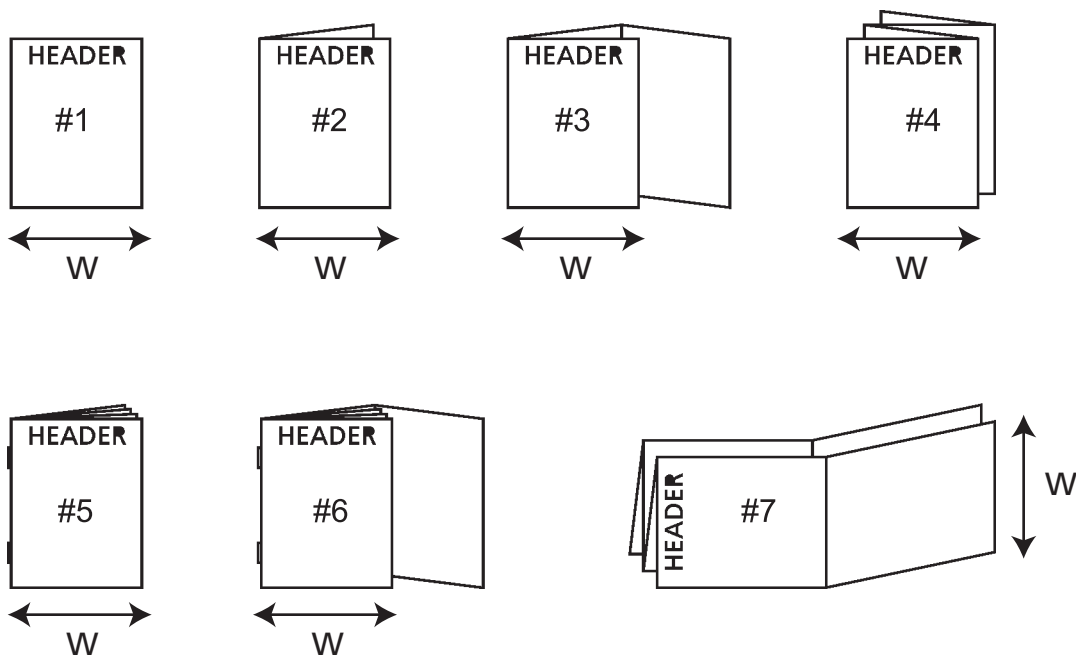

- BD Diagnostics inspects per procedure 838-07096-00 upon entry to the plant.
- Instructions also included in the **BD Totalys™ SlidePrep** User's Manual (500005570); review if changes are made to this document.
- If updating please review the 750-07071-XX installation pack
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Label Design	REVISED BY By Nancy Carlsen at 10:30 am, Mar 17, 2016	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION.	 Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA
Proofer	PROOFING APPROVED BY By Helaine Jones at 1:42 pm, Mar 17, 2016		
Checked By	THIRD EYE BY By Kim Stone at 10:48 am, Mar 23, 2016		
Part Number:	779-07075-00	Category and Description Insert BD Tris Buffered Saline Pack pH 8.0	Sheet: 1 of 8 Scale: N/A A

Tris Buffered Saline Pack pH 8.0

REF 490518

Tris Buffered Saline Pack pH 8.0

INTENDED USE

Tris Buffered Water is intended for use in the processing of **BD SurePath™** Liquid-based Pap Test samples as outlined in the instructions for use.

WARNINGS AND PRECAUTIONS

Warning

H315 Causes skin irritation. **H319** Causes serious eye irritation.

P280 Wear protective gloves/protective clothing/eye protection/face protection. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P302+P352** IF ON SKIN: Wash with plenty of soap and water.

STORAGE

Tris Buffered Water Stock Solution:

This Stock Solution of Tris Buffered Water must be stored refrigerated (2–10 °C) and is stable for up to four (4) weeks. The bottle is to be capped when not in use. Discard solution after four (4) weeks. Use a clean or new bottle when making a new batch of Stock Solution.

Tris Buffered Water Working Solution:

The Working Solution of Tris Buffered Water is stable for two (2) weeks at room temperature (15–30 °C). Discard solution after two (2) weeks. Use a clean or new bottle when making a new batch of Working Solution.

MATERIALS REQUIRED

Materials Provided

Tris Buffered Saline, pH 8.0

One (1) 1 L bottle

One (1) 4 L bottle

Note: 1 L bottle and 4 L bottle are provided only with:

- **BD PrepStain™** Install Kit, Cat. No. 491289
- **BD Totalys™** SlidePrep Install Kit, Cat. No. 491292

Materials Required But Not Provided

Deionized water (DI)

Measuring device, 200 mL

PROCEDURE

TRIS BUFFERED WATER STOCK SOLUTION

1. Fill the 1 L bottle labeled Tris Buffered Water Stock Solution approximately half full with DI water.
2. Empty 1 pack of 50 mM Tris Buffered Saline, pH 8.0 into the bottle and swirl until dissolved.
3. Fill bottle with DI water until full and cap the bottle.
4. Invert the bottle several times to ensure complete mixing.
5. Label the bottle with the date the Stock Solution was prepared and the expiration date. Discard after four (4) weeks.

TRIS BUFFERED WATER WORKING SOLUTION

1. Measure 200 mL of the Tris Buffered Water Stock Solution and pour into the 4 L bottle labeled Tris Buffered Water Working Solution.
2. Fill the bottle with 3.4 L of deionized water and cap the bottle.
3. Invert the bottle several times to ensure complete mixing.
4. Label the bottle with the date the Working Solution was prepared and the expiration date. Discard after two (2) weeks.

BD Diagnostics Technical Service and Support: contact your local BD representative or www.bd.com/ds.

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CONSERVACIÓN

Tris Buffered Water Stock Solution (Solución madre):

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Tris Buffered Water Working Solution (Solución activa):

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Agua desionizada (DI)

Dispositivo de medición, 200 mL

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TRIS BUFFERED WATER STOCK SOLUTION (SOLUCIÓN MADRE)

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TRIS BUFFERED WATER WORKING SOLUTION (SOLUCIÓN ACTIVA)

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CONSERVAÇÃO

Tris Buffered Water Stock Solution (Solução Base):

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PROCEDIMENTO

TRIS BUFFERED WATER STOCK SOLUTION (SOLUÇÃO BASE)

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TriPath Imaging, Inc.
780 Plantation Drive
Burlington, NC 27215 USA

Supplier:

Sigma-Aldrich Corporation
CH-9471 Buschs S6
Switzerland
Tel: +41(0)817552511
Fax: +41(0)817552815

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. © 2016 BD