# **BD BBL Streptocard Acid Latex Test Kit** (avsedd för *in vitro*-diagnostik)

L010796(03)  
2017-02  
Svenska

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## REFERENSER

1. Ederer, G.M., Herrmann, M.M., Bruce, R. Matsen, J.M. and Chapman, S.S. (1972). Rapid Extraction Method with Pronase B for Grouping Beta-Haemolytic Streptococci. Appl. Microbiol., 23, 285.
2. EL Kholy, A., Wannamaker, L.W. and Krause, R.M. (1974). Simplified Extraction Procedure for Serological Grouping of Beta-Hemolytic Streptococci. Appl. Microbiol., 28, 836.
3. Elliot, S.D. and Tai, J.Y. (1978). The Type-Specific Polysaccharides of *Streptococcus suis*. J. Exp. Med., 148, 1699.
4. Facklam, R.R. (1980). Streptococci and Aerococci, Ch. 8 in Manual of Clinical Microbiology, 3rd Ed., Edited by Lennette, E.H. Balows, A., Hausler, W.J., and Truant, J.P. American Society for Microbiology, Washington, D.C. pages 88-110.
5. Facklam R.R. (1977). Physiological Differentiation of Viridans Streptococci. J. Clin. Microbiol., 5, 184.
6. Fuller, A.T. (1938). The Formamide Method for the Extraction of Polysaccharides from Haemolytic Streptococci. Brit. J. Exp. Path., 19, 130.
7. Maxted, W.R. (1948). Preparation of Streptococcal Extracts for Lancefield Grouping. Lancet, ii, 255.
8. Nowlan, S.S. and Deibel, R.H. (1967). Group Q Streptococci. I. Ecology, Serology, Physiology and Relationships to Established Enterococci. J. Bact., 94, 291.
9. Petts, D.N. (1984). Early Detection of Streptococci in Swabs by Latex Agglutination Before Culture. J. Clin. Microbiol., 19, 432.
10. Rantz, L.A. and Randall, E. (1955). Use of Autoclaved Extracts of Haemolytic Streptococci for Serological Grouping. Stanford Med. Bull., 13, 290.
11. Watson, B.K., Moellering, R.C. and Kunz, L.J. (1975). Identification of Streptococci. Use of Lysozyme and Streptomyces albus filtrate in the Preparation of Extracts of Lancefield Grouping. J. Clin. Microbiol., 1, 274.

Teknisk service: Kontakta närmaste BD-representant eller besök [www.bd.com](http://www.bd.com).

Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbicante / Аткарушы / 제조업체 / Gamintojas / Ražotājs / Tilvirker / Producent / Producător / Производитель / Výrobca / Proizvođač / Tilverkare / Üretici / Виробник / 生产厂商

Use by / Използвайте до / Spotfæbjeite do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / 사용 기한 / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейін пайдалануға / Naudokite iki / Izlietot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Исползовать до / Použite do / Upotrebiti do / Använd före / Son kullanna tarihi / Використати до/line / 使用截止日期

YYYY-MM-DD / YYYY-MM (MM = end of month)  
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)  
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)  
AAAA-MM-DD / AAAA-MM (MM = slutning af måned)  
JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)  
EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)  
AAAA-MM-DD / AAAA-MM (MM = fin del mes)  
AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)  
AAAA-MM-JJ / AAAA-MM (MM = fin du mois)  
GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)  
ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja)  
AAAA-MM-GG / AAAA-MM (MM = fine mese)  
ЖЖЖЖ-АА-КК / ЖЖЖЖ-АА / (АА = айдың соңы)  
YYYY-MM-DD/YYYY-MM (MM = 월말)  
MMMM-MM-DD / MMMM-MM (MM = mensesio pabaiga)  
GGGG-MM-DD/GGGG-MM (MM = mensesia beigas)  
JJJJ-MM-DD / JJJJ-MM (MM = einde maand)  
AAAA-MM-DD / AAAA-MM (MM = sluten av månaden)  
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)  
AAAA-MM-DD / AAAA-MM (MM = fim do mês)  
AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)  
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)  
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)  
GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)  
AAAA-MM-DD / AAAA-MM (MM = slutet av månaden)  
YYYY-AA-GG / YYYY-AA (AA = ayın sonu)  
PPPP-MM-DD / PPPP-MM (MM = кінець місяця)  
YYYY-MM-DD / YYYY-MM (MM = 月末)

Catalog number / Каталоген номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalogusszám / Numero di catalogo / Каталог номери / 카탈로그 번호 / Katalogo / numeris / Kataloga numurs / Catalogus number / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalogové číslo / Kataloški broj / Katalog numarası / Номер за каталогом / 目录号

Authorized Representative in the European Community / Оторизирани представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret repræsentant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatus esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizuirani predstavnik u Europskoj uniji / Meghatalmazott képviselő az Európai Közösségekben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті өкіл / 유럽 공동체의 위임 대표 / Įgaliotasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autorisert representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupca v Európskom spoločenstve / Autorizovano predstavnístvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Автура Топлутуҗу Yetkilil Temsilcisi / Уповноважений представник у країнах ЄС / 歐洲共同體授權代表

In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiiniparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnostikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жүзгізетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietaisas / Medicinas ierīces, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispozitiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики in vitro / Medicinska pomůcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diyagnostik Tibbi Cihaz / Медицинский пристрій для діагностики in vitro / 体外诊断医疗设备

Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrensning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperatuuri piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны шектеу / 온도 제한 / Laikymo temperatūra / Temperatūras ierobežojumi / Temperaturlimit / Temperaturbegrensning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ohraničenie teploty / Ograničenje temperature / Temperaturgräns / Sıcaklık sınırlaması / Обмеження температури / 温度限制

Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Топтама коды / 배치 코드(로트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot nummer / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партии / 批号 (亚批)

Contains sufficient for <n> tests / Съдържа много е достатъчно за <n> теста / Dostatečné množství pro <n> testů / Inneholder tilstrækkelig til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Küllaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenuto sufficiente per <n> test / <n> тесттері үшін жеткілікті / <n> 테스트가 충분히 포함됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor <n> testen / Innholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Conținut suficient pentru <n> teste / Достаточное для <n> тестов(a) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli miktarda içerir / Вистачить для аналізів: <n> / 足够进行 <n> 次检测

Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / 사용 지침 참조 / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i bruksanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se brugsanvisningen / Kullanım Talimatları'na başvurun / Див. інструкції з використання / 请参阅使用说明

Do not reuse / Не използвайте отново / Nepoužívejte opakovaně / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / No reutilizar / Mitte kasutada korduvalt / Ne pas réutiliser / Ne koristiti ponovo / Egsyzer használatos / Non riutilizzare / Пайдаланбаныз / 재사용 금지 / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolositi / Не использовать повторно / Nepoužívejte opakovaně / Ne upotrebljavajte ponovo / Får ej återanvändas / Tekrar kullanmayın / Не використовувати повторно / 请勿重复使用

Serial number / Серийн номер / Sériové číslo / Seriennummer / Seriennummer / Σειριακός αριθμός / N° de serie / Seerianumber / Numéro de série / Serijski broj / Sorozatszám / Numero di serie / Топтамалық номери / 일련 번호 / Serijos numeris / Sērijas numurs / Serie nummer / Numer seryjny / Numéro de série / Număr de serie / Серийный номер / Seri numarası / Номер серії / 序列号

|                                                                                     |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                          |
|-------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|    | For IVD Performance evaluation only / Само за оценка качеството на работа на IVD / Pouze pro vyhodnocení výkonu IVD / Kun til evaluering af IVD ydelse / Nur für IVD-Leistungsbewertungszwecke / Μόνο για αξιολόγηση απόδοσης IVD / Sólo para la evaluación del rendimiento en diagnóstico in vitro / Ainult IVD seadme hindamiseks / Réserve à l'évaluation des performances IVD / Samo u znanstvene svrhe za In Vitro Dijagnostiku / Kizárolag in vitro diagnosztikához / Solo per valutazione delle prestazioni IVD / Жасанды жағдайда «пробирка ішінде» диагностикада тек жұмысты бағалау үшін / IVD 성능 평가에 대해서만 사용 / Tik IVD prietaisų veikimo charakteristikoms tikrinti / Vienīgi IVD darbības novērtēšanai / Uitsluitend voor doeltreffendheidsonderzoek / Kun for evaluering av IVD-ytelse / Tylko do oceny wydajności IVD / Uso exclusivo para avaliação de IVD / Numai pentru evaluarea performanței IVD / Только для оценки качества диагностики in vitro / Určené iba na diagnostiku in vitro / Samo za procenu učinka u in vitro dijagnostici / Endast för utvärdering av diagnostisk användning in vitro / Yalnızca IVD Performans değerlendirmesi için / Тільки для оцінювання якості діагностики in vitro / 仅限 IVD 性能评估 |
|    | For US: "For Investigational Use Only"                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                   |
|    | Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiir / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūros žemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrense / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sıcaklık alt sınırı / Мінімальна температура / 温度下限                                                                                                                                                                                                                                                                                                                                                                                                                                                      |
| <b>CONTROL</b>                                                                      | Control / Контролно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Controllo / Бақылау / 컨트롤 / Kontrolé / Kontrolle / Controle / Controllo / Контроль / kontroll / 对照                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |
| <b>CONTROL +</b>                                                                    | Positive control / Положителен контрол / Pozitivní kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitivna kontrola / Pozitív kontroll / Controllo positivo / Оң бақылау / 양성 컨트롤 / Teigijama kontrolé / Pozitívá kontrol / Positieve controle / Kontrola dodatnia / Controllo positivo / Положительный контроль / Pozitif kontrol / Позитивний контроль / 阳性对照试剂                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                     |
| <b>CONTROL -</b>                                                                    | Negative control / Отрицателен контрол / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negativna kontrola / Negativ kontroll / Controllo negativo / Негативтік бақылау / 음성 컨트롤 / Neigijama kontrolé / Negatívá kontrol / Negativeve controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативний контроль / 阴性对照试剂                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                          |
| <b>STERILE EO</b>                                                                   | Method of sterilization: ethylene oxide / Метод на стерилизация: етиленов оксид / Způsob sterilizace: etylenoxid / Steriliseringmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseringmetode: etüleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etilén-oxid / Metoda di sterilizzazione: ossido di etilene / Sterilizacija: etilēns – этилен тотығы / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksis / Gesteriliseerd met behulp van ethylenoxide / Steriliseringmetode: etylenoksid / Metoda sterilizacji: tiенок etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metodá sterilizácie: etylenoxid / Metoda sterilizacije: etilen oksid / Steriliseringmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: этиленоксидом / 灭菌方法: 环氧乙烷                                                                                                                                                                   |
| <b>STERILE R</b>                                                                    | Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Způsob sterilizace: záření / Steriliseringmetode: bestråling / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseringmetode: kiirgus / Méthode de stérilisation : irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metoda di sterilizzazione: irradiazione / Sterilizacija: eļģāne / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringmetode: bestråling / Metoda sterylizacji: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: iradiere / Метод стерилизации: облучение / Metodá sterilizácie: ožiarenie / Metoda sterilizacije: ozračevanje / Steriliseringmetod: strålning / Sterilizasyon yöntemi: ırdıyasyon / Метод стерилизації: опроміннення / 灭菌方法: 辐射                                                                                                                                                                                                                                                               |
|    | Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologilised riskid / Risques biologiques / Biološki rizik / Biológiai veszélyes / Rischio biologico / Биологический тауекелдер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyojik Riskler / Біологічна небезпека / 生物学风险                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                      |
|   | Caution, consult accompanying documents / Внимание, направте справка в придружаващите документи / Pozor! Pristudujte si příloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatust! Lugeka kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Urozorenje, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dmesio, žiūrėkite priedamus dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Vystraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацію / 小心, 请参阅附带文档。                        |
|  | Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / 상한 온도 / Aukščiausia laikymo temperatūra / Ausgējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrense / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sıcaklık üst sınırı / Максимальна температура / 温度上限                                                                                                                                                                                                                                                                                                                                                                                                                                              |
|  | Keep dry / Пазете сухо / Skladujte v suchém prostředí / Opbevarer tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / 건조 상태 유지 / Laikykite sausiai / Uzglabāt sausu / Droog houden / Holdes tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávať v suchu / Držite na suvom mestu / Förvaras tørt / Kuru bir şekilde muhafaza edin / Бергити від вологи / 请保持干燥                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                             |
|  | Collection time / Време на събиране / Čas odběru / Opsamlingstidspunkt / Entnahmeuhrzeit / Ωρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жинау уакыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de colheita / Ora colectării / Время сбора / Doba odberu / Vreme prikupljanja / Uppsamlingstid / Toplama zamanı / Час забору / 采集时间                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                    |
|  | Peel / Обенете / Otevfete zde / Abn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Ўстиғи қабатын алып таста / 벗기기 / Plešti čia / Atimēt / Schillen / Trekk av / Oderwać / Destacar / Se dezlipeste / Отклеить / Odrhните / Oljuštiti / Dra isår / Ayırma / Відклеїти / 撕下                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                          |
|  | Perforation / Перфорация / Perforace / Perforering / Διήτρηση / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Тесик тесу / 절취선 / Perforacija / Perforacija / Perforatie / Perforacja / Perfuração / Perforare / Перфорация / Perforácia / Perforasyon / Перфорация / 穿孔                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                        |
|  | Do not use if package damaged / Не използвайте, ако опаковката е повредена / Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packungnicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά. / No usar si el paquete está dañado / Mitte kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használni, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Ереп пакет бұзылған болса, пайдаланба / 패키지가 손상된 경우 사용 금지 / Jei pakuotė pažeista, nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Не использовать при повреждении упаковки / Nepoužívať, ak je obal poškodený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки / 如果包装破损, 请勿使用                                                                     |
|  | Keep away from heat / Пазете от топлина / Nevystavujte přilišnému teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgusest / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Салқын жерде сақта / 열을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargatő no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / Не напевать / Uchovávať mimo zdroja tepla / Držite dalje od toplote / Får ej utsättas för värme / Isidan uzak tutun / Бергити від дії тепла / 请远离热源                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |
|  | Cut / Срежете / Odstřihněte / Klip / Schneiden / Cortar / Lõigata / Découper / Rezi / Vágja ki / Tagliare / Keciңiz / 잘라내기 / Kirpti / Nogriez / Knippen / Kutt / Odciąć / Cortar / Decupați / Отрезать / Odstrihните / Iseći / Klipp / Kesme / Розрізати / 剪下                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                            |

Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmedatum / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuurpäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жинаған тізбекүні / 수집 날짜 / Paemimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colheita / Data colectării / Дата сбора / Dátum odberu / Datum prikupljanja / Uppsamlingsdatum / Toplama tarihi / Дата забору / 采集日期

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Becton, Dickinson and Company  
7 Loveton Circle  
Sparks, MD 21152 USA

Benex Limited  
Pottery Road, Dun Laoghaire  
Co. Dublin, Ireland

ATCC is a trademark of the American Type Culture Collection.

© 2017 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.