Part Number:	500017022	BALTSO0191 Version 13.0 Template 4 Inserts	
Category and Description: Package Insert, BD CytoRich™ Red Preservative	Rev from: 03 Rev to: 04		
	Job Number: 618-18		

Catalog Number: 491336, 491303, 491304, 491459

Blank (Sheet) Size: Length: 11" Width: 8.5"

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
Number of Pages: 20 Number of Sheets: 10

Page Size: Length: 11" Width: 8.5" Final Folded Size: 8.5" X 11"

Ink Colors: Number of Colors: 2 PMS #: 032 Red; Standard Black

Printed Two Sides: Yes: ☒ No: ☐

Style (see illustrations below): # 1 (3-hole punched); Stapled top left corner

Vendor Printed: ☒ Online / In House Printed: ☐ Web Printed: ☐

See Specification control no. N/A for material information.

Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Company confidential. This document is the property of Becton, Dickinson and Company and is not to be used outside the company without written permission. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Revised By: **REVISED BY**
By Tori Pagani at 11:50 am, Jun 22, 2018

Proofing Approved By: **PROOFING APPROVED BY**
By Terrence Means at 12:10 pm, Jun 22, 2018

Third Eye By: **THIRD EYE BY**
By Daria Kuznetsova at 12:33 pm, Jun 22, 2018

REF	491336	BD CytoRich™ Red Preservative
REF	491303	BD PrepStain™ Non-GYN Test Kit
REF	491304	BD Totalys™ SlidePrep Non-GYN Test Kit
REF	491459	BD Non-GYN Stain Kit
English: pages 1 – 3 Deutsch: Seiten 7 – 9 Español: páginas 13 – 15 Français: pages 4 – 6 Italiano: pagine 10 – 12 Português: páginas 16 – 18		

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokyny vám poskytne místní zástupce společnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteyttä lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőjétől. / Нұсқаулар үшін жергілікті BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielem BD. / Contacte o reprezentante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указаний обратитесь к местному представителю компании BD. / Instrukcie získate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD.

INTENDED USE

1. **BD CytoRich™ Red Preservative** is intended to preserve cells and small tissue fragments in suspension.
2. **BD CytoRich Red Preservative** is used to prepare cells and small tissue fragments for cytological and histological examination.
3. **BD CytoRich Red Preservative** lyses red blood cells and solubilizes proteins.
4. The preserved samples are compatible with immuno-histochemical staining and results are similar to those achieved with neutral buffered formalin.

SUMMARY AND EXPLANATION

BD CytoRich Red Preservative is an alcohol-based hemolytic fixative. It is for cytology and small biopsy processing. It is packaged in leak-resistant plastic containers. Its ingredients include alcohols, formaldehyde, and buffers.

BD CytoRich Red Preservative is used for diagnostic non-cervicovaginal cytology. In this context, it is a fixative for cells and small tissue samples that have been collected from human body sources including, but not limited to, sputum, washings, brushings, scrapings, body fluids, exudates, and fine needle aspirates.

MECHANISMS OF HEMOLYSIS AND FIXATION

BD CytoRich Red Preservative lyses red blood cells, cross-links soluble globulin proteins, and the alcohol fixes cells and small tissue fragments. One milliliter of **BD CytoRich Red Preservative** can lyse red blood cells and stabilize soluble proteins from 25–50 microliters of whole blood.

1. Red blood cell membranes are selectively emulsified and further disrupted by a combination of alcohol and osmotic lysis. The red blood cells of rare individuals with hemoglobinopathy may lyse slowly or resist complete lysis. Senescent red blood cells resist lysis.
2. **BD CytoRich Red Preservative** contains less than 1% formaldehyde. Globulin proteins that are released from red blood cells and collected with blood plasma and interstitial tissue fluids are cross-linked by formaldehyde in solution before they are denatured by alcohols and are thus prevented from forming protein super-aggregates (flocculent precipitates).
3. Other cell membranes act as selective barriers that allow alcohols to pass more rapidly than formaldehyde into cells' cytoplasm and nucleoplasm. This causes cells to appear alcohol fixed.

4. Subsequently, intercellular junctions and intracellular proteins are cross-linked by formaldehyde, which accounts for the long-term stability of the cells and tissue fragments in suspension.

LIMITATIONS OF THE PROCEDURE

1. **BD CytoRich Red Preservative** is not sold as a fixative for screening cervicovaginal cytology.
2. Cytologic samples should be fixed in **BD CytoRich Red Preservative** as soon as possible after collection.
3. A sample that has become degraded prior to fixation will be unsatisfactory for examination.
4. **BD CytoRich Red Preservative** should not be used to fix tissue fragments larger than 5 millimeters in average diameter.
5. All supplies are intended for single use only and cannot be reused.

WARNINGS AND PRECAUTIONS

Cytologic specimens may contain infectious agents. Wear suitable protective clothing, gloves, and eye/face protection. Follow appropriate biohazard precautions when handling samples.

BD CytoRich Red Preservative is an aqueous solution containing isopropanol, methanol, ethylene glycol, and formaldehyde. Do not ingest.

Danger

H226 Flammable liquid and vapor. **H302** Harmful if swallowed. **H319** Causes serious eye irritation. **H317** May cause an allergic skin reaction. **H350** May cause cancer. **H371** May cause damage to organs. **P210** Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P233** Keep container tightly closed. **P240** Ground/bond container and receiving equipment. **P241** Use explosion-proof electrical/ventilating/lighting equipment. **P242** Use only non-sparking tools. **P243** Take precautionary measures against static discharge. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P272** Contaminated work clothing should not be allowed out of the workplace. **P201** Obtain special instructions before use. **P202** Do not handle until all safety precautions have been read and understood. **P281** Use personal protective equipment as required. **P260** Do not breathe dust/fume/gas/mist/vapors/spray. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P337+P313** If eye irritation persists: Get medical advice/attention. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P333+P313** If skin irritation or rash occurs: Get medical advice/attention. **P301+P312** IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. **P330** Rinse mouth. **P309+P311** IF exposed or if you feel unwell: Call a POISON CENTER or doctor/physician. **P321** Specific treatment (see on this label). **P363** Wash contaminated clothing before reuse. **P370+P378** In case of fire: Use for extinction: CO₂, powder or water spray. **P403** Store in a well-ventilated place. **P235** Keep cool. **P405** Store locked up. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

BD EA/OG Combo Stain contains ethanol, methanol and acetic acid. Do not ingest.

Danger

H225 Highly flammable liquid and vapor. **H312+H332** Harmful in contact with skin or if inhaled. **H370** Causes damage to organs.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P233** Keep container tightly closed. **P240** Ground/bond container and receiving equipment.

P241 Use explosion-proof electrical/ventilating/lighting/equipment.

P242 Use only non-sparking tools. **P243** Take precautionary measures against static discharge. **P260** Do not breathe dust/fume/gas/mist/vapours/spray. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P312** Call a POISON CENTER or doctor/physician if you feel unwell. **P362+P364** Take off contaminated clothing and wash it before reuse. **P370+P378** In case of fire: Use for extinction: CO₂, powder or water spray. **P403+P235** Store in a well-ventilated place. Keep cool. **P405** Store locked up.

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

BD Hematoxylin Stain 0.5 contains ethylene glycol. Do not ingest.

Warning

H302 Harmful if swallowed. **H315** Causes skin irritation. **H319** Causes serious eye irritation. **H335** May cause respiratory irritation.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area.

P280 Wear protective gloves/protective clothing/eye protection/face protection. **P301+P312** IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. **P302+P352** IF ON SKIN: Wash with plenty of soap and water. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P330** Rinse mouth. **P332+P313** If skin irritation occurs: Get medical advice/attention. **P337+P313** If eye irritation persists: Get medical advice/attention. **P362** Take off contaminated clothing. **P403+P233** Store in a well-ventilated place. Keep container tightly closed. **P405** Store locked up. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

PRECAUTIONS

1. For *in vitro* diagnostic use.
2. For professional use only.
3. Good laboratory practices should be followed.
4. Avoid splashing or generating aerosols. Operators should use appropriate hand, eye/face and clothing protection.
5. Do not pipette by mouth.
6. Do not ingest (contains alcohols other than ethanol and formaldehyde).
7. Used supplies and product should be disposed of properly in accordance with institutional and governmental regulations.

MATERIALS REQUIRED

BD CytoRich Red Preservative can be used with the **BD PrepStain™** Slide Processor or **BD Totalys™** SlidePrep or in a manual process. Refer to the Operator's Manual for the **BD PrepStain** Slide Processor or **BD Totalys** SlidePrep for complete information regarding staining reagents, components and accessories. Not all of the materials listed below are required for preparing slides manually.

Materials Provided

Reagents

- **BD CytoRich Red Preservative**, 3,600 mL

Materials Sold Separately by BD

Reagents

- **BD Non-GYN Stain Kit**
 - 1 – **BD Hematoxylin Stain 0.5**, 480 mL
 - 1 – **BD EA/OG Combo Stain**, 480 mL

Instruments, Equipment and Accessories

- **BD PrepStain** Slide Processor
- **BD Totalys** SlidePrep
- Easy Aspirator Kit
- Centrifuge and accessories
- **BD** Centrifuge Tubes
- **BD SurePath™** PreCoat Slides
- **BD** Settling Chambers
- **BD PrepStain** Transfer Tips
- **BD Totalys** Transfer Tips
- *Kit Alternative #1: **BD PrepStain Non-GYN Test Kit***
 - 2 x 96 – **BD** Centrifuge Tubes
 - 2 x 96 – **BD SurePath** PreCoat Slides
 - 2 x 96 – **BD** Settling Chambers
 - 2 x 96 – **BD PrepStain** Transfer Tips
- *Kit Alternative #2: **BD Totalys SlidePrep Non-GYN Test Kit***
 - 2 x 96 – **BD** Centrifuge Tubes
 - 2 x 96 – **BD SurePath** PreCoat Slides
 - 2 x 96 – **BD** Settling Chambers
 - 2 x 96 – **BD Totalys** Transfer Tips

Materials Required But Not Provided by BD

- Specimen Collection Container
- Vortex Mixer
- Precision Pipettes with Disposable Tips
- Deionized Water (pH 7.5 to 8.5)
- Reagent Grade Alcohol
- Xylene or Xylene substitute
- Mounted Media
- Coverslips

STORAGE

1. The storage limit of the **BD CytoRich** Red Preservative is up to 24 months from date of manufacture (without a patient specimen) at room temperature (15–30 °C).
2. Do not use fluid beyond the expiration date printed on the label.
3. The storage limit for **BD CytoRich** Red Preservative with cytology sample should be determined in accordance with your laboratory's procedures.

GENERAL SPECIMEN PREPARATION, BACKGROUND

1. Label the specimen collection container before use.
2. One milliliter of **BD CytoRich** Red Preservative can lyse and stabilize 25–50 microliters of whole blood. Using this as a guideline, one drop (50 microliters) of aqueous cell slurry may be cleared of its red cells and protein and fixed by one milliliter of **BD CytoRich** Red Preservative.
3. The minimum fixation time is 30 minutes.
4. Cytologic preparations may be prepared using the **BD PrepStain** Slide Processor, **BD Totalys** SlidePrep, standard cytocentrifugation methods, or smear techniques.
5. Refer to the **BD CytoRich** Non-GYN Procedures and Protocols Manual for complete information regarding specific specimen handling and collection procedures.

GENERAL SPECIMEN PREPARATION, METHOD

If fresh (unfixed) samples are used, follow General Procedure Steps 1 through 4.

If fixed samples are used, follow General Procedure Steps 1, 3 and 4.

GENERAL PROCEDURE

1. Concentration
 - a. Mix to homogenize the sample by simple inversion or vortexing.
 - b. Assess the sample for and remove clots, particles, and mucus.
 - i. It is important to remove any visible clots or particles. Failure to do so may cause clogs in the **BD PrepStain** or **BD Totalys** SlidePrep vacuum bundles. Filtering may be accomplished by pouring the specimen through any course medium such as gauze. Any captured material may be used for cell block procedures.
 - ii. Follow standard laboratory procedures to homogenize and/or emulsify mucus.
 - c. Transfer an aliquot, or for small volume samples the entire amount to a 50 mL centrifuge tube.
 - d. Centrifuge for 10 minutes at 600 g.
 - e. Decant the supernatant fluid.
 - f. Vortex Sample for 15 ± 5 seconds to homogenize sample.
2. Fixation
 - a. Depending on cell pellet volume, add enough **BD CytoRich** Red Preservative to cover the sample in a 1:1 ratio.
 - b. Vortex for 15 ± 5 seconds and let sit for a minimum of 30 minutes.
 - c. Centrifuge for 10 minutes at 600 g.
 - d. Decant supernatant and vortex.

3. Washing

- a. If no visible or small pellet is identified, add 6–10 mL of buffered DI water directly to 50 mL centrifuge tube, vortex for 15 ± 5 seconds and transfer entire contents to a 12 mL tube.
- b. If moderate to large pellet is identified, transfer a representative sample (1–5 drops) to 12 mL tube and add 10 mL buffered DI water.
- c. Transfer the 12 mL tubes to centrifuge tube holder(s) and centrifuge for 5 minutes at 600 g.
- d. Decant and vortex for 15 ± 5 seconds to homogenize sample.
 - i. Decanting should be done in a single, rapid motion by inverting each tube rack 180 degrees so as to not disturb the cell pellet.
 - ii. While inverted, carefully blot the mouth of all tubes in the rack with absorbent paper.
 - iii. Turn the rack upright after 3 to 5 seconds.

4. Processing

- a. Load the labeled 12 mL centrifuge tube holders onto the **BD PrepStain** Slide Processor or the **BD Totalys** SlidePrep for processing.
- b. Follow 'Non-GYN Slide Processing' instructions in the applicable User's Manual.

BIBLIOGRAPHY

1. Hansen T, Pedersen H, Brauner V, Hariri J. Control specimens for immunocytochemistry in liquid-based cytology. *Cytopathology*. 2011 Aug;22(4):243-6.
2. Raistrick J, Shambayati B, Dunsmuir W. Collection fluid helps preservation in voided urine cytology. *Cytopathology*. 2008 Apr;19(2):111-7.
3. Gottschall EB, McGinley JN, Spoelstra N, Knott K, Wolfe P, Rose C, Singh M, Thompson HJ. Effect of cytological fixative and environmental conditions on nuclear morphometric characteristics of squamous epithelial cells in sputum. *Cytometry B Clin Cytom*. 2005 Sep;67(1):19-26.
4. Davis-Devine S, Day SJ, Freund GG. New red blood cell lysing fixative for use in fine needle aspiration and fluid cytology. *Acta Cytol*. 2003 Jul-Aug;47(4):630-6.
5. Tao LC, Smith JW. Fine-needle aspiration biopsy using a newly-developed pencil-grip syringe holder. *Diagn Cytopathol*. 1999 Feb;20(2):99-104.
6. Yang GC, Wan LS, Papella J, Waisman J. Compact Cell Blocks. Use for Body Fluids, Fine Needle Aspirations and Endometrial Brush Biopsies. *Acta Cytol* 1998. May-Jun;42(3):703-706.
7. Weidmann J, Chaubal A, Bibbo M. Cellular fixation. A study of CytoRich Red and Cytospin Collection Fluid. *Acta Cytol*. 1997 Jan-Feb;41(1):182-7.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

REF 491336	BD CytoRich Red Preservative (Conservateur rouge)
REF 491303	BD PrepStain Non-GYN Test Kit (Kit de tests non-gyn)
REF 491304	BD Totalys SlidePrep Non-GYN Test Kit (Kit de tests non-gyn)
REF 491459	BD Non-GYN Stain Kit (Trousse de coloration non gynécologique)

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