
Revisions

BALTSO0191 Version 8.0 Template 4
Inserts

Rev from	Rev to	JOB #
02	03	7714-15

NOTES:

- BD Catalog Number: 491335, 491303, 491304, 491333, 491432, 491433
- Blank (Sheet) Size: Length: 11" Width: 8.5"
- Number of Pages: 14 Number of Sheets: 7
- Page Size: Length: 11" Width: 8.5" Final Folded Size: 11"L x 8.5"W
- Ink Colors: No. of Colors: 2 PMS#: 032 Red, Standard Black
- Printed two sides: Yes ☒ No ☐
- Style (see illustrations below): # #1: 3-hole punch; stapled at top left corner

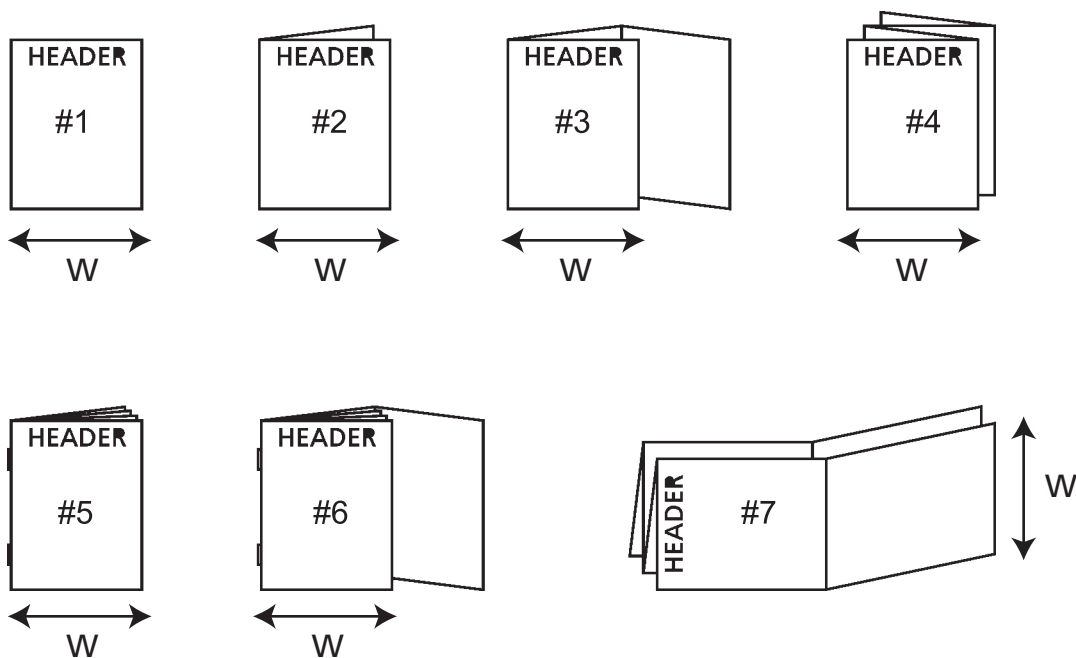- BD Diagnostics inspects per procedure 838-07096-00 upon entry to their plant.
- See specification control no. N/A for material information
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Label Design	REVISED BY By Nancy Carlsen at 8:51 am, Sep 25, 2015	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION.	 BD Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA	
Proofer	PROOFING APPROVED BY By Helaine Jones at 12:47 pm, Sep 25, 2015			
Checked By	THIRD EYE BY By Sarah Cork at 11:31 am, Sep 28, 2015			
Part Number:	779-08495-00	Category and Description		Sheet: 1 of 15
		BD CytoRich Blue Preservative Package Insert	Scale: N/A	

REF 491335	BD CytoRich™ Blue Preservative
REF 491303	BD PrepStain™ Non-GYN Test Kit
REF 491304	BD Totalys™ SlidePrep Non-GYN Test Kit
REF 491333	BD Non-GYN Test Kit
REF 491432	BD Totalys™ Non-GYN Test Kit – Japan
REF 491433	BD PrepStain™ Non-GYN Test Kit – Japan
English: pages 1 – 2 Deutsch: Seiten 5 – 6 Español: páginas 9 – 10 Français : pages 3 – 4 Italiano: pagine 7 – 8 Português: páginas 11 – 12	

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INTENDED USE

BD CytoRich™ Blue Preservative is an alcohol-based preservative fluid intended for preservation of cytology cells in suspension and their subsequent preparation for cytological evaluation.

SUMMARY AND EXPLANATION

Non-cervicovaginal cytologic screening involves the microscopic examination of cell samples that have been harvested from human body sources, including sputum, bronchial washings, breast tissue aspirations, spinal fluid, other fluids, exudates and fine needle aspirates of specific organs. These cell samples are usually fixed in an alcohol-based liquid (e.g. **BD CytoRich Blue Preservative**) and are concentrated with centrifugation. These samples are then either smeared or sedimented onto a glass microscope slide and stained by the Papanicolaou or other staining technique. **BD CytoRich Blue Preservative** is packaged in leak-resistant plastic containers. Its ingredients include alcohols and polyethylene glycol.

LIMITATIONS OF THE PROCEDURE

- Cytologic samples should be fixed in **BD CytoRich Blue Preservative** as soon as possible after collection.
- A sample which has become degraded prior to fixation may be unsatisfactory for cytologic examination.
- All supplies are intended for single use only and cannot be reused.

WARNINGS AND PRECAUTIONS

Cytologic specimens may contain infectious agents. Wear suitable protective clothing, gloves, and eye/face protection. Follow appropriate biohazard precautions when handling samples.

BD CytoRich Blue Preservative contains an aqueous solution of denatured ethanol. The mixture contains methanol and ethylene glycol. Do not ingest.

Danger

H225 Highly flammable liquid and vapor. **H371** May cause damage to organs. **P233** Keep container tightly closed. **P264** Wash thoroughly after handling. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P403+P235** Store in a well-ventilated place. Keep cool. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

779-08495-00(03) 2015-10

BD EA/OG Combo Stain contains ethanol, methanol and acetic acid. Do not ingest.

Danger

H225 Highly flammable liquid and vapor. **H312+H332** Harmful in contact with skin or if inhaled. **H370** Causes damage to organs.

P264 Wash thoroughly after handling. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P312** Call a POISON CENTER or doctor/physician if you feel unwell. **P403+P235** Store in a well-ventilated place. Keep cool. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

BD Hematoxylin Stain 0.75 contains ethylene glycol. Do not ingest.

Warning

H302 Harmful if swallowed. **H315** Causes skin irritation. **H319** Causes serious eye irritation. **H335** May cause respiratory irritation.

P264 Wash thoroughly after handling. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P301+P312** IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. **P403+P233** Store in a well-ventilated place. Keep container tightly closed. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

PRECAUTIONS

- For *in vitro* diagnostic use.
- For professional use only.
- Good laboratory practices should be followed.
- Avoid splashing or generating aerosols. Operators should use appropriate hand, eye/face and clothing protection.
- Do not pipette by mouth.
- Do not ingest (contains denatured alcohol).
- Used supplies and product should be disposed of properly in accordance with institutional and governmental regulations.

MATERIALS REQUIRED

BD CytoRich Blue Preservative can be used with the **BD PrepStain™ Slide Processor** or **BD Totalys™ SlidePrep*** or in a manual process. Refer to the Operator's Manual for the **BD PrepStain Slide Processor** or **BD Totalys SlidePrep** for complete information regarding staining reagents, components and accessories. Not all of the materials listed below are required for preparing slides manually.

Materials Provided

Reagents

- BD CytoRich Blue Preservative**, 3600 mL

Materials Sold Separately by BD

Reagents

- BD Non-GYN Stain Kit**
 - 1 – **BD Hematoxylin Stain 0.5**, 480 mL
 - 1 – **BD EA/OG Combo Stain**, 480 mL

Instruments, Equipment and Accessories

- BD PrepStain Slide Processor**
- BD Totalys SlidePrep***
- Easy Aspirator Kit
- Centrifuge and accessories
- BD Centrifuge Tubes**
- BD SurePath™ PreCoat Slides**
- BD Settling Chambers**
- BD PrepStain Transfer Tips**
- BD Totalys Transfer Tips**
- Kit Alternative #1: BD PrepStain Non-GYN Test Kit**
 - 2 x 96 – **BD Centrifuge Tubes**
 - 2 x 96 – **BD SurePath PreCoat Slides**
 - 2 x 96 – **BD Settling Chambers**
 - 2 x 96 – **BD PrepStain Transfer Tips**

* The **BD Totalys SlidePrep** is not approved for use in all markets. Contact your local sales representative for availability.

- *Kit Alternative #2: **BD Totalys**™ SlidePrep Non-GYN Test Kit**
 - 2 x 96 – **BD** Centrifuge Tubes
 - 2 x 96 – **BD SurePath** PreCoat Slides
 - 2 x 96 – **BD** Settling Chambers
 - 2 x 96 – **BD Totalys** Transfer Tips

Materials Required But Not Provided By BD

- Specimen Collection Container
- Vortex Mixer
- Precision Pipettes with Disposable Tips
- Deionized Water (pH 7.5 to 8.5)
- Reagent Grade Alcohol
- Xylene or Xylene substitute
- Mounted Media
- Coverslips

STORAGE

1. The storage limit of the **BD CytoRich**™ Blue Preservative (without a patient specimen) is up to 24 months from date of manufacture at room temperature (15–30 °C).
2. Do not use fluid beyond the expiration date printed on the label.
3. The storage limit for **BD CytoRich** Blue Preservative with cytology sample should be determined in accordance with your laboratory's procedures.

PROCEDURE

- Label the specimen collection container before use.
- **BD CytoRich** Blue Preservative contains 50% w/v alcohol and should be used by mixing one (1) volume of sample with one (1) volume of **BD CytoRich** Blue Preservative.
- The minimum fixation time is 30 minutes.
- Cytologic preparations may be prepared using the **BD PrepStain**™ Slide Processor, **BD Totalys**™ SlidePrep*, standard cytocentrifugation methods, or smear techniques.

BIBLIOGRAPHY

1. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881-906.
2. Barrett DL: Cytocentrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, pp 80-83.
3. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412-446.
4. Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495.
5. Hajdu SI: A note on the history of Carbo wex. Acta Cytol 1983; 27:204-206.
6. Raistrick J, Shambayati B, Dunsmuir W. Collection fluid helps preservation in voided urine cytology. Cytopathology. 2008 Apr;19(2):111-7.

Technical Information: In the United States contact BD Technical Service and Support at 800-638-8663 or www.bd.com/ds.

* The **BD Totalys** SlidePrep is not approved for use in all markets. Contact your local sales representative for availability.

REF	491335	BD CytoRich Blue Preservative (Conservateur bleu)
REF	491303	BD PrepStain Non-GYN Test Kit (Kit de test non-gyn)
REF	491304	BD Totalys SlidePrep Non-GYN Test Kit (Kit de test non-gyn)
REF	491333	BD Non-GYN Test Kit (Kit de test non gynécologique)
REF	491432	BD Totalys Non-GYN Test Kit (Kit de test non-gyn) – Japon
REF	491433	BD PrepStain Non-GYN Test Kit (Kit de test non-gyn) – Japon

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TriPath Imaging, Inc.
Benex Limited
Becton Dickinson Pty Ltd.