Part Number:	500017021	BALTSO0191 Version 13.0 Template 4 Inserts	
Category and Description: BD CytoRich™ Blue Preservative Package Insert	Rev from: 02 Rev to: 03		
	Job Number: 692-18		

Catalog Number: 491335, 491303, 491304, 491459

Blank (Sheet) Size: Length: 11" Width: 8.5"

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
Number of Pages: 12 Number of Sheets: 6

Page Size: Length: 11" Width: 8.5" Final Folded Size: 5.5"L X 8.5"W

Ink Colors: Number of Colors: 2 PMS #: 032 Red; PMS Standard Black

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Revised By: REVISED BY
By Tori Pagani at 9:35 am, Sep 11, 2018

Proofing Approved By: PROOFING APPROVED BY
By Terrence Means at 12:39 pm, Sep 11, 2018

Third Eye By: THIRD EYE BY
By Daria Kuznetsova at 12:57 pm, Sep 11, 2018

REF	491335	BD CytoRich™ Blue Preservative
REF	491303	BD PrepStain™ Non-GYN Test Kit
REF	491304	BD Totalys™ SlidePrep Non-GYN Test Kit
REF	491459	BD Non-GYN Stain Kit
English: pages 1 – 2 Deutsch: Seiten 4 – 5 Español: páginas 7 – 8 Français: pages 2 – 3 Italiano: pagine 5 – 6 Português: páginas 8 – 9		

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokyny vám poskytne miestni zástupce spoločnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteyts lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőjétől. / Нұсқаулар үшін жергілікті BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielem BD. / Contacte o reprezentante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указаний обратитесь к местному представителю компании BD. / Instrukcie ziskate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD

INTENDED USE

BD CytoRich™ Blue Preservative is an alcohol-based preservative liquid intended for preservation of cytology cells in suspension and their subsequent preparation for cytological evaluation.

SUMMARY AND EXPLANATION

Non-cervicovaginal cytologic screening involves the microscopic examination of cell samples that have been harvested from human body sources, including sputum, bronchial washings, breast tissue aspirations, spinal fluid, other fluids, exudates and fine needle aspirates of specific organs. These cell samples are usually fixed in an alcohol-based liquid (e.g. BD CytoRich Blue Preservative) and are concentrated with centrifugation. These samples are then either smeared or sedimented onto a glass microscope slide and stained by the Papanicolaou or other staining technique. BD CytoRich Blue Preservative is packaged in leak-resistant plastic containers. Its ingredients include alcohols and polyethylene glycol.

LIMITATIONS OF THE PROCEDURE

- Cytologic samples should be fixed in BD CytoRich Blue Preservative as soon as possible after collection.
- A sample which has become degraded prior to fixation may be unsatisfactory for cytologic examination.
- All supplies are intended for single use only and cannot be reused.

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BD CytoRich Blue Preservative contains an aqueous solution of denatured ethanol. The mixture contains methanol and ethylene glycol. Do not ingest.

Danger

H225 Highly flammable liquid and vapor. **H371** May cause damage to organs. **P210** Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P241** Use explosion-proof electrical/ventilating/lighting equipment. **P260** Do not breathe dust/fume/gas/mist/vapors/spray. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P240** Ground/bond container and receiving equipment. **P233** Keep container tightly closed. **P242** Use only non-sparking tools. **P243** Take precautionary measures against static discharge. **P264** Wash thoroughly after handling.

P270 Do not eat, drink or smoke when using this product. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P370+P378** In case of fire: Use for extinction: CO₂, powder or water spray. **P308+P311** IF exposed or concerned: Call a POISON CENTER or doctor/physician. **P405** Store locked up. **P403+P235** Store in a well-ventilated place. Keep cool. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

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BD Hematoxylin Stain 0.50 contains ethylene glycol. Do not ingest.

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P261 Avoid breathing dust/fume/gas/mist/vapors/spray. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P301+P312** IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. **P330** Rinse mouth. **P302+P352** IF ON SKIN: Wash with plenty of soap and water. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P332+P313** If skin irritation occurs: Get medical advice/attention. **P337+P313** If eye irritation persists: Get medical advice/attention. **P403+P233** Store in a well-ventilated place. Keep container tightly closed. **P405** Store locked up. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

PRECAUTIONS

1. For *in vitro* diagnostic use.
2. For professional use only.
3. Good laboratory practices should be followed.
4. Avoid splashing or generating aerosols. Operators should use appropriate hand, eye/face and clothing protection.
5. Do not pipette by mouth.
6. Do not ingest (contains denatured alcohol).
7. Used supplies and product should be disposed of properly in accordance with institutional and governmental regulations.

MATERIALS REQUIRED

BD CytoRich Blue Preservative can be used with the BD PrepStain™ Slide Processor or BD Totalys™ SlidePrep or in a manual process. Refer to the Operator's Manual for the BD PrepStain Slide Processor or BD Totalys SlidePrep for complete information regarding staining reagents, components and accessories. Not all of the materials listed below are required for preparing slides manually.

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Materials Required But Not Provided By BD

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STORAGE

1. The storage limit of the **BD CytoRich Blue Preservative** (without a patient specimen) is up to 24 months from date of manufacture at room temperature (15–30 °C).
2. Do not use fluid beyond the expiration date printed on the label.
3. The storage limit for **BD CytoRich Blue Preservative** with cytology sample should be determined in accordance with your laboratory's procedures.

PROCEDURE

- Label the specimen collection container before use.
- **BD CytoRich Blue Preservative** contains 50% w/v alcohol and should be used by mixing one (1) volume of sample with one (1) volume of **BD CytoRich Blue Preservative**.
- The minimum fixation time is 30 minutes.
- Cytologic preparations may be prepared using the **BD PrepStain Slide Processor**, **BD Totalys SlidePrep**, manual methods, cytocentrifugation methods, or smear techniques.

BIBLIOGRAPHY

1. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881–906.
2. Barrett DL: Cytocentrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, pp 80–83.
3. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412–446.
4. Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495.
5. Hajdu SI: A note on the history of Carbo wex. Acta Cytol 1983; 27:204–206.
6. Raistrick J, Shambayati B, Dunsmuir W. Collection fluid helps preservation in voided urine cytology. Cytopathology. 2008 Apr;19(2):111–7.

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Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:
Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

© 2018 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.

Part Number:	500017022	BALTSO0191 Version 13.0 Template 4 Inserts	
Category and Description: Package Insert, BD CytoRich™ Red Preservative	Rev from: 03		Rev to: 04
	Job Number:		618-18

Catalog Number: 491336, 491303, 491304, 491459

Blank (Sheet) Size: Length: 11" Width: 8.5"

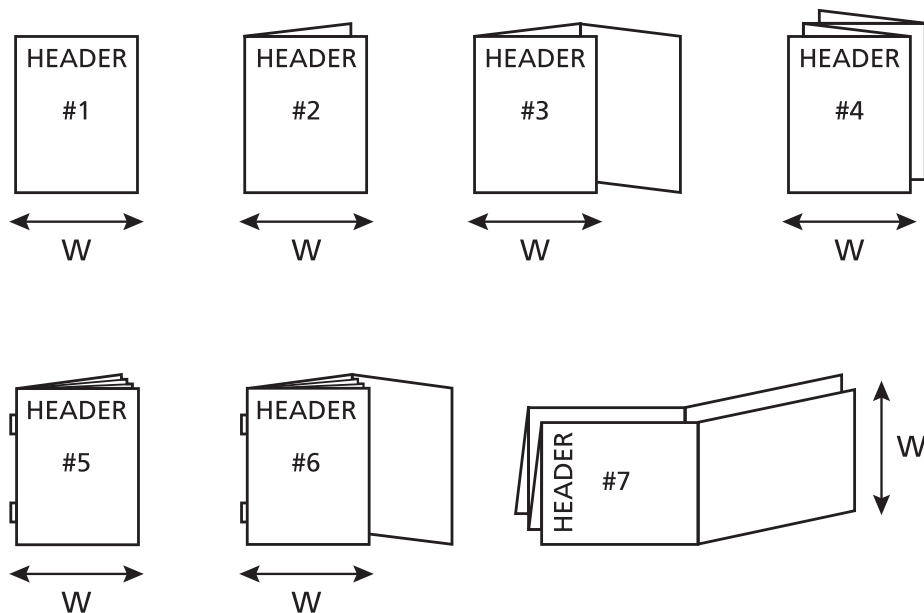
Number of Pages: 20 Number of Sheets: 10

Page Size: Length: 11" Width: 8.5" Final Folded Size: 8.5" X 11"

Ink Colors: Number of Colors: 2 PMS #: 032 Red; Standard Black

Printed Two Sides: Yes: ☒ No: ☐

Style (see illustrations below): # 1 (3-hole punched); Stapled top left corner

Vendor Printed: ☒ Online / In House Printed: ☐ Web Printed: ☐

See Specification control no. N/A for material information.

Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Company confidential. This document is the property of Becton, Dickinson and Company and is not to be used outside the company without written permission. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Revised By: **REVISED BY**
By Tori Pagani at 11:50 am, Jun 22, 2018

Proofing Approved By: **PROOFING APPROVED BY**
By Terrence Means at 12:10 pm, Jun 22, 2018

Third Eye By: **THIRD EYE BY**
By Daria Kuznetsova at 12:33 pm, Jun 22, 2018

REF	491336	BD CytoRich™ Red Preservative
REF	491303	BD PrepStain™ Non-GYN Test Kit
REF	491304	BD Totalys™ SlidePrep Non-GYN Test Kit
REF	491459	BD Non-GYN Stain Kit
English: pages 1 – 3 Deutsch: Seiten 7 – 9 Español: páginas 13 – 15 Français: pages 4 – 6 Italiano: pagine 10 – 12 Português: páginas 16 – 18		

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokyny vám poskytne místní zástupce společnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteyttä lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőjétől. / Нұсқаулар үшін жергілікті BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielem BD. / Contacte o reprezentante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указаний обратитесь к местному представителю компании BD. / Instrukcie získate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD.

INTENDED USE

1. **BD CytoRich™ Red Preservative** is intended to preserve cells and small tissue fragments in suspension.
2. **BD CytoRich Red Preservative** is used to prepare cells and small tissue fragments for cytological and histological examination.
3. **BD CytoRich Red Preservative** lyses red blood cells and solubilizes proteins.
4. The preserved samples are compatible with immuno-histochemical staining and results are similar to those achieved with neutral buffered formalin.

SUMMARY AND EXPLANATION

BD CytoRich Red Preservative is an alcohol-based hemolytic fixative. It is for cytology and small biopsy processing. It is packaged in leak-resistant plastic containers. Its ingredients include alcohols, formaldehyde, and buffers.

BD CytoRich Red Preservative is used for diagnostic non-cervicovaginal cytology. In this context, it is a fixative for cells and small tissue samples that have been collected from human body sources including, but not limited to, sputum, washings, brushings, scrapings, body fluids, exudates, and fine needle aspirates.

MECHANISMS OF HEMOLYSIS AND FIXATION

BD CytoRich Red Preservative lyses red blood cells, cross-links soluble globulin proteins, and the alcohol fixes cells and small tissue fragments. One milliliter of **BD CytoRich Red Preservative** can lyse red blood cells and stabilize soluble proteins from 25–50 microliters of whole blood.

1. Red blood cell membranes are selectively emulsified and further disrupted by a combination of alcohol and osmotic lysis. The red blood cells of rare individuals with hemoglobinopathy may lyse slowly or resist complete lysis. Senescent red blood cells resist lysis.
2. **BD CytoRich Red Preservative** contains less than 1% formaldehyde. Globulin proteins that are released from red blood cells and collected with blood plasma and interstitial tissue fluids are cross-linked by formaldehyde in solution before they are denatured by alcohols and are thus prevented from forming protein super-aggregates (flocculent precipitates).
3. Other cell membranes act as selective barriers that allow alcohols to pass more rapidly than formaldehyde into cells' cytoplasm and nucleoplasm. This causes cells to appear alcohol fixed.

4. Subsequently, intercellular junctions and intracellular proteins are cross-linked by formaldehyde, which accounts for the long-term stability of the cells and tissue fragments in suspension.

LIMITATIONS OF THE PROCEDURE

1. **BD CytoRich Red Preservative** is not sold as a fixative for screening cervicovaginal cytology.
2. Cytologic samples should be fixed in **BD CytoRich Red Preservative** as soon as possible after collection.
3. A sample that has become degraded prior to fixation will be unsatisfactory for examination.
4. **BD CytoRich Red Preservative** should not be used to fix tissue fragments larger than 5 millimeters in average diameter.
5. All supplies are intended for single use only and cannot be reused.

WARNINGS AND PRECAUTIONS

Cytologic specimens may contain infectious agents. Wear suitable protective clothing, gloves, and eye/face protection. Follow appropriate biohazard precautions when handling samples.

BD CytoRich Red Preservative is an aqueous solution containing isopropanol, methanol, ethylene glycol, and formaldehyde. Do not ingest.

Danger

H226 Flammable liquid and vapor. **H302** Harmful if swallowed.

H319 Causes serious eye irritation. **H317** May cause an allergic skin reaction. **H350** May cause cancer. **H371** May cause damage to organs.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P233** Keep container tightly closed. **P240** Ground/bond container and receiving equipment.

P241 Use explosion-proof electrical/ventilating/lighting equipment.

P242 Use only non-sparking tools. **P243** Take precautionary

measures against static discharge. **P280** Wear protective gloves/

protective clothing/eye protection/face protection. **P264** Wash

thoroughly after handling. **P270** Do not eat, drink or smoke when

using this product. **P272** Contaminated work clothing should not

be allowed out of the workplace. **P201** Obtain special instructions

before use. **P202** Do not handle until all safety precautions have

been read and understood. **P281** Use personal protective equipment

as required. **P260** Do not breathe dust/fume/gas/mist/vapors/spray.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for

several minutes. Remove contact lenses, if present and easy to do.

Continue rinsing. **P337+P313** If eye irritation persists: Get medical

advice/attention. **P303+P361+P353** IF ON SKIN (or hair): Take

off immediately all contaminated clothing. Rinse skin with water/

shower. **P333+P313** If skin irritation or rash occurs: Get medical

advice/attention. **P301+P312** IF SWALLOWED: Call a POISON

CENTER or doctor/physician if you feel unwell. **P330** Rinse mouth.

P309+P311 IF exposed or if you feel unwell: Call a POISON CENTER

or doctor/physician. **P321** Specific treatment (see on this label).

P363 Wash contaminated clothing before reuse. **P370+P378** In case

of fire: Use for extinction: CO₂, powder or water spray. **P403** Store

in a well-ventilated place. **P235** Keep cool. **P405** Store locked up.

P501 Dispose of contents/container in accordance with local/regional/

national/international regulations.

BD EA/OG Combo Stain contains ethanol, methanol and acetic acid. Do not ingest.

Danger

H225 Highly flammable liquid and vapor. **H312+H332** Harmful in contact with skin or if inhaled. **H370** Causes damage to organs.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P233** Keep container tightly closed. **P240** Ground/bond container and receiving equipment.

P241 Use explosion-proof electrical/ventilating/lighting/equipment.

P242 Use only non-sparking tools. **P243** Take precautionary measures against static discharge. **P260** Do not breathe dust/fume/gas/mist/vapours/spray. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P312** Call a POISON CENTER or doctor/physician if you feel unwell. **P362+P364** Take off contaminated clothing and wash it before reuse. **P370+P378** In case of fire: Use for extinction: CO₂, powder or water spray. **P403+P235** Store in a well-ventilated place. Keep cool. **P405** Store locked up.

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

BD Hematoxylin Stain 0.5 contains ethylene glycol. Do not ingest.

Warning

H302 Harmful if swallowed. **H315** Causes skin irritation. **H319** Causes serious eye irritation. **H335** May cause respiratory irritation.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area.

P280 Wear protective gloves/protective clothing/eye protection/face protection. **P301+P312** IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. **P302+P352** IF ON SKIN: Wash with plenty of soap and water. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P330** Rinse mouth. **P332+P313** If skin irritation occurs: Get medical advice/attention. **P337+P313** If eye irritation persists: Get medical advice/attention. **P362** Take off contaminated clothing. **P403+P233** Store in a well-ventilated place. Keep container tightly closed. **P405** Store locked up. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

PRECAUTIONS

1. For *in vitro* diagnostic use.
2. For professional use only.
3. Good laboratory practices should be followed.
4. Avoid splashing or generating aerosols. Operators should use appropriate hand, eye/face and clothing protection.
5. Do not pipette by mouth.
6. Do not ingest (contains alcohols other than ethanol and formaldehyde).
7. Used supplies and product should be disposed of properly in accordance with institutional and governmental regulations.

MATERIALS REQUIRED

BD CytoRich Red Preservative can be used with the **BD PrepStain™** Slide Processor or **BD Totalys™** SlidePrep or in a manual process. Refer to the Operator's Manual for the **BD PrepStain** Slide Processor or **BD Totalys** SlidePrep for complete information regarding staining reagents, components and accessories. Not all of the materials listed below are required for preparing slides manually.

Materials Provided

Reagents

- **BD CytoRich Red Preservative**, 3,600 mL

Materials Sold Separately by BD

Reagents

- **BD Non-GYN Stain Kit**
 - 1 – **BD Hematoxylin Stain 0.5**, 480 mL
 - 1 – **BD EA/OG Combo Stain**, 480 mL

Instruments, Equipment and Accessories

- **BD PrepStain** Slide Processor
- **BD Totalys** SlidePrep
- Easy Aspirator Kit
- Centrifuge and accessories
- **BD** Centrifuge Tubes
- **BD SurePath™** PreCoat Slides
- **BD** Settling Chambers
- **BD PrepStain** Transfer Tips
- **BD Totalys** Transfer Tips
- *Kit Alternative #1: **BD PrepStain Non-GYN Test Kit***
 - 2 x 96 – **BD** Centrifuge Tubes
 - 2 x 96 – **BD SurePath** PreCoat Slides
 - 2 x 96 – **BD** Settling Chambers
 - 2 x 96 – **BD PrepStain** Transfer Tips
- *Kit Alternative #2: **BD Totalys SlidePrep Non-GYN Test Kit***
 - 2 x 96 – **BD** Centrifuge Tubes
 - 2 x 96 – **BD SurePath** PreCoat Slides
 - 2 x 96 – **BD** Settling Chambers
 - 2 x 96 – **BD Totalys** Transfer Tips

Materials Required But Not Provided by BD

- Specimen Collection Container
- Vortex Mixer
- Precision Pipettes with Disposable Tips
- Deionized Water (pH 7.5 to 8.5)
- Reagent Grade Alcohol
- Xylene or Xylene substitute
- Mounted Media
- Coverslips

STORAGE

1. The storage limit of the **BD CytoRich** Red Preservative is up to 24 months from date of manufacture (without a patient specimen) at room temperature (15–30 °C).
2. Do not use fluid beyond the expiration date printed on the label.
3. The storage limit for **BD CytoRich** Red Preservative with cytology sample should be determined in accordance with your laboratory's procedures.

GENERAL SPECIMEN PREPARATION, BACKGROUND

1. Label the specimen collection container before use.
2. One milliliter of **BD CytoRich** Red Preservative can lyse and stabilize 25–50 microliters of whole blood. Using this as a guideline, one drop (50 microliters) of aqueous cell slurry may be cleared of its red cells and protein and fixed by one milliliter of **BD CytoRich** Red Preservative.
3. The minimum fixation time is 30 minutes.
4. Cytologic preparations may be prepared using the **BD PrepStain** Slide Processor, **BD Totalys** SlidePrep, standard cytocentrifugation methods, or smear techniques.
5. Refer to the **BD CytoRich** Non-GYN Procedures and Protocols Manual for complete information regarding specific specimen handling and collection procedures.

GENERAL SPECIMEN PREPARATION, METHOD

If fresh (unfixed) samples are used, follow General Procedure Steps 1 through 4.

If fixed samples are used, follow General Procedure Steps 1, 3 and 4.

GENERAL PROCEDURE

1. Concentration
 - a. Mix to homogenize the sample by simple inversion or vortexing.
 - b. Assess the sample for and remove clots, particles, and mucus.
 - i. It is important to remove any visible clots or particles. Failure to do so may cause clogs in the **BD PrepStain** or **BD Totalys** SlidePrep vacuum bundles. Filtering may be accomplished by pouring the specimen through any course medium such as gauze. Any captured material may be used for cell block procedures.
 - ii. Follow standard laboratory procedures to homogenize and/or emulsify mucus.
 - c. Transfer an aliquot, or for small volume samples the entire amount to a 50 mL centrifuge tube.
 - d. Centrifuge for 10 minutes at 600 g.
 - e. Decant the supernatant fluid.
 - f. Vortex Sample for 15 ± 5 seconds to homogenize sample.
2. Fixation
 - a. Depending on cell pellet volume, add enough **BD CytoRich** Red Preservative to cover the sample in a 1:1 ratio.
 - b. Vortex for 15 ± 5 seconds and let sit for a minimum of 30 minutes.
 - c. Centrifuge for 10 minutes at 600 g.
 - d. Decant supernatant and vortex.

3. Washing

- a. If no visible or small pellet is identified, add 6–10 mL of buffered DI water directly to 50 mL centrifuge tube, vortex for 15 ± 5 seconds and transfer entire contents to a 12 mL tube.
- b. If moderate to large pellet is identified, transfer a representative sample (1–5 drops) to 12 mL tube and add 10 mL buffered DI water.
- c. Transfer the 12 mL tubes to centrifuge tube holder(s) and centrifuge for 5 minutes at 600 g.
- d. Decant and vortex for 15 ± 5 seconds to homogenize sample.
 - i. Decanting should be done in a single, rapid motion by inverting each tube rack 180 degrees so as to not disturb the cell pellet.
 - ii. While inverted, carefully blot the mouth of all tubes in the rack with absorbent paper.
 - iii. Turn the rack upright after 3 to 5 seconds.

4. Processing

- a. Load the labeled 12 mL centrifuge tube holders onto the **BD PrepStain** Slide Processor or the **BD Totalys** SlidePrep for processing.
- b. Follow 'Non-GYN Slide Processing' instructions in the applicable User's Manual.

BIBLIOGRAPHY

1. Hansen T, Pedersen H, Brauner V, Hariri J. Control specimens for immunocytochemistry in liquid-based cytology. *Cytopathology*. 2011 Aug;22(4):243-6.
2. Raistrick J, Shambayati B, Dunsmuir W. Collection fluid helps preservation in voided urine cytology. *Cytopathology*. 2008 Apr;19(2):111-7.
3. Gottschall EB, McGinley JN, Spoelstra N, Knott K, Wolfe P, Rose C, Singh M, Thompson HJ. Effect of cytological fixative and environmental conditions on nuclear morphometric characteristics of squamous epithelial cells in sputum. *Cytometry B Clin Cytom*. 2005 Sep;67(1):19-26.
4. Davis-Devine S, Day SJ, Freund GG. New red blood cell lysing fixative for use in fine needle aspiration and fluid cytology. *Acta Cytol*. 2003 Jul-Aug;47(4):630-6.
5. Tao LC, Smith JW. Fine-needle aspiration biopsy using a newly-developed pencil-grip syringe holder. *Diagn Cytopathol*. 1999 Feb;20(2):99-104.
6. Yang GC, Wan LS, Papella J, Waisman J. Compact Cell Blocks. Use for Body Fluids, Fine Needle Aspirations and Endometrial Brush Biopsies. *Acta Cytol* 1998. May-Jun;42(3):703-706.
7. Weidmann J, Chaubal A, Bibbo M. Cellular fixation. A study of CytoRich Red and Cytospin Collection Fluid. *Acta Cytol*. 1997 Jan-Feb;41(1):182-7.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

REF 491336	BD CytoRich Red Preservative (Conservateur rouge)
REF 491303	BD PrepStain Non-GYN Test Kit (Kit de tests non-gyn)
REF 491304	BD Totalys SlidePrep Non-GYN Test Kit (Kit de tests non-gyn)
REF 491459	BD Non-GYN Stain Kit (Trousse de coloration non gynécologique)

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For US: "For Investigational Use Only"

Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuriipiir / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шери / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrænse / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sicaklık alt sınırı / Мінімальна температура / 温度下限

Control / Контролно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Controllo / Бақылау / 컨트롤 / Kontrolé / Kontrolle / Controle / Controllo / Контроль / kontroll / Контроль / 对照

Positive control / Положительный контроль / Pozitivní kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiveve kontroll / Contrôle positif / Pozitivna kontrola / Pozitiv kontroll / Controllo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívá kontrolle / Positiveve controle / Controllo dodatnia / Controllo positivo / Control pozitiv / Положительный контроль / Pozitif kontrol / Позитивный контроль / 阳性对照试剂

Negative control / Отрицательный контроль / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negativeve kontroll / Contrôle négatif / Negativna kontrola / Negativ kontroll / Controllo negativo / Негативтік бақылау / 음성 컨트롤 / Neigiama kontrolė / Negatívá kontrolle / Negativeve controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативный контроль / 阴性对照试剂

	Method of sterilization: ethylene oxide / Μεθοδoς στeριλiζαcιoς: eθυλενωxιδo / Steriliseringsmetode: eθυλενωxιδo / Sterilisationsmethode: Eθυλενωxιδo / Μέθοδος αποστέρησης: αεθυλενωξείδιο / Método de esterilización: óxido de etileno / Steriliseerimismetod: eütleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módsszere: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Sterilizacija: adici – eθιλεn totγiγi / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksīds / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringsmetode: etylenoksid / Metoda sterilizacji: tlenek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metóda sterilizácie: etylenoxid / Metoda sterilizacije: etilen oksid / Steriliseringsmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: етиленоксидом / 灭菌方法: 环氧乙烷		Do not use if package damaged / He ισπολζαvαιτε, ακο οπακoκvατα ε ποvρεδeνα / Nepoužívejte, je-li obal poškozený / Má ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packungnicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιό. / No usar si el paquete está dañado / Mitte kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használja, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Erep pakert bryzany bolca, pайдaлaнбa / Пакетизиға сонaтiн жeғу сaғуp жiзi / Jei pakuotė pažeista, nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Má ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / He использовать при повреждении упаковки / Nepoužívejte, ak je obal poškozený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / He використувувати за пошкодженої упаковки / 如果包装破损, 请勿使用
	Method of sterilization: irradiation / Μεθοδoς στeριλiζαcιoς: ιραδιαcιa / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστέρησης: ακτινοβολία / Método de esterilización: irradiación / Steriliseerimismetod: kiirgus / Méthode de stérilisation : irradiation / Metoda sterilizacije: zračenje / Sterilizálás módsszere: besugárzás / Metodo di sterilizzazione: irradiazione / Sterilizacija: adici – cəəle təcisi / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringsmetode: bestråling / Metoda sterilizacji: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: iradiere / Метод стерилизации: облучение / Metóda sterilizácie: ožiarenie / Metoda sterilizacije: ozračavanje / Steriliseringsmetod: strålning / Sterilizasyon yöntemi: irradyasyon / Метод стерилизації: опроміненням / 灭菌方法: 辐射		Keep away from heat / Παζετε οτ τοπινα / Nevystavujte přehřnému teplu / Má ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgusest / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Сапкын жерде сакта / Ялы өйөөйа ҳап / Laikyti atokiau nuo šilumos šaltinių / Sargāt no karstuma / Beschermen tegen warmte / Má ikke utsettes for varme / Przechowywać z dala od źródła ciepła / Manter ao abrigo do calor / A se feri de căldură / He нагpeвaть / Uchovávať mimo zdroja tepla / Držite dalje od toplote / Får ej utsättas för värme / Isidan uzak tutun / Берегти від дії тєпa / 请远离热源
	Biological Risks / Бiологичниpиpискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologilised riskid / Risques biologiques / Biološki rizik / Biológiai veszélyes / Rischio biologico / Биологичнык төуекендер / 생물학적 위험 / Biologinis pavojus / Biologisks riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyolojik Riskler / Біологічна небезпека / 生物学风险		Cut / Срежeтe / Odstřihněte / Klip / Schneiden / Κόψτε / Cortar / Lõigata / Découper / Reži / Vágja ki / Tagliare / Keci / Kiri / Knip / Knippen / Kutt / Odciać / Cortar / Decupați / Отрезать / Odstrihnite / Iseći / Klipp / Kesme / Pozpazati / 剪下
	Caution, consult accompanying documents / Вниманиe, напpавeтe cпpавкa в пpидpукавaщиe дoкyмeнтa / Pozor! Prostudujte si příloženou dokumentaci / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatus! Lugeka kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Upozorenie, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dmesio, žiūrėkite priedamam dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Вниманиe: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Paźnij! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, bilirlikte verilen belgelere başvurun / Увага: див. супутню документацию / 小心, 请参阅附带文档。		Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmedatum / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuupeäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жинаған тізбекүні / 수집 날짜 / Paémimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colecta / Data colectării / Дата сбора / Datum odboru / Datum prikupljanja / Uppsamlingsdatum / Toplama tarihi / Дата забору / 采集日期
	Upper limit of temperature / Горeн лимит на температура / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dovoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның руқсат етілген жоғарғы шегі / 상한 온도 / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurilimiet / Øvre temperaturgrense / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sicaklık üst sınırı / Максимальна температура / 温度上限		Keep away from light / Παζεте οτ cветлiна / Nevystavujte světlu / Má ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svjetla / Fény nem érheti / Tenere al riparo dalla luce / Қаpaңғылаңған жерде ұста / 빛을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Má ikke utsettes for lys / Przechowywać z dala od źródła światła / Manter ao abrigo da luz / Feriti de lumină / Хранит в темноте / Uchovávať mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Işıktan uzak tutun / Берегти від дії світла / 请远离光线
	Hydrogen gas generated / Обpазован e водород газ / Možnost uniklu plynného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadržji hydrogen vodik / Hidrogeén gázt fejleszt / Produzione di gas idrogeno / Газтектеc cυтeрi пaйдa бoлды / 수소 가스 생성됨 / Išskiria vandenilio dujas / Rodas ūdeņradis / Waterstofgas gegenereerd / Hydrogengass generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Выделение водорода / Vyrobené pouzité vodíka / Oslobada se vodonik / Genererad vätagas / Açığa çıkan hidrojen gazı / Реакция з виділенням водню / 会产生氢气		Hydrogen gas generated / Обpазован e водород газ / Možnost uniklu plynného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadržji hydrogen vodik / Hidrogeén gázt fejleszt / Produzione di gas idrogeno / Газтектеc cυтeрi пaйдa бoлды / 수소 가스 생성됨 / Išskiria vandenilio dujas / Rodas ūdeņradis / Waterstofgas gegenereerd / Hydrogengass generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Выделение водорода / Vyrobené pouzité vodíka / Oslobada se vodonik / Genererad vätagas / Açığa çıkan hidrojen gazı / Реакция з виділенням водню / 会产生氢气
	Collection time / Вpеme на cъбиpанe / Čas odběru / Opsamlingstidspunkt / Entnahmeuhrzeit / Ωρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жинау уақыты / 수집 시간 / Paémimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de colecta / Ora colectării / Вpемя cбopа / Doba odboru / Vreme prikupljanja / Uppsamlingstid / Toplama zamanı / Час забору / 采集时间		Patient ID number / ИД номер на пациента / ID pacienta / Patientens ID-nummer / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de ID del paciente / Patsiendi ID / No d'identification du patient / Identifikacijski broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттің идентификациялық нөмірі / 환자 ID 번호 / Paciento identifikavimo numeris / Pacienta ID numurs / Identificationnummer van de patiënt / Pasientens ID-nummer / Numer ID pacienta / Número da ID do doente / Număr ID pacient / Идентификационный номер пациента / Identifikačné číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarası / Идентификатор пациента / 患者标识号
	Peel / Обeлeтe / Otevfete zde / Abn / Abziehen / Αποκολλήστε / Desprendre / Koorida / Décoller / Otvořiti skini / Húzza le / Staccare / Ўcтиғи қабатын алып таста / 벗기기 / Plešti čia / Atliimt / Schillen / Trekk av / Oderwać / Destacar / Se dezlipeste / Отклеить / Odtrhnite / Oljuštiti / Dra isår / Ayırma / Відклеїти / 撕下		Fragile, Handle with Care / Чyпливo, Рaбoтeтe з нeoбoдoдимoтo вниманиe. / Křehké. Při manipulaci postupujte opatrně. / Forsigtig, kan gå i stykker. / Zerbrechlich, vorsichtig handhaben. / Εύθραστο. Χειριστείτε το με προσοχή. / Frágil. Manipular con cuidado. / Öm, käsitsege ettevaatlikult. / Fragile. Manipuler avec précaution. / Lomljivo, rukujte pažljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura. / Сынныш, абайлап пайдаланыңыз. / 조심 깨치기 쉬운 처리 / Trapu, elkites atsargiai. / Trausis; rikoties uzmanigi / Breekbaar, voorzichtig behandelen. / Ömtålig, händter försiktigt. / Krucha zawartość, przenosić ostrożnie. / Frágil, Manuseie com Cuidado. / Frágil, manipulați cu atenție. / Хрупкое! Обращаться с осторожностью. / Křehké, vyžaduje sa opatrná manipulácia. / Lomljivo - rukujte pažljivo. / Bräckligt. Hantera försiktigt. / Kolay Kırılır, Dikkatli Taşın. / Тендітпa, зeртeвaтис з oбeрeжнocтю / 易碎, 小心轻放

	Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA		Benex Limited Pottery Road, Dun Laoghaire Co. Dublin, Ireland	Australian Sponsor: Becton Dickinson Pty Ltd. 4 Research Park Drive Macquarie University Research Park North Ryde, NSW 2113 Australia
---	---	---	---	---

Part Number:	500017023	BALTSO0191 Version 13.0 Template 4 Inserts
Category and Description: Package Insert, BD CytoRich™ Clear Preservative	Rev from: 03	Rev to: 04
	Job Number:	831-18

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
Revised By: **REVISED BY**
By Sonia F.Thompson at 5:25 pm, Nov 05, 2018

Proofing Approved By: **PROOFING APPROVED BY**
By Daria Kuznetsova at 10:44 am, Nov 06, 2018

Third Eye By: **THIRD EYE BY**
By Terrence Means at 11:42 am, Nov 06, 2018

REF	490719	BD CytoRich™ Clear Preservative
REF	491303	BD PrepStain™ Non-GYN Test Kit
REF	491304	BD Totalys™ SlidePrep Non-GYN Test Kit
REF	491459	BD Non-GYN Stain Kit
English: pages 1 – 2 Deutsch: Seiten 5 – 6 Español: páginas 9 – 10 Français : pages 3 – 4 Italiano: pagine 7 – 8 Português: páginas 11 – 12		

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokupny vám poskytné místní zástupce společnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteyttä lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőjétől. / Нұсқаулар үшін жерілікті BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielstwem BD. / Contacte o reprezentante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указаний обратитесь к местному представителю компании BD. / Instrukcie ziskate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD.

INTENDED USE

BD CytoRich™ Clear Preservative is an alcohol-based preservative fluid intended for collection, preparation, and examination of cytology specimens from the human body for diagnostic purposes.

SUMMARY AND EXPLANATION

Non-cervicovaginal cytologic screening involves the microscopic examination of cell samples that have been harvested from human body sources. These cell samples are usually fixed in an alcohol-based liquid (e.g. **BD CytoRich Clear Preservative**) and are concentrated with centrifugation. These samples are then either smeared or sedimented onto a glass microscope slide and stained.

BD CytoRich Clear Preservative is packaged in leak-resistant plastic containers. Its ingredients include alcohols and buffers.

LIMITATIONS OF THE PROCEDURE

- Cytologic samples should be fixed in **BD CytoRich Clear Preservative** as soon as possible after collection.
- A sample that has become degraded prior to fixation will be unsatisfactory for cytologic examination.
- All supplies are intended for single use only and cannot be reused.

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BD CytoRich Clear Preservative contains an aqueous solution of denatured ethanol. The mixture contains small amounts of methanol and isopropanol. Do not ingest.

Warning

H226 Flammable Liquid and Vapour.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P241** Use explosion-proof electrical/ventilating/lighting/equipment. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P240** Ground/bond container and receiving equipment. **P233** Keep container tightly closed. **P242** Use only non-sparking tools. **P243** Take precautionary measures against static discharge. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P370+P378** In case of fire: Use for extinction: CO₂, powder or water spray. **P403+P235** Store in a well-ventilated place. Keep cool. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

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P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P233** Keep container tightly closed. **P240** Ground/bond container and receiving equipment. **P241** Use explosion-proof electrical/ventilating/lighting equipment. **P242** Use only non-sparking tools. **P243** Take precautionary measures against static discharge. **P260** Do not breathe dust/fume/gas/mist/vapors/spray. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P312** Call a POISON CENTER or doctor/physician if you feel unwell. **P321** Specific treatment (see on this label). **P362+P364** Take off contaminated clothing and wash it before reuse. **P370+P378** In case of fire: Use for extinction: CO₂, powder or water spray. **P403+P235** Store in a well-ventilated place. Keep cool. **P405** Store locked up. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

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P261 Avoid breathing dust/fume/gas/mist/vapors/spray. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P301+P312** IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. **P330** Rinse mouth. **P302+P352** IF ON SKIN: Wash with plenty of soap and water. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P332+P313** If skin irritation occurs: Get medical advice/attention. **P337+P313** If eye irritation persists: Get medical advice/attention. **P403+P233** Store in a well-ventilated place. Keep container tightly closed. **P405** Store locked up. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

PRECAUTIONS

1. For *in vitro* diagnostic use.
2. For professional use only.
3. Good laboratory practices should be followed.
4. Avoid splashing or generating aerosols. Operators should use appropriate hand, eye/face and clothing protection.
5. Do not pipette by mouth.
6. Do not ingest (contains denatured alcohol).
7. **BD CytoRich** Clear Preservative was tested for antimicrobial effectiveness against *Escherichia coli*, *Pseudomonas aeruginosa*, *Staphylococcus aureus*, *Candida albicans*, *Mycobacterium tuberculosis* and *Aspergillus niger* and found to be effective.
8. **BD CytoRich** Clear Preservative samples inoculated with 10^6 CFU/mL of each species yielded no growth after 14 days (28 days for *Mycobacterium tuberculosis*) of incubation under standard conditions. However, universal precautions for safe handling of biological fluids should be practiced at all times.
9. Used supplies and product should be disposed of properly in accordance with institutional and governmental regulations.

MATERIALS REQUIRED

BD CytoRich™ Clear Preservative can be used with the **BD PrepStain**™ Slide Processor or **BD Totalys**™ SlidePrep or in a manual process. Refer to the Operator's Manual for the **BD PrepStain** Slide Processor or **BD Totalys** SlidePrep for complete information regarding staining reagents, components and accessories. Not all of the materials listed below are required for preparing slides manually.

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- **BD PrepStain** Slide Processor
- **BD Totalys** SlidePrep
- Easy Aspirator Kit
- Centrifuge and accessories
- **BD Centrifuge Tubes**

- **BD SurePath**™ PreCoat Slides
- **BD Settling Chambers**
- **BD PrepStain** Transfer Tips
- **BD Totalys** Transfer Tips
- **Kit Alternative #1: BD PrepStain Non-GYN Test Kit**
 - 2 x 96 – **BD Centrifuge Tubes**
 - 2 x 96 – **BD SurePath** PreCoat Slides
 - 2 x 96 – **BD Settling Chambers**
 - 2 x 96 – **BD PrepStain** Transfer Tips
- **Kit Alternative #2: BD Totalys SlidePrep Non-GYN Test Kit**
 - 2 x 96 – **BD Centrifuge Tubes**
 - 2 x 96 – **BD SurePath** PreCoat Slides
 - 2 x 96 – **BD Settling Chambers**
 - 2 x 96 – **BD Totalys** Transfer Tips

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STORAGE

1. The storage limit for **BD CytoRich** Clear Preservative without cytologic samples is up to 36 months from date of manufacture at ambient temperature (15–30 °C).
2. Do not use fluid beyond the expiration date printed on the label.
3. The storage limit for **BD CytoRich** Clear Preservative with cytology sample should be determined in accordance with your laboratory's procedures.

PROCEDURE

- Label the specimen collection container before use.
- **BD CytoRich** Clear Preservative should be used by mixing one (1) volume of sample with one (1) volume of **BD CytoRich** Clear Preservative.
- Cytologic preparations may be prepared using the **BD PrepStain** Slide Processor, **BD Totalys** SlidePrep, standard cytocentrifugation methods, or smear techniques.

BIBLIOGRAPHY

1. Raistrick J, Shambayati B, Dunsmuir W. Collection fluid helps preservation in voided urine cytology. *Cytopathology*. 2008 Apr;19(2):111-7.
2. Veneti S et al. Liquid-based cytology in breast fine needle aspiration. Comparison with the conventional smear. *Acta Cytol*. 2003; Mar-Apr;47(2):188-92.
3. Saccomanno G: Sputum fixatives. *Acta Cytol* 1977; 21:495.
4. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881-906.
5. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412-446.

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