

INTENDED USE

BD Difco™ Tinsdale Enrichment Desiccated is used with **BD Difco** Tinsdale Agar Base for primary isolation and differentiation of *Corynebacterium diphtheriae*.

SUMMARY AND EXPLANATION

Tinsdale Agar Base, supplemented with Tinsdale Enrichment, is employed in the cultural diagnosis of diphtheria. Diphtheria, an acute infectious disease primarily of the upper respiratory tract but occasionally of the skin,¹ is caused by toxigenic strains of *Corynebacterium diphtheriae*. The three biotypes are mitis, intermedius and gravis.¹ The signs and symptoms of the disease are a pharyngeal membrane, sore throat, malaise, headache and nausea.² Death can result from respiratory obstruction by the membrane or myocarditis caused by the toxin.²

Tinsdale³ developed a serum-cystine-thiosulfite-tellurite agar medium for the primary isolation and differentiation of *C. diphtheriae*. This formulation distinguished between *C. diphtheriae* and diphtheroids which exhibited similar characteristics. The differential principle is based on the capacity of *C. diphtheriae* to produce a brown or black halo around the colonies.

Billings⁴ simplified Tinsdale Basal Medium by using Proteose Peptone No. 3 as a nutrient source. This modification improved the differential qualities and recovery of *C. diphtheriae*. Moore and Parsons⁵ confirmed the halo formation of *C. diphtheriae* with one exception; *C. ulcerans* occasionally produced colonies similar to *C. diphtheriae* and required biochemical identification.

PRINCIPLES OF THE PROCEDURE

Tinsdale Enrichment Desiccated contains bovine serum and horse serum which provide essential growth factors. L-cystine and sodium thiosulfate provide sulfur for H₂S production. Potassium tellurite is a selective agent. The formation of black to brown halos surrounding the colony results from the reduction of potassium tellurite by H₂S to metallic tellurite.

Stabbing the complete Tinsdale Agar with an inoculating needle accentuates darkening of the medium by *C. diphtheriae*.

REAGENTS**BD Difco Tinsdale Enrichment Desiccated**

Approximate Formula* Per Liter Purified Water

Bovine Serum	333	mL
Horse Serum	380	mL
L-Cystine	2.0	g
Potassium Tellurite	1.4	g
Sodium Thiosulfate	2.8	g

* Adjusted and/or supplemented as required to meet performance criteria.

Warnings and Precautions:

For Laboratory Use.

Observe aseptic techniques and established precautions against microbiological hazards throughout all procedures. After use, prepared plates, specimen containers and other contaminated materials must be sterilized by autoclaving before discarding.

Storage instructions: On receipt, store tubes at 2–8 °C.

PROCEDURE

Material Provided: **BD Difco** Tinsdale Enrichment, Desiccated.

Materials Required But Not Provided: Ancillary culture media, reagents, quality control organisms and laboratory equipment as required for this procedure.

Method of Preparation:

Observe aseptic techniques.

1. Prepare and autoclave **BD Difco** Tinsdale Agar Base according to label directions. Cool to 50–55 °C.
2. Rehydrate **BD Difco** Tinsdale Enrichment Desiccated with 15 mL sterile purified water. Rotate in an end-over-end motion to dissolve completely.
3. Add the rehydrated enrichment to the cooled medium base in the ratio of 15 mL enrichment to 100 mL base. Mix thoroughly.
4. Test samples of the finished product for performance using stable, typical control cultures.

USER QUALITY CONTROL

Prepare Tinsdale Agar per label directions. Inoculate to obtain discrete colonies and stab several times using an inoculating needle; incubate at $35 \pm 2^\circ\text{C}$ for 18–48 hours.

ORGANISM	ATCC®	INOCULUM CFU	RECOVERY	APPEARANCE
<i>Corynebacterium diphtheriae</i> biotype gravis	9060	10^2 – 10^3	Good	Brown with halos
<i>Corynebacterium diphtheriae</i> biotype mitis	8032	10^2 – 10^3	Good	Brown with halos
<i>Klebsiella pneumoniae</i>	13883	10^2 – 10^3	Marked to complete inhibition	—
<i>Streptococcus pyogenes</i>	19615	10^2 – 10^3	Poor to fair	Brown to black without halos

RESULTS

The appearance of brown-black colored colonies surrounded by brown-black halos is presumptive evidence for *C. diphtheriae*.¹

LIMITATIONS OF THE PROCEDURE

1. Tinsdale Agar is not suitable as a primary plating medium, since it may not support the growth of some strains of *C. diphtheriae*.¹
2. *C. ulcerans*, *C. pseudotuberculosis* and (rarely) *Staphylococcus* species may produce a characteristic halo on Tinsdale Agar.¹
3. Do not read Tinsdale Agar early because several organisms may exhibit slight browning on this medium in 18 hours.¹
4. Incubation in 5–10% CO₂ retards the development of halos on Tinsdale Agar.¹
5. On media containing tellurite, diphtheria bacilli are shorter and stain more uniformly; however, granules are less readily observed than when grown on Loeffler's medium.⁶
6. Further biochemical tests may be necessary to distinguish between *C. diphtheriae* and *C. ulcerans* due to similar reactions on this medium.

AVAILABILITY

Cat. No. Description

- 234210 **BD Difco™** Tinsdale Enrichment Desiccated, 6 x 15 mL.
278610 **BD Difco™** Tinsdale Agar Base, 500 g

REFERENCES

1. Isenberg (ed.). 1992. Clinical microbiology procedures handbook, vol. 1. American Society for Microbiology, Washington, D.C.
2. Funke and Bernard. 1999. In Murray, Baron, Pfaller, Tenover and Tenover (ed.), Manual of clinical microbiology, 7th ed. American Society for Microbiology, Washington, D.C.
3. Tinsdale. 1947. J. Pathol. Bacteriol. 59:461.
4. Billings. 1956. An investigation of Tinsdale Tellurite medium: its usefulness and mechanisms of halo-formation. M.S. thesis. University of Michigan, Ann Arbor, Mich.
5. Moore and Parsons. 1958. J. Infect. Dis. 102:88.
6. Bailey and Scott. 1966. Diagnostic microbiology, 2nd ed. The C.V. Mosby Company, St. Louis, Mo.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

Change History

Revision	Date	Change Summary
(02)	2019-08	Quality Control organisms <i>Corynebacterium diphtheriae</i> biotype gravis changed from 8028 to 9060 and <i>Corynebacterium diphtheriae</i> biotype mitis changed from 8024 to 8032.

US Customers only: For symbol glossary, refer to bd.com/symbols-glossary



Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbicante / Аткарушы / 제조업체 / Gamintojas / Ražotājs / Tilvirker / Producent / Producător / Производитель / Výrobca / Proizvođač / Tillverkare / Üretici / Виробник / 生产厂商



Use by / Исползвайте до / Spotřebujte do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / 사용 기한 / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейін пайдалануға / Naudokite iki / Izlietot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Исползовать до / Použít do / Upotrebiti do / Använd före / Son kulanma tarihi / Використати до/line / 使用截止日期

YYYY-MM-DD / YYYY-MM (MM = end of month)
 ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
 AAAA-MM-DD / AAAA-MM (MM = slutning af måned)
 JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)
 EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)
 AAAA-MM-DD / AAAA-MM (MM = fin del mes)
 AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)
 AAAA-MM-JJ / AAAA-MM (MM = fin du mois)
 GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)
 ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja)
 AAAA-MM-GG / AAAA-MM (MM = fine mese)
 ЖЖЖЖ-АА-КК / ЖЖЖЖ-АА / (АА = айдың соңы)
 YYYY-MM-DD/YYYY-MM (MM = 월말)
 MMMM-MM-DD / MMMM-MM (MM = mēnesio pabaiga)
 GGGG-MM-DD/GGGG-MM (MM = mēneša beigas)
 JJJJ-MM-DD / JJJJ-MM (MM = einde maand)
 AAAA-MM-DD / AAAA-MM (MM = sluten av måneden)
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
 AAAA-MM-DD / AAAA-MM (MM = fim do mês)
 AAAA-LZ / AAAA-LZ (LL = sfârșitul lunii)
 ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
 GGGG-MM-DD / GGGG-MM (MM = kraj meseca)
 AAAA-MM-DD / AAAA-MM (MM = sluet av månaden)
 YYYY-AA-GG / YYYY-AA (AA = ayın sonu)
 PPPP-MM-DD / PPPP-MM (MM = кінець місяця)
 YYYY-MM-DD / YYYY-MM (MM = 月末)



Catalog number / Каталоген номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalógusszám / Numero di catalogo / Каталог нөмірі / 카탈로그 번호 / Katalogo / numeris / Kataloga numurs / Catalogus number / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalogové číslo / Kataloški broj / Katalog numarası / Номер за каталогом / 目录号



Authorized Representative in the European Community / Оторизирани представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret repræsentant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatud esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizuirani predstavnik u Europskoj uniji / Meghatalmazott képviselő az Európai Közösségben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті екіл / 유럽 공동체의 위임 대표 / Įgaliotasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autorisert representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupca v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Ανρρα Topluluđu Yetkilil Temsilcisi / Уповноважений представник у країнах ЄС / 欧洲共同体授权代表



In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařizení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiiniparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnosztikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жүргізетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietaisai / Medicinas ierices, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispositiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики in vitro / Medicinska pomůcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diagnostik Tibbi Cihaz / Медицинский прибор для диагностики in vitro / 体外诊断医疗设备



Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrensning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperatuuri piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны шектеу / 온도 제한 / Laikymo temperatūra / Temperatūras ierobežojumi / Temperatuurlimit / Temperaturbegrensning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ohraničenie teploty / Ograničenje temperature / Temperaturgräns / Sicaklık sınırlaması / Обмеження температури / 温度限制



Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Топтама коды / 배치 코드(코드) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot nummer / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партии / 批号 (亚批)



Contains sufficient for <n> tests / Съдържа достатъчно е достатъчно за <n> теста / Dostatečné množství pro <n> testů / Inneholder tilstrækkelig til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Küllaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenido suficiente per <n> test / <n> testrepi үшін жеткілікті / <n> 테스트가 충분히 포함됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor <n> testen / Innholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Conținut suficient pentru <n> teste / Достаточное для <n> тестов(a) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli miktarda içerir / Вистачить для аналізів: <n> / 足够进行 <n> 次检测



Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / 사용 지침 참조 / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i brugsanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se brugsanvisningen / Kullanım Talimatları'na başvurun / Див. інструкції за використання / 请参阅使用说明



Do not reuse / Не използвайте отново / Nepoužívejte opakovaně / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / No reutilizar / Mitte kasutada korduvalt / Ne pas réutiliser / Ne koristiti ponovo / Egyyszer használatos / Non riutilizzare / Пайдаланбаңыз / 재사용 금지 / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolosiți / Не использовать повторно / Nepoužívať opakovane / Ne upotrebljavajte ponovo / Får ej återanvändas / Tekrar kullannmayin / Не використовувати повторно / 请勿重复使用



Serial number / Серийн номер / Sériové číslo / Seriennummer / Seriennummer / Σειριακός αριθμός / N° de serie / Seerianummer / Numéro de série / Serijski broj / Sorozatszám / Numero di serie / Топтамалық нөмірі / 일련 번호 / Serijos numeris / Sērijas numurs / Serie nummer / Numer seryjny / Número de série / Număr de serie / Серийный номер / Seri numarasi / Номер серії / 序列号



For IVD Performance evaluation only / Само за оценка качеството на работа на IVD / Pouze pro vyhodnocení výkonu IVD / Kun til evaluering af IVD ydelse / Nur für IVD-Leistungsbewertungszwecke / Μόνο για αξιολόγηση απόδοσης IVD / Sólo para la evaluación del rendimiento en diagnóstico in vitro / Ainult IVD seadme hindamiseks / Réserve à l'évaluation des performances IVD / Samo u znanstvene svrhe za In Vitro Dijagnostiku / Kizárólag in vitro diagnosztikához / Solo per valutazione delle prestazioni IVD / Жасанды жағдайда «пробирка ішінде» диагностикада тек жұмысты бағалау үшін / IVD 성능 평가에 대해서만 사용 / Tik IVD prietaisų veikimo charakteristikoms tikrinti / Vienīgi IVD darbības novērtēšanai / Uitsluitend voor doeltreffendheidsonderzoek / Kun for evaluering av IVD-ytelse / Tylko do oceny wydajności IVD / Uso exclusivo para avaliação de IVD / Numai pentru evaluarea performanței IVD / Только для оценки качества диагностики in vitro / Určenie iba na diagnostiku in vitro / Samo za procenu učinka u in vitro dijagnostici / Endast för utvärdering av diagnostisk användning in vitro / Yalnızca IVD Performans değerlendirmesi için / Только для оцінювання якості діагностики in vitro / 仅限 IVD 性能评估

For US: "For Investigational Use Only"



Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiir / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrænse / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sıcaklık alt sınırı / Мінімальна температура / 温度下限



Control / Контролно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Controllo / Бақылау / 컨트롤 / Kontrolé / Kontrolle / Controle / Controllo / Контроль / kontroll / Контроль / 对照



Positive control / Положительный контроль / Pozitívna kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitívna kontrola / Pozitív kontroll / Controllo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívna kontrola / Positiieve controle / Kontrola dodatnia / Controllo positivo / Control pozitiv / Положительный контроль / Pozitif kontrol / Позитивный контроль / 阳性对照试剂



Negative control / Отрицательный контроль / Negatívna kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negatívna kontrola / Negatív kontroll / Controllo negativo / Негативный бақылау / 음성 컨트롤 / Neigiama kontrolė / Negatívna kontrola / Negatieve controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативный контроль / 阴性对照试剂



Method of sterilization: ethylene oxide / Метод на стерилизация: этиленов оксид / Způsob sterilizace: ethylenoxid / Steriliseringmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseringmetode: etüleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Стерилизация әдісі – этилен тотығы / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksis / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringmetode: etylenoksid / Metoda sterylizacji: tienek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Метод стерилизації: этиленоксид / Metoda sterilizácie: etylénoxid / Metoda sterilizacije: etilen oksid / Steriliseringmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: этиленоксидом / 灭菌方法: 环氧乙烷



Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Způsob sterilizace: záření / Steriliseringmetode: bestrålning / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseringmetode: kiirgus / Méthode de stérilisation : irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metoda di sterilizzazione: irradiazione / Стерилизация әдісі – сәулелі түсімі / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringmetode: bestrålning / Metoda sterilizacije: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: iradiere / Метод стерилизації: облучение / Metoda sterilizácie: ožiarenie / Metoda sterilizacije: ozračivanje / Steriliseringmetod: strålning / Sterilizasyon yöntemi: ırradyasyon / Метод стерилизації: опроміненням / 灭菌方法: 辐射



Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologiskised riskid / Risques biologiques / Biološki rizik / Biológiallag veszélyes / Rischio biologico / Биологиялык төуекелдер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologiskh risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biolojisk Riskler / Биологічна небезпека / 生物学风险



Caution, consult accompanying documents / Внимание, направьте справка в придружаващите документи / Pozor! Prostudujte si příloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitedokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatus! Lugeka kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Urozoorenje, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dėmesio, žiūrėkite pridėdamus dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацію / 小心, 请参阅附带文档。



Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / 상한 온도 / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrænse / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sıcaklık üst sınırı / Максимальна температура / 温度上限



Keep dry / Пазете сухо / Skladujte v suchém prostředí / Opbevares tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / 건조 상태 유지 / Laikykite sausi / Uzglabā saus / Droog houden / Houdes tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávať v suchu / Držite na suvom mestu / Förvaras tørt / Kuru bir şekilde muhafaza edin / Беретти від вологи / 请保持干燥



Collection time / Време на събиране / Čas odběru / Opsamlingsstidspunkt / Entnahmehurzeit / Ώρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жинау уакыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de colheita / Ora colectării / Время сбора / Doba odberu / Vreme prikupljanja / Uppsamlingstid / Toplama zamanı / Час забору / 采集时间



Peel / Обелете / Otevfete zde / Abn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Үстіңгі қабатын алып таста / 벗기기 / Pléști ăia / Atfimt / Schillen / Trekk av / Oderwać / Destacar / Se dezpește / Отклеить / Odrhniite / Oljuštiti / Dra isår / Ayırma / Відклеїти / 撕下



Perforation / Перфорация / Perforace / Perforering / Διάρθρωση / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Тесик тесу / 절취선 / Perforacija / Perforácia / Perforatie / Perforacja / Perfuração / Perforare / Перфорация / Perforácia / Perforasyon / Перфорация / 穿孔



Do not use if package damaged / Не използвайте, ако опаковката е повредена / Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packungnicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά / No usar si el paquete está dañado / Mitte kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használni, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Егер пакет бүзылган болса, пайдаланба / 패키지가 손상된 경우 사용 금지 / Jei pakuotė pažeista, nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Не использовать при повреждении упаковки / Nepoužívať, ak je obal poškodený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки / 如果包装破损, 请勿使用



Keep away from heat / Πазете от топлина / Nevystavujte přílišnému teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgusest / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Салқын жерде сақта / 열을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródła ciepła / Manter ao abrigo do calor / A se feri de căldură / Не нагревать / Uchovávať mimo zdroja tepla / Držite dalje od toplote / Får ej utsättas för värme / Іsідан узак тутун / Берегти від дії тепла / 请远离热源



Cut / Срезете / Odstřihněte / Klip / Schneiden / Κόψτε / Cortar / Lõigata / Découper / Reži / Vágja ki / Tagliare / Keciñiz / 잘라내기 / Kirpti / Nogriez / Knippen / Kutt / Odciąć / Cortar / Decupați / Отрезать / Odstrihnite / Iseći / Klipp / Kesme / Поспізати / 剪下



Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmedatum / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuurpäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жинаған тізбекүні / 수집 날짜 / Paēmimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colheita / Data colectării / Дата сбора / Dátum odberu / Datum prikupljanja / Uppsamlingsdatum / Toploma tarihi / Дата забору / 采集日期



µL/test / µL/тест / µL/Test / µL/εξέταση / µL/prueba / µL/teszt / µL/테스트 / мкл/тест / µL/tyrimas / µL/pārbaude / µL/teste / мкл/анализ / µL/检测



Keep away from light / Πазете от светлина / Nevystavujte svētlū / Må ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svjetla / Fény nem érheti / Tenere al riparo dalla luce / Қараңғыланған жерде ұста / 빛을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Må ikke utsettes for lys / Przechowywać z dala od źródła światła / Manter ao abrigo da luz / Feriți de lumină / Хранить в темноте / Uchovávať mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Іsіktan uzak тутун / Берегти від дії світла / 请远离光线



H₂ Hydrogen gas generated / Образуван е водород газ / Možnost úniku plyného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadržaj hydrogen vodik / Hidrogén gázt fejleszt / Produzione di gas idrogeno / Газтөктес сутері пайда болды / 수소 가스 생성됨 / Išskiria vandenilio dujas / Rodas ūdeņradis / Waterstofgas gegenereerd / Hydrogengass generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Выделение водорода / Vyrobené použitím vodíka / Oslobada se vodonik / Genererad vätegas / Açığa çıkan hidrojen gazı / Реакція з виділенням водню / 会产生氢气



Patient ID number / ИД номер на пациента / ID pacienta / Patientens ID-nummer / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de ID del paciente / Patsiendi ID / No d'identification du patient / Identifikacijski broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттің идентификациялық нөмірі / 환자 ID 번호 / Paciento identifikavimo numeris / Pacienta ID numurs / Identificatienummer van de patiënt / Pasientens ID-nummer / Numer ID pacienta / Número da ID do doente / Număr ID pacient / Идентификационный номер пациента / Identifikačné číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarası / Идентификатор пациента / 患者标识号



Fragile, Handle with Care / Чупливо, Роботете с необходимото внимание. / Křehké. Při manipulaci postupujte opatrně. / Forsiktig, kan gå i stykker. / Zerbrechlich, vorsichtig handhaben. / Εύθραστο. Χειριστείτε το με προσοχή. / Frágil. Manipular con cuidado. / Óm, käsitsege ettevaatlikult. / Fragile. Manipuler avec précaution. / Lomljivo, rukujte pažljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura. / Сығыш, абайлап пайдаланыңыз. / 조심 깨지기 쉬운 처리 / Trapu, elkitės atsargiai. / Trausls; rūkotiės uzmanīgi / Breekbaar, voorzichtig behandelen. / Ømtålig, håndter forsiktig. / Krucha zawartość, przenosić ostrożnie. / Frágil, Manuseie com Cuidado. / Frágil, manipulați cu atenție. / Хрупкое! Обращаться с осторожностью. / Křehké, vyžaduje sa opatrná manipulácia. / Lomljivo - rukujte pažljivo. / Bräckligt. Hantera försiktigt. / Kolay Kırılır, Dikkatli Taşıyın. / Тендітна, звертатися з обережністю / 易碎，小心轻放



Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA



Becton, Dickinson France,
S.A.38800 Le Pont de Claix, France

ATCC® is a trademark of the American Type Culture Collection.

BD, the BD Logo, and Difco are trademarks of Becton, Dickinson and Company or its affiliates. © 2019 BD. All rights reserved.