
Revisions

BALTSO0191 Version 10.0 Template 4
Inserts

Rev from	Rev to	JOB #
07	08	8763-16

NOTES:

- BD Catalog Number: 446255
- Blank (Sheet) Size: Length: See Spec Width: See Spec
- Number of Pages: 18 Number of Sheets: See Spec
- Page Size: Length: See Spec Width: See Spec Final Folded Size: See Spec
- Ink Colors: No. of Colors: 2 PMS#: 032 Red, Standard Black
- Printed two sides: Yes ☒ No ☐
- Style (see illustrations below): # See Spec

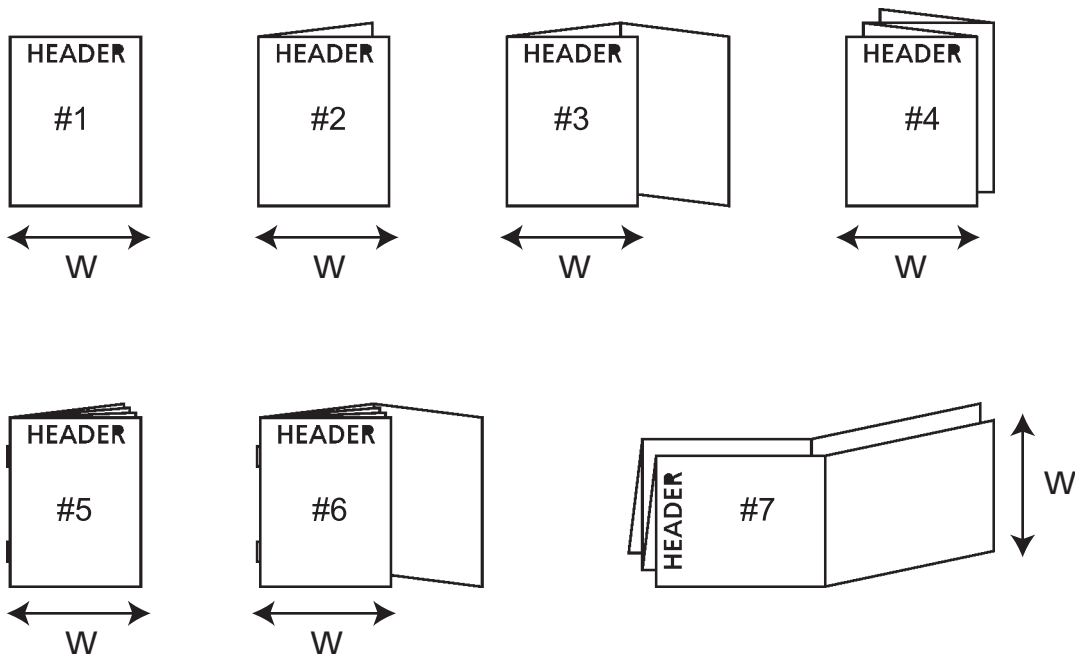- Vendor Printed ☒ Online/In House Printed ☐ Web ☐
- See Nogales Spec D13467RM
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Label Design	REVISED BY By Sonia Thompson at 1:32 pm, Oct 06, 2016	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION.	 BD Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA
Proofer	PROOFING APPROVED BY By Natalie Morio at 7:39 am, Oct 07, 2016		
Checked By	THIRD EYE BY By Nichole Graham at 11:18 am, Oct 11, 2016		
Part Number:	L005510	Category and Description Package Insert, BD Affirm™ VPill AmbientTemperature Transport System	Sheet: 1 of 19 Scale: N/A A

BD Affirm™ VPIII Ambient Temperature Transport System

For collection and extended transport of vaginal specimens for use in the BD Affirm VPIII Microbial Identification Test.

English: pages 1 – 3 Italiano: pagine 9 – 11
Français : pages 3 – 6 Español: páginas 11 – 14
Deutsch: Seiten 6 – 8

L005510(04)
2016-10

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokyny vám poskytne miestni zástupce spoločnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteys lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőtől. / Нұсқаулар үшін жергілікті BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielstwem BD. / Contacte o representante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указаний обратитесь к местному представителю компании BD. / Instrukcie získate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD.

INTENDED USE

BD Affirm™ VPIII Ambient Temperature Transport is a sterile ready-to-use system intended for the collection, transport and preservation of vaginal specimens for use only with the **BD Affirm VPIII Microbial Identification Test**. The **BD Affirm VPIII Ambient Temperature Transport System (ATTS)** should be used with those specimens where transport times are expected to exceed 1 h at ambient temperature (15–30 °C) or 4 h at refrigerated temperatures (2–8 °C).

SUMMARY AND EXPLANATION

One of the routine procedures in the use of the **BD Affirm VPIII Microbial Identification Test** involves the collection and stable transportation of a vaginal specimen from the patient to the laboratory. This can be accomplished using the **BD Affirm VPIII Ambient Temperature Transport System**.

The **BD Affirm VPIII Ambient Temperature Transport System** is designed to stabilize the nucleic acid of *Candida* species, *Gardnerella vaginalis* and *Trichomonas vaginalis* during specimen transport at ambient temperature (15–30 °C) for up to 72 h. This system has also been qualified for transport use at 2–8 °C. The ATTS Reagent Dropper contents are dispensed into the Sample Collection Tube. The patient vaginal specimen is immediately collected and the sterile swab containing the patient specimen is placed in the Sample Collection Tube containing the ATTS Reagent. The swab shaft is broken at the scored point and the top portion of the swab is discarded. The Sample Collection Cap is placed over the exposed end of the swab and is firmly seated on the Sample Collection Tube. The capped Sample Collection Tube is sealed in the outer bag of the **BD Affirm VPIII Ambient Temperature Transport System** and transported to the laboratory within 72 h.

PRINCIPLES OF THE PROCEDURE

When a vaginal specimen is collected and processing is delayed past the recommended time frame, changes in the specimen may result in erroneous **BD Affirm VPIII** test results. The formulation of the ATTS Reagent and the design of the transport system allows preservation of the nucleic acids and inhibits growth of *Candida* sp., *Gardnerella vaginalis* and *Trichomonas vaginalis* from patient specimens for up to 72 h under ambient as well as refrigerated conditions.

REAGENTS

Warnings and Precautions:

For *in vitro* Diagnostic Use.

Specimens collected for transport with this procedure should be used only for testing with the **BD Affirm VPIII Microbial Identification Test**. This transport system was designed for use with those specimens where transport times are expected to exceed 1 h at ambient temperature or 4 h at refrigerated temperatures.

Patient specimen must be collected immediately after the ATTS Reagent has been dispensed into the Sample Collection Tube. **DO NOT** pre-dispense ATTS Reagent into the Sample Collection Tube.

ATTS REAGENT:

DANGER

H225 Highly flammable liquid and vapour. **H311+H331** Toxic in contact with skin or if inhaled. **H370** Causes damage to organs.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P241** Use explosion-proof electrical/ventilating/lighting/equipment. **P260** Do not breathe dust/fume/gas/mist/vapours/spray. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P240** Ground/bond container and receiving equipment. **P242** Use only non-sparking tools. **P243** Take precautionary measures against static discharge. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.

P321 Specific treatment (see on this label). **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P312** Call a POISON CENTER or doctor/physician if you feel unwell. **P370+P378** In case of fire: Use for extinction: CO₂, powder or water spray. **P361+P364** Take off immediately all contaminated clothing and wash it before reuse. **P405** Store locked up. **P403+P233** Store in a well-ventilated place. Keep container tightly closed. **P403+P235** Store in a well-ventilated place. Keep cool. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

WARNING: Pathogenic microorganisms, including hepatitis viruses and Human Immunodeficiency Virus, may be present in clinical specimens. "Standard Precautions"¹⁻⁴ and institutional guidelines should be followed in handling all items contaminated with blood and other body fluids.

Proper handling and disposal methods should be established. Wipe up any spillage of patient specimens immediately and disinfect with an appropriate disinfectant. Treat the cleaning materials as biohazardous waste.

The sterile swab should not be used if the packaging is opened or damaged.

③ **BD Affirm** VPIII Ambient Temperature Transport System is for single use only; reuse may cause a risk of infection and/or inaccurate results.

The Ambient Temperature Transport System should not be used if its foil pouch is open or the ATTS Reagent Dropper is damaged.

When specimen transport times exceed 72 h, a reduction in sensitivity may be observed.

Preparation and Storage of materials:

All materials are supplied ready for use. Store at ambient temperature (15–30 °C).

Keep away from heat – contents are flammable (see "Warnings and Precautions").

PROCEDURES

NOTE: READ ALL INSTRUCTIONS CAREFULLY BEFORE PROCEEDING.

Materials Provided:

1. Instructions for Sample Collection, 1 package insert.
2. **BD Affirm**™ VPIII Ambient Temperature Transport System, 10 Test Kit and 10 x 10 Test Kit (446255).
Each plastic pouch in the **BD Affirm** VPIII Ambient Temperature Transport System contains what is required to collect one vaginal specimen for use only with the **BD Affirm** VPIII Microbial Identification Test. Each **BD Affirm** VPIII Ambient Temperature Transport System includes:
 - Individually wrapped, pre-scored, sterile swab
 - Sample Collection Tube
 - Sample Collection Cap
 - ATTS Reagent Dropper (in foil pouch).
3. Patient Specimen Labels, 24 labels (10 Test Kit) and 240 labels (10 x 10 Test Kit) (446255).
4. ATTS Reagent Dropper Tool, 1 Tool (10 Test Kit) and 10 Tools (10x10 Test Kit) (446255).

Materials Required But Not Provided:

BD Affirm VPIII Microbial Identification Test, **BD MicroProbe** Processor with Lysis Block.

Preparation of Transport System and Collection of Vaginal Sample –

Sample Collection is a critical step. Personnel collecting vaginal fluid specimens for transport with this system should be well trained on adequate sample collection *technique*. All samples for use with the **BD Affirm** VPIII Microbial Identification Test must be collected using the materials provided with this system.

Refer to Procedural Chart Illustrations, Inside Back Cover

1. Open the seal on outer plastic pouch of **BD Affirm** VPIII Ambient Temperature Transport System and remove all components [each plastic pouch contains enough material for the collection and transport of one vaginal specimen].
2. Tear open the foil pouch and remove the ATTS Reagent Dropper.
3. Invert the ATTS Reagent Dropper over the Sample Collection Tube holding the dropper at the base of the cap.
4. Center the teeth of the ATTS Reagent Dropper Dispensing Tool at the center of the inverted ATTS Reagent Dropper.
5. Lightly squeeze the ATTS Reagent Dropper Dispensing Tool to break the ampule in the ATTS Reagent Dropper.
6. Release pressure on the ATTS Reagent Dropper Dispensing Tool while continuing to hold the dropper at the base of the cap.
7. Firmly squeeze the ATTS Reagent Dropper Dispensing Tool, **ONE TIME ONLY**, to dispense reagent from the ATTS Reagent Dropper into Sample Collection Tube.
8. Peel wrapper to expose patient swab. Remove swab. Discard wrapper.
9. Collect patient specimen/take sample.
 - Place the patient in position for a pelvic examination. Insert speculum into the vagina to permit visualization of the posterior vaginal fornix.*
 - Using the sterile swab, obtain a sample from the posterior vaginal fornix. Twist or roll the swab against the vaginal wall two or three times, ensuring the entire circumference of the swab has touched the vaginal wall. Swab the lateral vaginal wall while removing the swab.
10. Immediately place the patient swab in the Sample Collection Tube containing the ATTS Reagent.
11. Break swab shaft at pre-scored line just above the top of the tube. Discard remaining shaft into an infectious waste container.
12. Place the Sample Collection Cap over the exposed end of the swab and firmly press the cap onto the Sample Collection Tube. The cap will 'snap' onto the tube when it is properly seated.
13. Label the Sample Collection Tube with patient/lab identification information. Include date and time that sample was taken.

*During clinical trials, sites were provided with instructions to use an unlubricated speculum. In **BD Affirm** VPIII System analytical studies, there was no evidence of interference with water-based vaginal lubricant.

Transport and Storage of Vaginal Sample:

Send the capped Sample Collection Tube to the lab by sealing it in the emptied outer plastic pouch of the **BD Affirm** VPIII Ambient Temperature Transport System. Label plastic pouch with appropriate information.

LIMITATIONS OF THE PROCEDURE

The **BD Affirm** VPIII sample collection and specimen processing procedure has been designed for use with vaginal specimens that are only to be tested using the **BD Affirm** VPIII Microbial Identification Test. The use of this procedure with other specimen types has not been established.

The use of this product in a manner not specified above has not been established.

Separate swabs should be used for other tests, e.g. culture or microscopic slide samples. This kit is not intended for collecting culture specimens.

Swabs other than the ones specified for this system must not be used with the **BD Affirm** VPIII Ambient Temperature Transport System.

This system has been qualified for use at ambient conditions (15–30 °C) for up to 72 h transport time. The system has also been qualified for transport use at 2–8 °C for up to 72 h transport time.

When specimen transport times exceed 72 h, a reduction in sensitivity may be observed.

Patient specimen must be collected immediately after the ATTS Reagent has been dispensed into the Sample Collection Tube. **DO NOT** pre-dispense the ATTS Reagent into the Specimen Transport Tube.

PERFORMANCE CHARACTERISTICS

Performance characteristics for the **BD Affirm** VPIII Ambient Temperature Transport System (ATTS) with the **BD Affirm** VPIII System for the detection of *Candida* species, *Gardnerella vaginalis* and *Trichomonas vaginalis* were established in a multi-center study at three geographically diverse clinical sites. The study included 940 multi-swab specimens collected from female subjects suspected of vaginitis or bacterial vaginosis. Each specimen consisted of four simultaneously collected vaginal swabs. One of the swabs was transported in the **BD Affirm** VPIII Sample Collection set (control). The three remaining swabs were each transported in the **BD Affirm** VPIII Ambient Temperature Transport System (ATTS).

The control swab was tested using the **BD Affirm** VPIII System within one hour when stored at room temperature or within 4 h when refrigerated (i.e., time 0 - control). ATTS swabs were tested at time 0, 24 and/or 48 and/or 72 h.

The positivity rate of the control swabs compared to the ATTS swabs was calculated at each time point. The positivity rate for swabs transported with the ATTS swabs was within 10% of the positivity rate of the control swabs for up to 72 h.¹

AVAILABILITY

Cat. No.	Description
446255	BD Affirm ™ VPIII Ambient Temperature Transport System, 10 Test Kit and 10 x 10 Test Kit.
446252	BD Affirm ™ VPIII Microbial Identification Test, 24 Tests.
446257	BD Affirm ™ VPIII Microbial Identification Test, 120 Tests.
446251	BD Affirm ™ VPIII Bulk Sample Collection Swabs, 100 swabs.
250100	BD MicroProbe ™ Processor (120 V).
211918	BD MicroProbe ™ Processor (220/240 V).
446250	BD Affirm ™ VPIII Sample Collection Sets, 24 sets.

REFERENCES

1. Clinical and Laboratory Standards Institute. 2005. Approved Guideline M29-A3. Protection of laboratory workers from occupationally acquired infections, 3rd ed. CLSI, Wayne, Pa.
2. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53-80.
3. U.S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 5th ed. U.S. Government Printing Office, Washington, D.C.
4. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021-0045.

Technical Information: In the United States, contact BD Technical Service and Support: 1.800.638.8663 or www.bd.com.

BD Affirm VPIII Ambient Temperature Transport System

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Benex Limited