 **BD ProbeTec™ ET *Chlamydia trachomatis*/
Neisseria gonorrhoeae (CT/GC) Amplified DNA
Assay Endocervical Specimen Collection and
DRY TRANSPORT Kit**

English: pages 1 – 2 Italiano: pagine 6 – 7
Dansk: side 2 – 3 Español: páginas 7 – 8
Français: pages 3 – 4 Svenska: sidan 8 – 9
Deutsch: seiten 4 – 5

2797

L008315(05)
2019-12

R_x Only

Europe: Medical Device
93/42/EEC

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokyny vám poskytne místní zástupce společnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteyts lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőjétől. / Нұсқаулар үшін жергілікті BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliojoto atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielstwem BD. / Contacte o representante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указаний обратитесь к местному представителю компании BD. / Instrukcie získate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD.

INTENDED USE

The **BD ProbeTec™ ET *Chlamydia trachomatis*/*Neisseria gonorrhoeae* (CT/GC) Amplified DNA Assay Endocervical Specimen Collection and DRY TRANSPORT Kit** offers a convenient way to collect and transport specimens from the patient to the laboratory. It is designed for the cleaning and collection of endocervical specimens. This transport system is for use with the **BD ProbeTec ET *Chlamydia trachomatis* and *Neisseria gonorrhoeae* Amplified DNA assays on the BD ProbeTec ET System.**

SUMMARY AND EXPLANATION

The **BD ProbeTec ET *Chlamydia trachomatis*/*Neisseria gonorrhoeae* (CT/GC) Amplified DNA Assay Endocervical Specimen Collection and DRY TRANSPORT Kit** is a sterile, self-contained, disposable system. A large cleaning swab is included to clean the cervical os of blood and mucus prior to collection. The smaller polyurethane-tipped swab on a plastic shaft facilitates obtaining the specimen and maintaining the organisms for nucleic acid amplification on the **BD ProbeTec ET System.**

REAGENTS

Materials Provided: The **BD ProbeTec ET *Chlamydia trachomatis*/*Neisseria gonorrhoeae* (CT/GC) Amplified DNA Assay Endocervical Specimen Collection and DRY TRANSPORT Kit** contains 100 units. Each unit contains one polyurethane-tipped swab in a tube and a large rayon-tipped cleaning swab.

Materials Required But Not Provided: Gloves.

Storage Instructions: Store swabs at 15–30 °C. Do not use past the expiration date. Product sterility guaranteed if primary container is intact.

Precautions: For *in vitro* Diagnostic Use

Warnings:

1. Pathogenic microorganisms, including hepatitis viruses and Human Immunodeficiency Virus, may be present in clinical specimens. "Standard Precautions" ¹⁻⁴ and institutional guidelines should be followed in handling all items contaminated with blood and other body fluids.
2. Optimal performance of the **BD ProbeTec ET** amplified DNA assays requires proper specimen collection, handling, and transport.
3. Use only collection and transport systems labeled for use with the **BD ProbeTec ET *Chlamydia trachomatis* and *Neisseria gonorrhoeae* Amplified DNA Assays.**
4. The large swab is for cleaning the cervical os and should not be used for collecting the patient specimen.
5. Proper labeling should accompany each specimen to the laboratory.
6.  **BD ProbeTec ET *Chlamydia trachomatis*/*Neisseria gonorrhoeae* (CT/GC) Amplified DNA Assay Endocervical Specimen Collection and DRY TRANSPORT Kit** is for single use only; reuse may cause a risk of infection and/or inaccurate results.

SPECIMEN COLLECTION AND TRANSPORT

Endocervical Swab Specimen Collection and Transport

1. Remove excess mucus from the cervical os with the large-tipped cleaning swab provided in the **BD ProbeTec ET CT/GC Amplified DNA Assay Endocervical Specimen Collection and DRY TRANSPORT Kit** and discard.
2. Insert the endocervical specimen collection swab into the cervical canal and rotate for 15–30 s.
3. Withdraw the swab carefully. Avoid contact with the vaginal mucosa.
4. Immediately place the cap/swab into the transport tube. Make sure the cap is tightly secured to the tube.
5. Label the tube with patient information and date/time collected.

Swab Storage and Transport

The endocervical specimen collection swab must be stored and transported to the laboratory and/or test site at 2–27 °C within 4–6 days of collection. Storage up to 4 days has been validated with clinical specimens; storage up to 6 days has been demonstrated with seeded specimens. In addition, storage up to 30 days at 2–8 °C has been demonstrated with seeded specimens.

NOTE: If specimens cannot be transported directly to the testing laboratory under ambient temperatures (15–27 °C) and must be shipped, an insulated container with ice should be used with either an overnight or 2-day delivery vendor.

For domestic and international shipments, specimens should be labeled in compliance with applicable state, federal, and international regulations covering the transport of clinical specimens and etiologic agents/infectious substances. Time and temperature conditions for storage must be maintained during transport.

REFERENCES

1. Clinical and Laboratory Standards Institute. 2005. Approved Guideline M29-A3. Protection of laboratory workers from occupationally acquired infections, 3rd ed. CLSI, Wayne, PA.
2. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. *Infect. Control Hospital Epidemiol.* 17:53-80.
3. U.S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 5th ed. U.S. Government Printing Office, Washington, D.C.
4. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). *Official Journal L262*, 17/10/2000, p. 0021-0045.

Technical Information: In the United States contact BD Technical Service and Support at 800.638.8663 or bd.com.

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US Customers only: For symbol glossary, refer to bd.com/symbols-glossary

Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbrikante / Аткарушы / 제조업체 / Gamintojas / Ražotājs / Tilvirker / Producent / Producător / Производител / Výrobca / Proizvođač / Tillverkare / Üretici / Виробник / 生产厂商

Use by / Исполн. до / Spotføjbedje do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / 사용 기한 / Upotrebiti do / Felhasználhatóság dátuma / Usare entro / Дейін пайдалануға / Naudokite iki / Izlietot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Исползовать до / Použite do / Upotrebiti do / Använd före / Son kullanna tarhi / Використати долине / 使用截止日期

YYYY-MM-DD / YYYY-MM (MM = end of month)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = slutning af måned)
JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)
EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)
AAAA-MM-DD / AAAA-MM (MM = fin del mes)
AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)
AAAA-MM-JJ / AAAA-MM (MM = fin du mois)
GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)
EEEE-HH-NN / EEEE-HH (HH = hónap utolsó napja)
AAAA-MM-GG / AAAA-MM (MM = fine mese)
ЖОЖОЖ-АА-КК / ЖОЖОЖ-АА / (АА = айдың соңы)
YYYY-MM-DD/YYYY-MM (MM = 월말)
MMMM-MM-DD / MMMM-MM (MM = mēnesis pabeiga)
GGGG-MM-DD/GGGG-MM (MM = mēneša beigas)
JJJJ-MM-DD / JJJJ-MM (MM = einde maand)
AAAA-MM-DD / AAAA-MM (MM = sluttan av måneden)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = fim do mês)
AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
GGGG-MM-DD / GGGG-MM (MM = kraj meseca)
AAAA-MM-DD / AAAA-MM (MM = slutet av månaden)
YYYY-AA-GG / YYYY-AA (AA = ayın sonu)
PPPP-MM-DD / PPPP-MM (MM = кінець місяця)
YYYY-MM-DD / YYYY-MM (MM = 月末)

Catalog number / Каталоген номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalognummer / Numéro catalogue / Kataloški broj / Katalógusszám / Numero di catalogo / Каталог номери / 카탈로그 번호 / Katalogo / numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalógové číslo / Kataloški broj / Katalog numarası / Номер за каталогом / 目录号

EC REP

Authorized Representative in the European Community / Оторизиран представител в Европейската общност / Autorizovaný zástupce pro Evropskou společnost / Autoriseret representant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatud esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizuirani predstavnik u Europi / Meghatalmazott képviselő az Európai Közösségben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті екін / 유럽 공동체의 위임 대표 / Igalotaisis atstovas Europos Bendrijoje / Pilsvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autoriseret representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Représentant autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupca v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Autoriserad representant i Europeiska gemenskapen / Avrupa Topluluğu Yetkili Temsilcisi / Уповноважений представник у країнах ЄС / 欧洲共同体授权代表

IVD

In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiinalaparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnostiskati orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жұмыс істейтін медициналық диагностика аспабы / In Vitro Diagnostic 의기 기구 / In vitro diagnostikos prietaisais / Medicinas ierices, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medicinsk udstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispositiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики in vitro / Medicinska pomôcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diagnostic Tibbi Cihaz / Медицинский прибор для диагностики in vitro / 体外诊断医疗设备

Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrænsning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperaturiir piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температурны шектеу / 온도 제한 / Laikymo temperatūra / Temperaturatvorožjumi / Temperaturilimiet / Temperaturbegrænsning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ohrančenje toploty / Ograničenje temperature / Temperaturgräns / Sicaklık sınırlaması / Обмеження температури / 温度限制

LOT

Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-code (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotti) / Toplama kodu / 배치 코드(로트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot number / Batch-code (party) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serie / Partinummer (Lot) / Partii Kodu (Lot) / Код партии / 批号 (亚批)

Contains sufficient for <n> tests / Содержит достаточно для <n> теста / Dostatečné množstvo pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Kifüldalane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenido suficiente per <n> test / <n> тесттегі үшін жеткілікті / <n> 테스트가 충분됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor <n> testen / Innholder tilstrækkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Continut suficient pentru <n> teste / Достаточно для <n> тестов(a) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli malmaze içerir / Вистачить для аналізу: <n> / 足够检测 <n> 次检测

Consult Instructions for Use / Направление справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нускаўняў танысы апына / 사용 지침 참조 / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i brugsanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se brugsanvisningen / Kullaniin Talimatları na başvurun / Див. інструкції з використання / 请参阅使用说明

Do not reuse / Не используйте повторно / Neponužijajte opakovaně / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / Non reutilizar / Mitte kasutada korduvalt / Ne pas réutiliser / Ne koristiti ponovo / Egyszer használatos / Non riutilizzare / Пайдаланбаңыз / 재사용 금지 / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolositi / Ne использовать повторно / Neponužijajte opakovane / Ne upotrebļavajate ponovo / Får ej återanvändas / Tekrar kulanmayın / Не використовувати повторно / 请勿重复使用

SN

Serial number / Serien номер / Sériové číslo / Serienummer / Seriennummer / Σειριακός αριθμός / N° de serie / Seerianumber / Numéro de série / Serijski broj / Sorozatszám / Numero di serie / Toplamalıklı nemi / Serijos numeris / Sérijas numurs / Serie number / Numer serijny / Número de série / Număr de serie / Серийный номер / Serii numarası / Номер серий / 序列号

For IVD Performance evaluation only / Само за оценка качества на работа на IVD / Pouze pro vyhodnocení výkonu IVD / Kun til evaluering af IVD ydelse / Nur für IVD-Leistungsbewertungszwecke / Μόνο για αξιολόγηση απόδοσης IVD / Sólo para la evaluación del rendimiento en diagnóstico in vitro / Ainult IVD seadme hindamiseks / Réserve à l'évaluation des performances IVD / Samo u znanstvene svrhe za In Vitro Dijagnostiku / Kizárólag in vitro diagnosztikához / Solo per valutazione delle prestazioni IVD / Жасанды жағдайда «пробирка ішінде» диагностикада тек жұмысты бағалау үшін / IVD 성능 평가에 대해서만 사용 / Tik IVD prietaisų veikimo charakteristikoms tikrinti / Vientigi IVD darbības novērtēšanai / Uitsluitend voor doeltreffendheidsonderzoek / Kun for evaluering af IVD-ydelse / Tyklo ko oenu vydanosti IVD / Uso exclusivo para avaliação de IVD / Numai pentru evaluarea performanței IVD / Только для оценки качества диагностики in vitro / Určeno iba na diagnostiku in vitro / Samo za procenu učinka u in vitro dijagnostici / Endast för utvärdering av diagnostisk användning in vitro / Valnizca IVD Performans degerlendirilmesi için / Tilkini для оцінювання якості діагностики in vitro / 仅限 IVD 性能评估

For US: "For Investigational Use Only"

Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperaturiiripiir / Limite inférieure de température / Najnižia dovoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Temperatuuratvorožjumi / Nedre temperaturgrænse / Dolna granica temperatury / Limite minimo de temperatura / Limita minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sicaklık alt sınırı / Мінімальна температура / 温度下限

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R_x Only

This only applies to US: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / S'applique uniquement aux États-Unis: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Vale solo per gli Stati Uniti: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Gilt nur für die USA: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Sólo se aplica a los EE.UU.: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner."

Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Becton Dickinson France S.A.S.
11 rue Aristide Bergès
38800 Le Pont de Claix, France

Australian Sponsor:
Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

BD, the BD Logo and ProbeTec are trademarks of Becton, Dickinson and Company or its affiliates. © 2019 BD.
All rights reserved.